1. Dr. Quinn creates the test.

2. Click on the tab "Tests" and then Click on "Create Test"

SAM Student Auscultation Manikin					Welcome CHSimulation Account Settings  About   Help   Sign Out			
Exam Room	Lessons	Assignments	Quizzes	Tests	Course Plan	Teachers	Classes	Stugents Arrhythmia
Test: Sam online 2020 unit 5 Remediation 🗸 Create Test								
								·

4. Following screen will pop up ( Get the test instructions; weight, fail point, time and due date from Dr.Quinn)

Add a new test	×	
Test Title:		
Test Instruction:	♥ ji Size default ♥ ji 数	[
Weight:		
Fill out all sections and click "Add" Click on "Add Question"		·
est: test 🗸		Create Te
<b>test</b> Instruction: test Weight: 100 Test Time: 100 min Fail Point: 78 Due: 07/15/2021 Disabled		Edit
Questions:	Import Que	stions Add Question

- 7. Write the question in the "Question Text" Section
- 8. Write down the choices

9. Choose the sound associated with the question from the "Play Sound" drop down menu and check off the box next to "Play Sound"

10. Put the right answer in the "Correct Answer" Section

Create/update a que	estion and its answers				
Question Text:	A 73-year-old woman with a history of heart failure is admitted for increasing pedal edema and dyspnea. She also describes symptoms of orthopnea and paroxysmal nocturnal dyspnea. She has been told she has an enlarged heart. As				
Question Type:	Multiple Choice Ver 2.mp3				
Choice A:	Clear to auscultation bilaterally				
Choice B:	Coarse rales bilaterally				
Choice C:	Pleural friction rub				
Choice D:	Wheezing in all lung fields				
Choice E:					
Correct Answer:	В				
Answer Explanation:					

## 11. Do the same to add all questions

12.Click on "Enable" right before starting the exam.

test	Edit
Instruction: test	Delete
Weight: 100 Test Time: 100 min Fail Point: 78 Due: 07/17/2021 Disabled Enable	

## 13. Click on "Disable" after exam is done.

test Instruction: test Weight: 100 Test Time: 100 min Fail Point: 78 Due: 07/17/2021 Enabled Disable	Edit Delete