



LABORATORY INFORMATION SYSTEM

SPECIMEN CONTAINER INFORAMTION

January 5, 2010



GENERAL LAB

	Primary Synonym	Specimen Type	Accession Class	Collection Method	Service Resource	Age From	Age To	Coll Priority	Min Vol	Container	Collection Class	Special Handling
1	11 Desoxycortisol	Blood	Chem Sendout Frozen	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	4.0	Red	Ref Frozen	Freeze
2	17 Hydroxycorticosteroids	Urine	Urine Boric	Urine	(ALL)	0 Minutes	150 Years	(ALL)	100.0	U24H Boric	Ref Refrig	Refrigerate
3	17 Hydroxyprogesterone	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.5	Red	Ref Refrig	Refrigerate
4	17 OHPregnenolone	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Red	Ref Refrig	Refrigerate
5	1ST Trimester SCR,H-HCG	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	2.0	SST	Ref Refrig	Refrigerate
6	21-Hydroxylase Ab	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Red	Ref Refrig	Refrigerate
7	5 HIAA	Urine	Urine Boric	Urine	(ALL)	0 Minutes	150 Years	(ALL)	100.0	U24H Boric	Ref Refrig	Refrigerate
8	5 Nucleotidase	Blood	Chem Sendout Frozen	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	5.0	SST	Ref Frozen	Freeze
9	A1 Antitrypsin, 24Hr Feces	Stool	Chem Sendout Frozen	Stool	(ALL)	0 Minutes	150 Years	(ALL)	2.0	Stool Container	Ref Frozen	Freeze
10	ABLHB	Blood	Blood Gases Arterial	Syringe	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Hep Syringe	Chem	Deliver Immediately
11	Acetaminophen	Blood	Blood	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)	0.3	SST (micro)	Chem	
12	Acetaminophen	Blood	Blood	Venous Draw	(ALL)	1 Months	150 Years	(ALL)	0.5	SST	Chem	Draw prior to next dose
13	Acetone Serum	Blood	Blood	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.3	SST	Chem	
14	Acetylcholine Receptor Antibodies	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	2.0	SST	Ref Refrig	Refrigerate
15	Acute Hepatitis Panel	Blood	Blood	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	SST	Chem	
16	Acylcarnitine, Plasma (NY)	Blood	Chem Sendout Frozen	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	10.0	Green	Ref Frozen	Freeze
17	Adenosine Deamin,Pleural	Pleural Fluid	Chem Sendout Frozen	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Sterile T	Ref Frozen	Freeze
18	Adenosine Deaminase, Fluid	CSF	CSF	Syringe	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Sterile Screw	Ref Refrig	Refrigerate
19	Adenovirus Group Antibodies	Blood	Chem Sendout	Venous Draw	(ALL)	6 Months	150 Years	(ALL)	2.0	Red	Ref Refrig	Refrigerate
20	Adenovirus Group Antibodies	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	6 Months	(ALL)	1.0	Red (micro)	Ref Refrig	Refrigerate
21	Adrenal Ab W/RFX	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	2.0	Red	Ref Refrig	Refrigerate
22	Adrenocorticotrophic Hormone	Blood	Chem lav sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	2.0	Lav	Ref Frozen	Freeze
23	AFP Amniotic Fluid	Amniotic Fluid	Body Fluid	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)	3.0	Sterile T	Chem	
24	Albumin	Blood	Blood	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)	0.2	SST (micro)	Chem	
25	Albumin	Blood	Blood	Venous Draw	(ALL)	1 Months	150 Years	(ALL)	0.3	SST	Chem	
26	Albumin Pericardial Fluid	Pericardial Fluid	Body Fluid	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)	0.3	Red	Chem	
27	Albumin Peritoneal Fluid	Peritoneal Fluid	Body Fluid	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)	0.3	Red	Chem	
28	Albumin Pleural Fluid	Pleural Fluid	Body Fluid	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)	0.3	Red	Chem	
29	Aldolase	Blood	Chem Sendout Frozen	Venous Draw	(ALL)	0 Minutes	6 Months	(ALL)	1.0	SST (micro)	Ref Frozen	Freeze
30	Aldolase	Blood	Chem Sendout Frozen	Venous Draw	(ALL)	6 Months	150 Years	(ALL)	4.0	SST	Ref Frozen	Freeze
31	Aldosterone	Blood	Chem Sendout	Venous Draw	(ALL)	6 Months	150 Years	(ALL)	5.0	Red	Ref Refrig	Refrigerate
32	Aldosterone	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	6 Months	(ALL)	1.5	Red (micro)	Ref Refrig	Refrigerate
33	Aldosterone 24hr Urine	Urine	Urine Boric	Urine	(ALL)	0 Minutes	150 Years	(ALL)	100.0	U24H Boric	Ref Refrig	Refrigerate
34	Alk Phos Isoenzymes	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	4.0	Red	Ref Refrig	Refrigerate
35	Alkaline Phosphatase	Blood	Blood	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)	0.2	SST (micro)	Chem	
36	Alkaline Phosphatase	Blood	Blood	Venous Draw	(ALL)	1 Months	150 Years	(ALL)	0.3	SST	Chem	
37	Allergen Almond	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.3	SST	Immune	
38	Allergen Alternaria	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.2	SST	Immune	
39	Allergen Apple	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.3	SST	Immune	
40	Allergen Barley	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.3	SST	Immune	
41	Allergen Beech Tree	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.3	SST	Immune	
42	Allergen Beef	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.3	SST	Immune	
43	Allergen Bermuda Grass	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.3	SST	Immune	
44	Allergen Birch Tree	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.3	SST	Immune	
45	Allergen Box Elder	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.3	SST	Immune	
46	Allergen Cacao	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.3	SST	Immune	
47	Allergen Candida Albicans	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.3	SST	Immune	
48	Allergen Casein	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.3	SST	Immune	
49	Allergen Casein Peds	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	12 Years	(ALL)	2.0	SST	Immune	
50	Allergen Cashew Nut	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.3	SST	Immune	
51	Allergen Cat Epithelium	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.3	SST	Immune	
52	Allergen Cheddar Cheese	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.3	SST	Immune	
53	Allergen Chicken	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.2	SST	Immune	
54	Allergen Clado Herb	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.3	SST	Immune	
55	Allergen Cockroach	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.3	SST	Immune	
56	Allergen Coconut	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.3	SST	Immune	

	Primary Synonym	Specimen Type	Accession Class	Collection Method	Service Resource	Age From	Age To	Coll Priority	Min Vol	Container	Collection Class	Special Handling
57	Allergen Cod Fish	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.3	SST	Immune	
58	Allergen Common Ragweed	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.3	SST	Immune	
59	Allergen Corn	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.3	SST	Immune	
60	Allergen Crab	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.3	SST	Immune	
61	Allergen Dermato Far	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.3	SST	Immune	
62	Allergen Dermato Pter	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.3	SST	Immune	
63	Allergen Dog Dander	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.3	SST	Immune	
64	Allergen Dog Epithelium	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.3	SST	Immune	
65	Allergen Egg White	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.3	SST	Immune	
66	Allergen Egg White Peds	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	12 Years	(ALL)	2.0	SST	Immune	
67	Allergen Egg Yolk	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.3	SST	Immune	
68	Allergen Egg Yolk Peds	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	12 Years	(ALL)	2.0	SST	Immune	
69	Allergen Elm Tree	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.2	SST	Immune	
70	Allergen English Plantain	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.3	SST	Immune	
71	Allergen Giant Ragweed	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.3	SST	Immune	
72	Allergen Grape	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.3	SST	Immune	
73	Allergen Hazelnut	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.3	SST	Immune	
74	Allergen Horse Dander	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.3	SST	Immune	
75	Allergen House Dust	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.3	SST	Immune	
76	Allergen June Grass	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.3	SST	Immune	
77	Allergen Latex	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.3	SST	Immune	
78	Allergen Melon	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.3	SST	Immune	
79	Allergen Milk	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.3	SST	Immune	
80	Allergen Milk Peds	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	12 Years	(ALL)	2.0	SST	Immune	
81	Allergen Mouse Epithelium	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.3	SST	Immune	
82	Allergen Oak Tree	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.3	SST	Immune	
83	Allergen Oats	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.3	SST	Immune	
84	Allergen Oats Peds	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	12 Years	(ALL)	2.0	SST	Immune	
85	Allergen Orange	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.3	SST	Immune	
86	Allergen Orchard Grass	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.3	SST	Immune	
87	Allergen Parrot Feathers	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.3	SST	Immune	
88	Allergen Peanut	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.3	SST	Immune	
89	Allergen Peanuts Peds	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	12 Years	(ALL)	2.0	SST	Immune	
90	Allergen Pear	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.3	SST	Immune	
91	Allergen Peas	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.3	SST	Immune	
92	Allergen Pecan	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.3	SST	Immune	
93	Allergen Penicillium Notatum	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.3	SST	Immune	
94	Allergen Pineapple	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.3	SST	Immune	
95	Allergen Pistachio	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.3	SST	Immune	
96	Allergen Pork	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.3	SST	Immune	
97	Allergen Rabbit Epithelium	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.3	SST	Immune	
98	Allergen Rat Epithelium	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.3	SST	Immune	
99	Allergen Rice	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.3	SST	Immune	
100	Allergen Rice	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.3	SST	Immune	
101	Allergen Rye	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.3	SST	Immune	
102	Allergen Sesame	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.3	SST	Immune	
103	Allergen Shrimp	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.3	SST	Immune	
104	Allergen Shrimp Peds	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	12 Years	(ALL)	2.0	SST	Immune	
105	Allergen Soybean	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.3	SST	Immune	
106	Allergen Soybean Peds	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	12 Years	(ALL)	2.0	SST	Immune	
107	Allergen Spinach	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.3	SST	Immune	
108	Allergen Strawberry	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.3	SST	Immune	
109	Allergen Sycamore Tree	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.3	SST	Immune	
110	Allergen Timothy Grass	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.3	SST	Immune	
111	Allergen Tomato	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.3	SST	Immune	
112	Allergen Tuna	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.3	SST	Immune	

	Primary Synonym	Specimen Type	Accession Class	Collection Method	Service Resource	Age From	Age To	Coll Priority	Min Vol	Container	Collection Class	Special Handling
113	Allergen Walnut	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.3	SST	Immune	
114	Allergen Wheat	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.3	SST	Immune	
115	Allergen Wheat Peds	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	12 Years	(ALL)	2.0	SST	Immune	
116	Allergen White Bean	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.3	SST	Immune	
117	Allergy Panel Animals	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	5.0	SST	Immune	
118	Allergy Panel Dust Mites	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	5.0	SST	Immune	
119	Allergy Panel Foods	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	5.0	SST	Immune	
120	Allergy Panel Inhalants	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	5.0	SST	Immune	
121	Allergy Panel Molds	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	5.0	SST	Immune	
122	Alpha Fetoprotein Maternal	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	4.0	SST	Ref Refrig	Refrigerate
123	Alpha Fetoprotein Tumor Marker	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.4	Red	Ref Refrig	Refrigerate
124	Alpha Fucosidase A	Blood	Chem Sendout	Venous Draw	(ALL)	1 Minutes	150 Years	(ALL)	7.0	Yellow (ACD)	Ref Refrig	Keep on Ice
125	Alpha Galactosidase A	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	10.0	Green	Ref Room	Room temperate
126	Alpha L Iduronidase	Blood	Blood	Venous Draw	(ALL)	1 Minutes	150 Years	(ALL)	10.0	Green	Chem	Keep on Ice
127	Alpha Subunit	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Red	Ref Refrig	Refrigerate
128	Alpha1 Antitrypsin	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Red	Ref Refrig	Refrigerate
129	Alpha1 Antitrypsin Phenotyping	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Red	Chem	Refrigerate
130	Alpha1 Antitrypsin Phenotyping	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	3.0	Lav	Chem	Refrigerate
131	ALT	Blood	Blood	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)	0.3	SST (micro)	Chem	
132	ALT	Blood	Blood	Venous Draw	(ALL)	1 Months	150 Years	(ALL)	0.3	SST	Chem	
133	Aluminum Serum	Blood	Chem Royal	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)	1.5	Royal (Serum)	Ref Refrig	Refrigerate
134	Aluminum Serum	Blood	Chem Royal	Venous Draw	(ALL)	1 Months	150 Years	(ALL)	5.0	Royal (Serum)	Ref Refrig	Refrigerate
135	Aluminum Urine	Urine	Urine	Urine	(ALL)	0 Minutes	150 Years	(ALL)	20.0	U Aliquot	Ref Refrig	
136	Amikacin Peak	Serum	Peak	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)	0.2	SST (micro)	Chem	
137	Amikacin Peak	Serum	Peak	Venous Draw	(ALL)	1 Months	150 Years	(ALL)	0.2	SST	Chem	
138	Amikacin Random	Blood	Blood	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)	0.3	SST (micro)	Chem	
139	Amikacin Random	Blood	Blood	Venous Draw	(ALL)	1 Months	150 Years	(ALL)	0.3	SST	Chem	
140	Amikacin Trough	Serum	Trough	Venous Draw	(ALL)	1 Months	150 Years	(ALL)	0.3	SST	Chem	
141	Amikacin Trough	Serum	Trough	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)	0.3	SST (micro)	Chem	
142	Amino Acids Quant. Urine	Urine	Urine	Urine	(ALL)	0 Minutes	1 Years	(ALL)	2.0	U Aliquot	Chem	Freeze
143	Amino Acids Quant. Urine	Urine	Urine	Urine	(ALL)	1 Years	150 Years	(ALL)	10.0	U Aliquot	Chem	Freeze
144	Amino Acids Qualitative Plasma	Blood	Blood Heparin	Venous Draw	(ALL)	0 Minutes	7 Months	(ALL)	2.0	Green	Chem	Freeze
145	Amino Acids Qualitative Plasma	Blood	Blood Heparin	Venous Draw	(ALL)	7 Months	150 Years	(ALL)	10.0	Green	Chem	Freeze
146	Amino Acids Qualitative Urine	Urine	Urine	Urine	(ALL)	0 Minutes	150 Years	(ALL)	5.0	UA	Chem	Freeze
147	Amino Acids Quant. Plasma	Blood	Blood Heparin	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	3.0	Green	Chem	Freeze
148	Amiodarone	Blood	Chem Sendout	Venous Draw	(ALL)	1 Months	150 Years	(ALL)	3.0	Red	Ref Refrig	Refrigerate
149	Amiodarone	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)	1.0	Red (micro)	Ref Refrig	Refrigerate
150	Amitriptyline	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	2 Months	(ALL)	1.0	Red (micro)	Ref Room	Room temperate
151	Amitriptyline	Blood	Chem Sendout	Venous Draw	(ALL)	61 Days	150 Years	(ALL)	3.0	Red	Ref Room	Room temperate
152	Ammonia	Blood Heparin	Blood Heparin	Venous Draw	(ALL)	1 Months	150 Years	(ALL)	3.0	Green	Chem	Deliver on ice
153	Ammonia	Blood Heparin	Blood Heparin	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)	1.0	Green	Chem	Deliver on ice
154	Amylase	Blood	Blood	Venous Draw	(ALL)	1 Months	150 Years	(ALL)	0.3	SST	Chem	
155	Amylase	Blood	Blood	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)	0.2	SST (micro)	Chem	
156	Amylase Isoenzymes	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	2.0	Red	Ref Refrig	Refrigerate
157	Amylase Peritoneal Fluid	Peritoneal Fluid	Body Fluid	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)	0.3	Red	Chem	
158	Amylase Pleural Fluid	Pleural Fluid	Body Fluid	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)	0.3	Red	Chem	
159	Amylase Timed Urine	Urine	Urine	Urine	(ALL)	0 Minutes	150 Years	(ALL)	2.0	U Aliquot	Urine Chem	Refrigerate
160	Amylase Urine	Urine	Urine	Urine	(ALL)	0 Minutes	150 Years	(ALL)	2.0	UA	Urine Chem	
161	Androstendione	Blood	Chem Sendout	Venous Draw	(ALL)	6 Months	150 Years	(ALL)	5.0	Red	Ref Refrig	Refrigerate
162	Androstendione	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	6 Months	(ALL)	2.0	Red (micro)	Ref Refrig	Refrigerate
163	Angiotensin Conv Enzy, CSF	CSF	Chem Sendout Frozen	Syringe	(ALL)	0 Minutes	150 Years	(ALL)	0.2	Sterile Screw	Ref Frozen	Freeze
164	Angiotensin Converting Enzyme	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	2.0	SST	Ref Refrig	Refrigerate
165	Anti CCP IgG	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.4	SST	Immune	
166	Anti DNA Antibodies Double Stranded	Blood	Blood	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	SST	Immune	
167	Anti GBM Abs	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	4.0	SST	Ref Refrig	Refrigerate
168	Anti Gliadin Antibodies	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	4.0	Red	Ref Refrig	Refrigerate

	Primary Synonym	Specimen Type	Accession Class	Collection Method	Service Resource	Age From	Age To	Coll Priority	Min Vol	Container	Collection Class	Special Handling
169	Anti GM 1	Blood	Blood	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	5.0	SST	Chem	Freeze
170	Anti JO 1	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	5.0	Red	Ref Refrig	Refrigerate
171	Anti MAG	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	5.0	SST	Ref Refrig	Refrigerate
172	Anti Mitochondrial Antibodies	Blood	Blood	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	SST	Immune	
173	Anti Ribonucleoprotein Antibodies	Blood	Blood	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	SST	Immune	
174	Anti Sjogren Syndrome A Antibodies	Blood	Blood	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	SST	Immune	
175	Anti Sjogren Syndrome B Antibodies	Blood	Blood	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	SST	Immune	
176	Anti Smith Antibodies	Blood	Blood	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	SST	Immune	
177	Anti Smooth Muscle Antibodies	Blood	Blood	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	SST	Immune	
178	Anti-68 KD(HSP-70) Ab	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Red	Ref Refrig	Refrigerate
179	Anticardiolipin Antibodies IgG IgM	Blood	Hem Red	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	3.0	SST	Ref Refrig	Refrigerate
180	Anticardiolipin Antobodies IgM	Blood	Hem Red	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	4.0	SST	Ref Refrig	Refrigerate
181	Antihistone Abs IgG and IgM	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	4.0	Red	Ref Refrig	Refrigerate
182	Antimicrosomal Antibody	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	4.0	SST	Ref Refrig	Refrigerate
183	Anti-Mullerian Hormone	Blood	Chem Sendout Frozen	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Red	Ref Frozen	Freeze
184	Antineutrophil Cytoplasmic Antibodies	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	4.0	SST	Ref Refrig	Refrigerate
185	Antinuclear Antibodies	Blood	Blood	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	SST	Immune	
186	Antiphospholipid Ab Panel	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	5.0	Red	Ref Room	Room temperate
187	Antiscleroderma 70 Antibodies	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	4.0	Red	Ref Room	Room temperate
188	Antistreptolysin O Antibodies	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.3	SST	Ref Refrig	Refrigerate
189	Antistriated Antibodies	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	4.0	SST	Ref Refrig	Refrigerate
190	Antithrombin III Antigen (Immunologic)	Blood	Hem Blue Sendouts	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	5.0	Blue	Ref Frozen	Freeze
191	Antithrombin III Functional	Blood	Blood Coag	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	4.5	Blue	Ref Frozen	Freeze
192	APC Resistance	Blood	Hem Blue Sendouts	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Blue	Ref Frozen	Freeze
193	Apolipoprotein A 1	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	4.0	SST	Ref Refrig	Refrigerate
194	Apolipoprotein B	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	4.0	SST	Ref Refrig	Refrigerate
195	APTT	Blood	Blood Coag	Venous Draw	(ALL)	12 Years	150 Years	(ALL)	0.5	Blue	Coag	Mix Immediately
196	APTT	Blood	Blood Coag	Venous Draw	(ALL)	0 Minutes	12 Years	(ALL)	0.5	Blue (Peds)	Coag	Mix Immediately
197	Arginine Vasopressin	Blood	Chem Sendout Frozen	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	3.0	Lav	Ref Frozen	Freeze
198	Arsenic Exposure Profile Urine	Urine	Urine 24hr	Urine	(ALL)	0 Minutes	150 Years	(ALL)	20.0	U Aliquot	Ref Refrig	
199	Arsenic, Blood	Blood	Chem Royal	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	5.0	Royal (EDTA)	Ref Refrig	Refrigerate
200	Arylsulfatase A	Blood	Chem Sendout	Venous Draw	(ALL)	1 Minutes	150 Years	(ALL)	7.0	Yellow (ACD)	Ref Refrig	Refrigerate
201	Arylsulfatase B	Blood	Chem Sendout	Venous Draw	(ALL)	1 Minutes	150 Years	(ALL)	10.0	Green	Ref Refrig	Refrigerate
202	Aspergillus AB Profile	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	4.0	Red	Ref Refrig	
203	Aspergillus Fumigatus Allergen M3	Blood	Blood	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.3	SST	Immune	
204	AST	Blood	Blood	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)	0.3	SST (micro)	Chem	
205	AST	Blood	Blood	Venous Draw	(ALL)	1 Months	150 Years	(ALL)	0.3	SST	Chem	
206	Automated Diff (Lab Order only)	Blood	Blood Hem	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)	0.2	Lav (micro)	Hem	
207	Automated Diff (Lab Order only)	Blood	Blood Hem	Venous Draw	(ALL)	1 Months	150 Years	(ALL)	1.0	Lav	Hem	
208	B. Pertussis Ab (IgG,A,M)	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Red	Ref Refrig	Refrigerate
209	Barbituates Urine	Urine	Urine	Urine	(ALL)	0 Minutes	150 Years	(ALL)	20.0	U Aliquot	Chem	
210	Barbiturates ID & QN 1	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Red	Ref Refrig	
211	Bartonella AB IGG IGM w RFL	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Red	Ref Refrig	Refrigerate
212	Basic Metabolic Panel	Blood	Blood	Venous Draw	(ALL)	31 Days	150 Years	(ALL)	0.5	SST	Chem	
213	Basic Metabolic Panel	Blood	Blood	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)	0.5	SST (micro)	Chem	
214	BB Director Review	OTHER	Blood Bank	Special Collection	(ALL)	0 Minutes	150 Years	(ALL)	0.1	NoContainer	BldBank	
215	Beta 2 Microglobulin Serum	Blood	Chem Sendout	Venous Draw	(ALL)	6 Months	150 Years	(ALL)	3.0	SST	Ref Refrig	Refrigerate
216	Beta 2 Microglobulin Serum	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	6 Months	(ALL)	1.0	SST (micro)	Ref Refrig	Refrigerate
217	Beta 2 Microglobulin Urine	Urine	Urine	Urine	(ALL)	7 Months	150 Years	(ALL)	10.0	U Aliquot	Ref Refrig	
218	Beta 2 Microglobulin Urine	Urine	Urine	Urine	(ALL)	0 Minutes	6 Months	(ALL)	1.0	U Aliquot	Ref Refrig	
219	Beta Galactocerebrosidase	Blood	Blood	Venous Draw	(ALL)	1 Minutes	150 Years	(ALL)	10.0	Green	Chem	Keep on Ice
220	Beta Galactosidase	Blood	Blood	Venous Draw	(ALL)	1 Minutes	150 Years	(ALL)	7.0	Yellow (ACD)	Ref Refrig	Keep on Ice
221	Beta Glucosidase	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	10.0	Green	Ref Frozen	Freeze
222	Beta Glucuronidase	Blood	Chem Sendout	Venous Draw	(ALL)	1 Minutes	150 Years	(ALL)	10.0	Green	Ref Frozen	Freeze
223	Beta HCG Qualitative	Blood	Julian Date	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.3	SST	Chem	
224	Beta HCG Quantitative	Blood	Chem Architect	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	SST	Chem	

	Primary Synonym	Specimen Type	Accession Class	Collection Method	Service Resource	Age From	Age To	Coll Priority	Min Vol	Container	Collection Class	Special Handling
225	Beta-2, Transferrin	Nasal Fluid	Chem Sendout Frozen	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)	0.5	Sterile Screw	Ref Frozen	Freeze
226	Beta-2, Transferrin	Body Fluid	Body Fluid	Syringe	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Sterile Screw	Ref Frozen	Freeze
227	Bile Acids, Fractionated	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	4.0	SST	Ref Refrig	Refrigerate
228	Bilirubin Amniotic Fluid	Amniotic Fluid	Body Fluid	Special Collection	(ALL)	0 Minutes	150 Years	(ALL)	5.0	Sterile Plastic	Chem	Protect from light
229	Bilirubin Direct	Blood	Blood	Venous Draw	(ALL)	1 Months	150 Years	(ALL)	0.3	SST	Chem	
230	Bilirubin Direct	Blood	Blood	Venous Draw	(ALL)	15 Days	1 Months	(ALL)	0.3	SST (micro)	Chem	
231	Bilirubin Total	Blood	Blood	Venous Draw	(ALL)	0 Minutes	2 Years	(ALL)	0.3	SST (micro)	Chem	
232	Bilirubin Total	Blood	Blood	Venous Draw	(ALL)	2 Years	150 Years	(ALL)	0.3	SST	Chem	
233	Bilirubin Total & Direct	Blood	Blood	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)	0.4	SST (micro)	Chem	
234	Bilirubin Total & Direct	Blood	Blood	Venous Draw	(ALL)	1 Months	150 Years	(ALL)	0.6	SST	Chem	
235	Bilirubin Total Peritoneal Fluid	Peritoneal Fluid	Body Fluid	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)	0.3	Red	Chem	
236	BK Virus DNA, QN PCR	Blood	Chem Sendout Frozen	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Lav	Ref Frozen	Freeze
237	BK Virus DNA, QN PCR	Urine	Chem Sendout Frozen	Urine	(ALL)	0 Minutes	150 Years	(ALL)	5.0	UA	Ref Frozen	Freeze
238	BK Virus DNA, QN PCR	CSF	Chem Sendout Frozen	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Sterile T	Ref Frozen	Freeze
239	BK/JC Virus DNA,QL PCR	Blood	Chem Sendout Frozen	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Lav	Ref Frozen	Freeze
240	BK/JC Virus DNA,QL PCR	CSF	Chem Sendout Frozen	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Sterile T	Ref Frozen	Freeze
241	BK/JC Virus DNA,QL PCR	U CC	Chem Sendout Frozen	Urine	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Sterile Screw	Ref Frozen	Freeze
242	Blood Gases Capillary	Blood Heparin	Blood Gases Capillary	Syringe	(ALL)	0 Minutes	150 Years	(ALL)	0.2	Capillary	Chem	Deliver Immediately
243	Blood Gases Intra Operative	Blood Heparin	Blood Gases Arterial	Syringe	(ALL)	0 Minutes	150 Years	(ALL)	3.0	Hep Syringe	Chem	
244	Blood Gases Arterial	Blood Heparin	Blood Gases Arterial	Syringe	(ALL)	0 Minutes	150 Years	(ALL)	0.1	Hep Syringe	Chem	Deliver Immediately
245	Blood Gases Perfusate	Perfusate	Body Fluid	Syringe	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Hep Syringe	Chem	
246	Blood Gases Venous	Blood Heparin	Blood Gases Venous	Syringe	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Hep Syringe	Chem	Deliver Immediately
247	Bone Marrow Interpretation	Bone Marrow Asp	Blood Hem	Slide	(ALL)	0 Minutes	150 Years	(ALL)	2.0	Slides	Hem	Deliver Immediately
248	Brain Natriuretic Peptide	Blood	Hgb A1c	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.5	Lav	Chem	
249	Brucella Antibodies	Blood	Chem Sendout Frozen	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	4.0	Red	Ref Frozen	Freeze
250	BUN	Blood	Blood	Venous Draw	(ALL)	1 Months	150 Years	(ALL)	0.2	SST	Chem	
251	BUN	Blood	Blood	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)	0.2	SST (micro)	Chem	
252	C Peptide	Blood	Chem Sendout Frozen	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	3.0	SST	Ref Frozen	Freeze
253	C Peptide #1	Blood	Blood	Venous Draw	(ALL)	0 Minutes	6 Months	(ALL)	1.0	SST (micro)	Chem	Freeze
254	C Peptide #1	Blood	Blood	Venous Draw	(ALL)	7 Months	150 Years	(ALL)	5.0	SST	Chem	Freeze
255	C Peptide #2	Blood	Blood	Venous Draw	(ALL)	0 Minutes	6 Months	(ALL)	1.0	SST (micro)	Chem	Freeze
256	C Peptide #2	Blood	Blood	Venous Draw	(ALL)	7 Months	150 Years	(ALL)	5.0	SST	Chem	Freeze
257	C Peptide #3	Blood	Blood	Venous Draw	(ALL)	0 Minutes	6 Months	(ALL)	1.0	SST (micro)	Chem	Freeze
258	C Peptide #3	Blood	Blood	Venous Draw	(ALL)	7 Months	150 Years	(ALL)	5.0	SST	Chem	Freeze
259	C Peptide #4	Blood	Blood	Venous Draw	(ALL)	0 Minutes	6 Months	(ALL)	1.0	SST (micro)	Chem	Freeze
260	C Peptide #4	Blood	Blood	Venous Draw	(ALL)	7 Months	150 Years	(ALL)	5.0	SST	Chem	Freeze
261	C Peptide #5	Blood	Blood	Venous Draw	(ALL)	0 Minutes	6 Months	(ALL)	1.0	SST (micro)	Chem	Freeze
262	C Peptide #5	Blood	Blood	Venous Draw	(ALL)	7 Months	150 Years	(ALL)	5.0	SST	Chem	Freeze
263	C Peptide #6	Blood	Blood	Venous Draw	(ALL)	0 Minutes	6 Months	(ALL)	1.0	SST (micro)	Chem	Freeze
264	C Peptide #6	Blood	Blood	Venous Draw	(ALL)	7 Months	150 Years	(ALL)	5.0	SST	Chem	Freeze
265	C Reactive Protein Serum	Blood	Blood	Venous Draw	(ALL)	0 Minutes	31 Days	(ALL)	0.3	SST (micro)	Chem	
266	C Reactive Protein Serum	Blood	Blood	Venous Draw	(ALL)	31 Days	150 Years	(ALL)	1.0	SST	Chem	
267	C1 Esterase Inhibitor	Blood	Chem Sendout Frozen	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	4.0	SST	Ref Frozen	Freeze
268	C3D Immune Complex	Blood	Chem Sendout Frozen	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Red	Ref Frozen	Freeze
269	Cadmium Blood	Blood	Chem Royal	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	2.0	Royal (EDTA)	Ref Refrig	Refrigerate
270	Cadmium Urine	Urine	Urine	Urine	(ALL)	0 Minutes	150 Years	(ALL)	20.0	U Aliquot	Ref Refrig	
271	Caffeine	Blood	Blood	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	3.0	SST	Chem	
272	Calcitonin	Blood	Chem Sendout Frozen	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	5.0	SST	Ref Frozen	Freeze
273	Calcium	Blood	Blood	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)	0.2	SST (micro)	Chem	
274	Calcium	Blood	Blood	Venous Draw	(ALL)	1 Months	150 Years	(ALL)	0.2	SST	Chem	
275	Calcium Urine Random	Urine	Urine	Urine	(ALL)	0 Minutes	150 Years	(ALL)	2.0	UA	Urine Chem	
276	Calcium Ionized Serum	Blood	Blood	Venous Draw	(ALL)	31 Days	150 Years	(ALL)	0.3	Red	Chem	
277	Calcium Ionized Serum	Blood	Blood	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)	0.3	Red (micro)	Chem	
278	Calcium Ionized Whole Blood	Blood Heparin	Blood Gases Arterial	Syringe	(ALL)	0 Minutes	150 Years	(ALL)	0.1	Hep Syringe	Chem	Deliver Immediately
279	Calcium Urine 24Hr	Urine	Urine 24hr	Urine	(ALL)	0 Minutes	150 Years	(ALL)	2.0	U24H	Urine Chem	Refrigerate
280	Calculi Urinary	Calculi	Urine	Special Collection	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Transport	Chem	Room temperate

	Primary Synonym	Specimen Type	Accession Class	Collection Method	Service Resource	Age From	Age To	Coll Priority	Min Vol	Container	Collection Class	Special Handling
281	Campylobacter Jejuni Ab	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Red	Ref Refrig	Refrigerate
282	Cancer Antigen 125	Blood	Chem E170	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	5.0	SST	Chem	Freeze
283	Cancer Antigen 27 29	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	3.0	SST	Ref Refrig	Refrigerate
284	Candida Antibodies	Blood	Blood	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	4.0	SST	Chem	
285	Carbamazapine	Blood	Blood	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)	0.2	SST (micro)	Chem	
286	Carbamazapine	Blood	Blood	Venous Draw	(ALL)	31 Days	150 Years	(ALL)	0.5	SST	Chem	Draw prior to next dose
287	Carbohydrate Antigen 19 9	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	5.0	SST	Ref Refrig	Refrigerate
288	Carbon Dioxide	Blood	Blood	Venous Draw	(ALL)	1 Months	150 Years	(ALL)	0.2	SST	Chem	
289	Carbon Dioxide	Blood	Blood	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)	0.2	SST (micro)	Chem	
290	Carbon Monoxide	Blood Heparin	Blood Gases Arterial	Arterial Draw	(ALL)	0 Minutes	150 Years	(ALL)	2.0	Hep Syringe	Chem	Deliver Immediately
291	Cardio CRP	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	SST	Ref Refrig	
292	Cardiolipin Ab (IgG)	Blood	Hem Red	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	2.0	SST	Ref Refrig	Refrigerate
293	Cardiolipin Ab (IgG,A,M)	Blood	Hem Red	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	5.0	SST	Ref Refrig	Refrigerate
294	Carnitine Total and Free	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Red	Ref Refrig	Refrigerate
295	Carotene	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	5.0	SST	Ref Refrig	Protect from light
296	Catecholamines Plasma	Blood	Chem Sendout Frozen	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	2.0	Green	Ref Frozen	Freeze
297	Catecholamines Fractionated Urinary Free	Urine	Urine HCL	Urine	(ALL)	0 Minutes	150 Years	(ALL)	30.0	U24H HCL	Ref Refrig	Refrigerate
298	CBC	Blood	Blood Hem	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)	0.2	Lav (micro)	Hem	
299	CBC	Blood	Blood Hem	Venous Draw	(ALL)	1 Months	150 Years	(ALL)	1.0	Lav	Hem	
300	CBC with Diff	Blood	Blood Hem	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)	0.2	Lav (micro)	Hem	
301	CBC with Diff	Blood	Blood Hem	Venous Draw	(ALL)	1 Months	150 Years	(ALL)	1.0	Lav	Hem	
302	CEA	Blood	Chem Architect	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	SST	Immune	
303	Celiac Comp Panel ,Infant (0-3 yrs)	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	5.0	Red	Ref Refrig	Refrigerate
304	Celiac Disease Comp Panel	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	2.0	Red	Ref Refrig	Refrigerate
305	Cell Count Abscess Drainage	Abcess Drainage	Body Fluid Hem	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Lav	Hem	Deliver Immediately
306	Cell Count CSF	CSF	CSF Hem	Syringe	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Sterile T	Hem	Deliver Immediately
307	Cell Count Dialysate Fluid	Dialysis Fluid	Body Fluid Hem	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Lav	Hem	Deliver Immediately
308	Cell Count J Tube Drainage	J-Tube Drainage	Body Fluid Hem	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Lav	Hem	Deliver Immediately
309	Cell Count Pericardial Fluid	Pericardial Fluid	Body Fluid Hem	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Lav	Hem	Deliver Immediately
310	Cell Count Peritoneal Fluid	Peritoneal Fluid	Body Fluid Hem	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Lav	Hem	Deliver Immediately
311	Cell Count Pleural Fluid	Pleural Fluid	Body Fluid Hem	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Lav	Hem	Deliver Immediately
312	Cell Count Stem Cell	Stem Cell	Blood	Stem Cell	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Lav	Hem	
313	Cell Count Synovial Fluid	Synovial fluid	Body Fluid Hem	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)	0.1	Lav	Hem	Deliver Immediately
314	Cell Count Body Fluid	Pleural Fluid	Body Fluid	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)	2.0	Lav	Hem	
315	Cell Count Body Fluid	Body Fluid	Body Fluid	Aspirate	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Lav	Hem	
316	Cell Count Body Fluid	Synovial fluid	Body Fluid	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)	2.0	Lav	Hem	
317	Cell Count Body Fluid	Pericardial Fluid	Body Fluid	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)	2.0	Lav	Hem	
318	Cell Count Body Fluid	Abcess Drainage	Body Fluid	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)	2.0	Lav	Hem	
319	Cell Count Body Fluid	Dialysis Fluid	Body Fluid	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)	2.0	Lav	Hem	
320	Cell Count Body Fluid	J-Tube Drainage	Blood	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)	2.0	Lav	Hem	
321	Cell Count Body Fluid	Peritoneal Fluid	Body Fluid	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)	2.0	Lav	Hem	
322	Ceruloplasmin	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	5.0	Red	Ref Refrig	Refrigerate
323	Chlamydia DNA by PCR	Cervical Swab	VM	Dacron Swab	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Vir/Chlam	Virol	Refrigerate
324	Chlamydia DNA by PCR	Urethral Swab	VM	Dacron Swab	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Vir/Chlam	Virol	Refrigerate
325	Chlamydia Species Ab(IgG)	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Red	Ref Refrig	Refrigerate
326	Chlamydia Species Ab(IgM)	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Red	Ref Refrig	Refrigerate
327	Chloride	Blood	Blood	Venous Draw	(ALL)	1 Months	150 Years	(ALL)	0.2	SST	Chem	
328	Chloride	Blood	Blood	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)	0.2	SST (micro)	Chem	
329	Chloride Blood	Blood Heparin	Blood Gases Arterial	Syringe	(ALL)	0 Minutes	150 Years	(ALL)	0.1	Hep Syringe	Chem	Deliver Immediately
330	Chloride Urine 24Hr	Urine	Urine	Urine	(ALL)	0 Minutes	150 Years	(ALL)	2.0	U24H	Urine Chem	Refrigerate
331	Chloride Cardioplegia Solution	Cardioplegia	Body Fluid	Pharmacy	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Red	Chem	
332	Chloride Urine Random	Urine	Urine	Urine	(ALL)	0 Minutes	150 Years	(ALL)	2.0	UA	Urine Chem	
333	Chlorpheniramine	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	5.0	Red	Ref Refrig	Refrigerate
334	Cholesterol	Blood	Blood	Venous Draw	(ALL)	31 Days	150 Years	(ALL)	0.3	SST	Chem	
335	Cholesterol	Blood	Blood	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)	0.3	SST (micro)	Chem	
336	Cholinesterase	Blood	Chem lav sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	5.0	Lav	Ref Refrig	Refrigerate

	Primary Synonym	Specimen Type	Accession Class	Collection Method	Service Resource	Age From	Age To	Coll Priority	Min Vol	Container	Collection Class	Special Handling
337	Chromatin Ab (IgG)	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Red	Ref Refrig	Refrigerate
338	Chromium Plasma	Blood	Chem Royal	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	5.0	Royal (EDTA)	Ref Refrig	Refrigerate
339	Chromium Urine	Urine	Urine 24hr	Urine	(ALL)	0 Minutes	150 Years	(ALL)	20.0	U Aliquot	Ref Room	Room temperate
340	Chromogranin A	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Red	Ref Refrig	Refrigerate
341	Chromosome Fish Technique	Blood	Blood	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	10.0	Green	Chem	Keep on Ice
342	Chymotrypsin, Stool	Stool	Stool	Stool	(ALL)	0 Minutes	150 Years	(ALL)	2.0	Transport	Ref Refrig	Refrigerate
343	Circulating Immune Complexes	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.3	Red	Ref Frozen	Freeze
344	Citric Acid, 24HR UR W/CR	Urine	Urine Boric	Urine	(ALL)	0 Minutes	150 Years	(ALL)	25.0	U24H Boric	Ref Refrig	Refrigerate
345	CK	Blood	Blood	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)	0.3	SST (micro)	Chem	
346	CK	Blood	Blood	Venous Draw	(ALL)	31 Days	150 Years	(ALL)	0.3	SST	Chem	
347	CK MB	Blood	Blood	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)	0.3	SST (micro)	Chem	
348	CK MB	Blood	Blood	Venous Draw	(ALL)	31 Days	150 Years	(ALL)	0.3	SST	Chem	
349	Clomipramine & Metabolite	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Royal (Serum)	Ref Refrig	Refrigerate
350	Clonazepam	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	5.0	Red	Ref Frozen	Freeze
351	Clozapine	Blood	Chem Sendout Frozen	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	3.0	Lav	Ref Frozen	Freeze
352	CMV Antibodies IgM Quantitative	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	2.0	SST	Ref Refrig	Refrigerate
353	CMV DNA,QL,Real-Time PCR	Blood	Chem lav sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Lav	Ref Refrig	Refrigerate
354	CMV DNA,QN,Real-Time PCR	Blood	Chem lav sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Lav	Ref Refrig	Refrigerate
355	CMV Immune Status IgG by EIA	Blood	VS	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.3	SST	Serology	
356	Coccidioides Antibodies	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	4.0	Red	Ref Refrig	Refrigerate
357	Coccidioides Antibodies CSF	CSF	CSF	Syringe	(ALL)	0 Minutes	150 Years	(ALL)	2.0	Sterile T	Ref Refrig	Refrigerate
358	Cold Hemagglutinins	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	3.0	Red	Ref Room	Room temperate
359	Complement Total	Blood	Chem Sendout Frozen	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Red	Ref Frozen	Freeze
360	Complement C1 Esterase Inhibitor Function	Blood	Chem Sendout Frozen	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	5.0	Red	Ref Frozen	Freeze
361	Complement C1q	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	3.0	Red	Ref Refrig	Refrigerate
362	Complement C2	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	5.0	Red	Ref Refrig	Refrigerate
363	Complement C3	Blood	Blood	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.5	SST	Chem	
364	Complement C3 Activator	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	5.0	Red	Ref Refrig	Refrigerate
365	Complement C4	Blood	Blood	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.5	SST	Chem	
366	Comprehensive Metabolic Panel	Blood	Blood	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)	0.5	SST (micro)	Chem	
367	Comprehensive Metabolic Panel	Blood	Blood	Venous Draw	(ALL)	31 Days	150 Years	(ALL)	0.5	SST	Chem	
368	COPATH Reflex	Thin Prep	Special Collectio	Thin Prep	(ALL)	0 Minutes	150 Years	(ALL)	0.5	Thin Prep	Ref Room	Room temperate
369	Copper	Blood	Chem Royal	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Royal (Serum)	Ref Refrig	Refrigerate
370	Copper Urine Random	Urine	Urine	Urine	(ALL)	0 Minutes	150 Years	(ALL)	0.3	U Aliquot	Ref Room	Room temperate
371	Copper Urine 24hr	Urine	Urine 24hr	Urine	(ALL)	0 Minutes	150 Years	(ALL)	7.0	Contact Laboratory	Ref Refrig	Refrigerate
372	Cord Blood Gas	Blood Heparin	Blood Gases Cord	Syringe	(ALL)	0 Minutes	150 Years	(ALL)	0.1	Hep Syringe	Chem	
373	Cortisol AM	Blood	Blood	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	SST	SP Chem	
374	Cortisol PM	Blood	Blood	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	SST	SP Chem	
375	Cortisol Urinary Free	Urine	Urine	Urine	(ALL)	0 Minutes	150 Years	(ALL)	10.0	U24H	Ref Refrig	Refrigerate
376	Cortisol Stimulation	Blood	Blood	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	SST	SP Chem	
377	Coxsackie A Ab CF QN	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	5.0	SST	Ref Refrig	Refrigerate
378	Coxsackie B Abs by CF	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	5.0	Red	Ref Refrig	Refrigerate
379	Creatinine	Blood	Blood	Venous Draw	(ALL)	1 Months	150 Years	(ALL)	0.2	SST	Chem	
380	Creatinine	Blood	Blood	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)	0.2	SST (micro)	Chem	
381	Creatinine Urine 24Hr	Urine	Urine 24hr	Urine	(ALL)	0 Minutes	150 Years	(ALL)	2.0	U24H	Urine Chem	Refrigerate
382	Creatinine Urine Random	Urine	Urine	Urine	(ALL)	0 Minutes	150 Years	(ALL)	2.0	UA	Urine Chem	
383	Creatinine Peritoneal Dialysate	DIALY	Body Fluid	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)	0.5	Red	Chem	
384	Creatinine Urine Timed	Urine	Urine	Urine	(ALL)	0 Minutes	150 Years	(ALL)	2.0	UA	Urine Chem	Refrigerate
385	Cross Linked N Telopeptide Urine	Urine	Urine	Urine	(ALL)	0 Minutes	150 Years	(ALL)	10.0	UA	Ref Frozen	Freeze
386	Cryofibrinogen Screen	Blood	Blood Hem	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	2.0	Blue	Hem	Incubate at 37.C
387	Cryoglobulin Screen	Blood	Hem Red	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	3.0	Red	Hem	Incubate at 37.C
388	Cryptococcus Antibodies	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	5.0	Red	Ref Refrig	Refrigerate
389	Crystal ID Body Fluids	Synovial fluid	Body Fluid	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)	0.3	Green	Hem	
390	Cyanide Blood	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	2.0	Green	Ref Refrig	Refrigerate
391	Cyclosporine	Blood	CHEM LAV	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	2.0	Lav	Chem	
392	Cysticercus IgG Ab, WB	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Red	Ref Refrig	Refrigerate

	Primary Synonym	Specimen Type	Accession Class	Collection Method	Service Resource	Age From	Age To	Coll Priority	Min Vol	Container	Collection Class	Special Handling
393	Cystine, QN,24 Hour Urine	Urine	Urine 24hr	Urine	(ALL)	0 Minutes	150 Years	(ALL)	2.0	U24H	Ref Frozen	Freeze
394	Cytogenetics Amniotic Fluid	Amniotic Fluid	Body Fluid	Special Collection	(ALL)	0 Minutes	150 Years	(ALL)	30.0	Sterile Plastic	Chem	Room temperate
395	Cytogenetics Bone Marrow	Bone Marrow	Blood	Special Collection	(ALL)	0 Minutes	150 Years	(ALL)	2.0	Green	Chem	Room temperate
396	Cytogenetics Fragile X	Blood	EDTA	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	9.0	Lav	Chem	Room temperate
397	Cytogenetics High Resolution	Blood	Blood	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	10.0	Green	Chem	Keep on Ice
398	Cytogenetics Peripheral Blood	Blood	Blood Heparin	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	3.0	Green	Chem	Refrigerate
399	Cytogenetics Tissue	Tissue	Special Collectio	Special Collection	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Sterile Plastic	Chem	Refrigerate
400	D Dimer	Blood	Blood Coag	Venous Draw	(ALL)	12 Years	150 Years	(ALL)	0.5	Blue	Coag	Mix Immediately
401	D Dimer	Blood	Blood Coag	Venous Draw	(ALL)	0 Minutes	12 Years	(ALL)	0.5	Blue	Coag	Mix Immediately
402	Dengue Fever (IgG,M)	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Red	Ref Refrig	
403	Deoxycorticosterone	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	2.0	SST	Ref Refrig	Refrigerate
404	Desipramine	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	5.0	Red	Ref Refrig	Refrigerate
405	DHEA	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	5.0	Red	Ref Refrig	Refrigerate
406	DHEAS	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	3.0	Red	Ref Refrig	Refrigerate
407	Digoxin	Blood	Blood	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)	0.3	SST (micro)	Chem	
408	Digoxin	Blood	Blood	Venous Draw	(ALL)	31 Days	150 Years	(ALL)	0.3	SST	Chem	Draw prior to next dose
409	Dihydrotestosterone	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	2.0	Red	Ref Refrig	Refrigerate
410	Diphtheria Antitoxoid	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	3.0	SST	Ref Refrig	Refrigerate
411	Disopyramide	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	3.0	Red	Ref Refrig	Refrigerate
412	DMCB Influenza A & B Rapid Antigen	Nasopharyn Swab	VC	Swab	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Cx Swab	Serology	
413	DMCB Influenza A & B Rapid Antigen	Nasopharyn Wash	VC	Vial	(ALL)	0 Minutes	150 Years	(ALL)	3.0	Sterile Plastic	Serology	
414	DMCB Influenza A & B Rapid Antigen	Nasopharyn Wash	VC	Vial	(ALL)	0 Minutes	150 Years	(ALL)	3.0	Sterile Plastic	Serology	
415	DMCB Influenza A & B Rapid Antigen	Nasopharyn Wash	VC	Vial	(ALL)	0 Minutes	150 Years	(ALL)	3.0	Sterile Plastic	Serology	
416	Drug Coma Overdose Profile Blood	Blood	Blood	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	15.0	Red	Chem	
417	Drug Coma Overdose Profile Blood & Urin	Urine	Urine	Urine	(ALL)	0 Minutes	150 Years	(ALL)	20.0	U Aliquot	Chem	
418	Drug Coma Overdose Profile Blood & Urin	Blood and Urine	Blood	Venous Draw and Urine	(ALL)	0 Minutes	150 Years	(ALL)	5.0	Red	Chem	
419	Drug Coma Overdose Profile Blood & Urin	Blood and Urine	Blood	Venous Draw and Urine	(ALL)	0 Minutes	150 Years	(ALL)	60.0	UA	Chem	
420	Drug Coma Overdose Profile Blood & Urin	Blood	Blood	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	15.0	Red	Chem	
421	Drug Profile 7 Drugs Blood	Blood	Blood	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	4.0	Red	Ref Refrig	Refrigerate
422	Drug Screen COGA Urine	Urine	Urine	Urine	(ALL)	0 Minutes	150 Years	(ALL)	15.0	U Aliquot	Ref Refrig	
423	E. Histolytica (IgG), Elisa	Blood	Chem Sendout Frozen	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Red	Ref Frozen	Freeze
424	EBV Acute Infection Antibodies	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	8.5	SST	Ref Refrig	Refrigerate
425	EBV DNA, QN, PCR	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Lav	Ref Refrig	Refrigerate
426	EBV Early Antigen IgG	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.3	SST	Ref Refrig	Refrigerate
427	EBV Viral Capsid Antigen IgG	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.3	SST	Ref Refrig	
428	Electrolyte Panel	Blood	Blood	Venous Draw	(ALL)	0 Minutes	2 Years	(ALL)	0.5	SST (micro)	Chem	
429	Electrolyte Panel	Blood	Blood	Venous Draw	(ALL)	2 Years	150 Years	(ALL)	0.5	SST	Chem	
430	ENA I	Blood	Blood	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.3	SST	Immune	
431	ENA II	Blood	Blood	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.3	SST	Immune	
432	Endomysial Scr (IgA)W/RFL	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Red	Ref Refrig	Refrigerate
433	Environmental Culture	Dialysate	MB Other	Vial	(ALL)	0 Minutes	150 Years	(ALL)	15.0	Sterile Plastic	Micro	
434	Environmental Culture	Water	MB Other	Vial	(ALL)	0 Minutes	150 Years	(ALL)	15.0	Sterile Plastic	Micro	
435	Eosinophil Urine	Urine	Urinalysis	Urine	(ALL)	0 Minutes	150 Years	(ALL)	5.0	UA	Urinalysis	
436	Eosinophil Nasal Smear	Nasal Swab	Special Collectio	Slide Prep	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Slides	Hem	
437	Epstein Barr Virus Nuclear Antigen IgG	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	3.0	Red	Ref Refrig	Refrigerate
438	Epstein-Barr Virus IFA CSF	CSF	Chem Sendout	Syringe	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Sterile T	Ref Refrig	Refrigerate
439	Erythrocyte Sedimentation Rate	Blood	Blood Hem	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Lav	Hem	
440	Erythropoietin	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	3.0	SST	Ref Refrig	Refrigerate
441	Estradiol	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	5.0	SST	Ref Refrig	Refrigerate
442	Estradiol, Ultra-Sensitive	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	SST	Ref Refrig	Refrigerate
443	Estriol	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	3.0	SST	Ref Refrig	Refrigerate
444	Estrogens Total	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	5.0	Red	Ref Refrig	Refrigerate
445	Estrone	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	3.0	Red	Ref Refrig	Refrigerate
446	Ethanol	Blood	Blood	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	SST	Chem	
447	Ethosuximide	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	5.0	Red	Ref Refrig	Refrigerate
448	Ethylene Glycol	Blood	Blood	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	5.0	Grey	Chem	Refrigerate

	Primary Synonym	Specimen Type	Accession Class	Collection Method	Service Resource	Age From	Age To	Coll Priority	Min Vol	Container	Collection Class	Special Handling
449	Expedited Maternal HIV-1 Antibody	Blood	VH	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	2.0	Red	HIV	
450	Expedited Needle Stick HIV-1	Blood	VH	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	SST	HIV	
451	Expedited Newborn HIV-1 Antibody	Venous	VH	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	2.0	Red	HIV	Deliver Immediately
452	Factor II Assay	Blood	Blood Coag	Venous Draw	(ALL)	0 Minutes	12 Years	(ALL)	0.5	Blue (Peds)	Coag	Mix Immediately
453	Factor II Assay	Blood	Blood Coag	Venous Draw	(ALL)	12 Years	150 Years	(ALL)	0.5	Blue	Coag	Mix Immediately
454	Factor IX Assay	Blood	Blood Coag	Venous Draw	(ALL)	0 Minutes	12 Years	(ALL)	0.5	Blue	Coag	Mix Immediately
455	Factor IX Assay	Blood	Blood Coag	Venous Draw	(ALL)	12 Years	150 Years	(ALL)	0.5	Blue	Coag	Mix Immediately
456	Factor V Assay	Blood	Blood Coag	Venous Draw	(ALL)	12 Years	150 Years	(ALL)	0.5	Blue	Coag	Mix Immediately
457	Factor V Assay	Blood	Blood Coag	Venous Draw	(ALL)	0 Minutes	12 Years	(ALL)	0.5	Blue (Peds)	Coag	Mix Immediately
458	Factor V Leiden Mutation	Blood	Hem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Lav	Ref Room	Room temperate
459	Factor VII Assay	Blood	Blood Coag	Venous Draw	(ALL)	0 Minutes	12 Years	(ALL)	0.5	Blue	Coag	Mix Immediately
460	Factor VII Assay	Blood	Blood Coag	Venous Draw	(ALL)	12 Years	150 Years	(ALL)	0.5	Blue	Coag	Mix Immediately
461	Factor VIII Act/Inh W/RFL	Blood	Blood	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	5.0	Blue	Ref Frozen	Freeze
462	Factor VIII Activity	Blood	Hem Blue Sendouts	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	3.0	Blue	Ref Frozen	Freeze
463	Factor VIII Assay	Blood	Blood Coag	Venous Draw	(ALL)	0 Minutes	12 Years	(ALL)	0.5	Blue (Peds)	Coag	Mix Immediately
464	Factor VIII Assay	Blood	Blood Coag	Venous Draw	(ALL)	12 Years	150 Years	(ALL)	0.5	Blue	Coag	Mix Immediately
465	Factor VIII Inhibitor	Blood	Blood Coag	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	4.5	Blue	Ref Frozen	Freeze
466	Factor X Assay	Blood	Blood Coag	Venous Draw	(ALL)	0 Minutes	12 Years	(ALL)	0.5	Blue (Peds)	Coag	Mix Immediately
467	Factor X Assay	Blood	Blood Coag	Venous Draw	(ALL)	12 Years	150 Years	(ALL)	0.5	Blue	Coag	Mix Immediately
468	Factor XI Assay	Blood	Blood Coag	Venous Draw	(ALL)	0 Minutes	12 Years	(ALL)	0.5	Blue (Peds)	Coag	Mix Immediately
469	Factor XI Assay	Blood	Blood Coag	Venous Draw	(ALL)	12 Years	150 Years	(ALL)	0.5	Blue	Coag	Mix Immediately
470	Factor XII Assay	Blood	Blood Coag	Venous Draw	(ALL)	0 Minutes	12 Years	(ALL)	0.5	Blue	Coag	Mix Immediately
471	Factor XII Assay	Blood	Blood Coag	Venous Draw	(ALL)	12 Years	150 Years	(ALL)	0.5	Blue	Coag	Mix Immediately
472	Factor XIII Screen	Blood	Blood Coag	Venous Draw	(ALL)	0 Minutes	12 Years	(ALL)	0.5	Blue	Coag	Mix Immediately
473	Factor XIII Screen	Blood	Blood Coag	Venous Draw	(ALL)	12 Years	150 Years	(ALL)	0.5	Blue	Coag	Mix Immediately
474	Familial Med Fever Mut, NY	Blood	Chem lav sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	3.0	Lav	Ref Room	Room temperate
475	Fatty Acd PRF,Peroxisomal	Blood	Chem Sendout Frozen	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Red	Ref Frozen	Freeze
476	Fatty Acids Free	Blood	Chem Sendout Frozen	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	3.0	SST	Ref Frozen	Freeze
477	Fecal Electrolytes	Stool	Stool	Stool	(ALL)	0 Minutes	150 Years	(ALL)	5.0	Transport	Ref Refrig	Refrigerate
478	Fecal Fat	Stool	Stool	Stool	(ALL)	0 Minutes	150 Years	(ALL)	1.0	72H Stool	Ref Frozen	Freeze
479	Fecal Fat Qualitative	Stool	Stool	Stool	(ALL)	0 Minutes	150 Years	(ALL)	5.0	Sterile T	Ref Refrig	Refrigerate
480	Fecal Reducing Substances	Stool	Stool	Stool	(ALL)	0 Minutes	150 Years	(ALL)	5.0	Transport	Ref Frozen	Freeze
481	Felbamate	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	5.0	Red	Ref Refrig	Refrigerate
482	Ferritin	Blood	Chem E170	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	SST	SP Chem	
483	Fibrinogen	Blood	Blood Coag	Venous Draw	(ALL)	12 Years	150 Years	(ALL)	0.5	Blue	Coag	Mix Immediately
484	Fibrinogen	Blood	Blood Coag	Venous Draw	(ALL)	0 Minutes	12 Years	(ALL)	0.5	Blue (Peds)	Coag	Mix Immediately
485	Fish,Her-2/Neu,Para Block	Biopsy	Chem Sendout	Slide	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Slides	Ref Room	Room temperate
486	FK 506 Blood	Blood	Chem FK506	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Lav	SP Chem	Draw prior to next dose
487	Fluoxetine	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	3.0	Red	Ref Refrig	Refrigerate
488	Fluphenazine	Blood	Blood	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	5.0	Red	Chem	Refrigerate
489	Folates RBC	Blood	Chem lav sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	6.0	Lav	Ref Refrig	Refrigerate
490	Folic Acid	Blood	Chem E170	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	SST	SP Chem	
491	Follicle Stimulating Hormone	Blood	Chem Architect	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	SST	SP Chem	
492	Free Kappa Lambda Light Chains	Serum	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	SST	Ref Refrig	Refrigerate
493	Free Kappa Lambda Random Urine	Urine	Chem Sendout	Urine	(ALL)	0 Minutes	150 Years	(ALL)	2.0	UA	Ref Refrig	Refrigerate
494	Fructosamine	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	SST	Ref Refrig	
495	G6PD Quantitative	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Lav (micro)	Ref Refrig	Whole Blood
496	Ganglioside Ab (IGG,IGM)	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	SST	Ref Refrig	Refrigerate
497	Gastrin	Blood	Chem Sendout Frozen	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	4.0	SST	Ref Frozen	Freeze
498	GCT	Serum	GCT	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.3	SST	Chem	
499	Gentamicin Peak	Serum	Peak	Venous Draw	(ALL)	31 Days	150 Years	(ALL)	0.3	SST	Chem	
500	Gentamicin Peak	Serum	Peak	Venous Draw	(ALL)	0 Minutes	31 Months	(ALL)	0.3	SST (micro)	Chem	
501	Gentamicin Random	Blood	Blood	Venous Draw	(ALL)	0 Minutes	31 Months	(ALL)	0.3	SST (micro)	Chem	
502	Gentamicin Random	Blood	Blood	Venous Draw	(ALL)	31 Days	150 Years	(ALL)	0.3	SST	Chem	
503	Gentamicin Trough	Serum	Trough	Venous Draw	(ALL)	0 Minutes	31 Days	(ALL)	0.3	SST (micro)	Chem	
504	Gentamicin Trough	Serum	Trough	Venous Draw	(ALL)	31 Days	150 Years	(ALL)	0.3	SST	Chem	

	Primary Synonym	Specimen Type	Accession Class	Collection Method	Service Resource	Age From	Age To	Coll Priority	Min Vol	Container	Collection Class	Special Handling
505	GGT	Blood	Blood	Venous Draw	(ALL)	31 Days	150 Years	(ALL)	0.3	SST	Chem	
506	GGT	Blood	Blood	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)	0.3	SST (micro)	Chem	
507	Giardia lamblia by EIA	Stool	MB Stool	Stool	(ALL)	0 Minutes	150 Years	(ALL)	2.0	O&P kit	Micro	Room temperate
508	GLIADIN AB(G,A)/RETICULIN AB (A) w/RF	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Red	Ref Refrig	Refrigerate
509	Glucagon Plasma	Blood	Chem Sendout Frozen	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	5.0	Lav	Ref Frozen	Freeze
510	Glucose 2 Hr PP	Serum	2hr PP	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)	0.2	SST (micro)	Chem	
511	Glucose 2 Hr PP	Serum	2hr PP	Venous Draw	(ALL)	1 Months	150 Years	(ALL)	0.3	SST	Chem	
512	Glucose CSF	CSF	CSF	Syringe	(ALL)	0 Minutes	150 Years	(ALL)	0.5	Sterile T	Chem	Deliver Immediately
513	Glucose Fasting	Blood	FBS	Venous Draw	(ALL)	31 Days	150 Years	(ALL)	1.0	Grey	Chem	
514	Glucose Fasting	Blood	FBS	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)	0.2	SST (micro)	Chem	
515	Glucose Random	Blood	Blood	Venous Draw	(ALL)	1 Months	150 Years	(ALL)	0.3	SST	Chem	
516	Glucose Random	Blood	Blood	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)	0.2	SST (micro)	Chem	
517	Glucose TPN Pharmacy	TPN	Special Collectio	Pharmacy	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Red	Chem	
518	Glucose Urine Random	Urine	Urine	Urine	(ALL)	0 Minutes	150 Years	(ALL)	2.0	UA	Urine Chem	
519	Glucose #1	Blood	Blood	Venous Draw	(ALL)	31 Days	150 Years	(ALL)	0.3	SST	Chem	
520	Glucose #1	Blood	Blood	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)	0.2	SST (micro)	Chem	
521	Glucose #2	Blood	Blood	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)	0.2	SST (micro)	Chem	
522	Glucose #2	Blood	Blood	Venous Draw	(ALL)	31 Days	150 Years	(ALL)	0.3	SST	Chem	
523	Glucose #3	Blood	Blood	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)	0.2	SST (micro)	Chem	
524	Glucose #3	Blood	Blood	Venous Draw	(ALL)	31 Days	150 Years	(ALL)	0.3	SST	Chem	
525	Glucose #4	Blood	Blood	Venous Draw	(ALL)	31 Days	150 Years	(ALL)	0.3	SST	Chem	
526	Glucose #4	Blood	Blood	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)	0.2	SST (micro)	Chem	
527	Glucose #5	Blood	Blood	Venous Draw	(ALL)	31 Days	150 Years	(ALL)	0.3	SST	Chem	
528	Glucose #5	Blood	Blood	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)	0.2	SST (micro)	Chem	
529	Glucose #6	Blood	Blood	Venous Draw	(ALL)	31 Days	150 Years	(ALL)	0.3	SST	Chem	
530	Glucose #6	Blood	Blood	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)	0.2	SST (micro)	Chem	
531	Glucose Load	Blood	Blood	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Grey	Chem	
532	Glucose Peritoneal Dialysate	DIALY	Body Fluid	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)	0.5	Red	Chem	
533	Glucose Peritoneal Fluid	Peritoneal Fluid	Body Fluid	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)	0.3	Red	Chem	
534	Glucose Pleural Fluid	Pleural Fluid	Body Fluid	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)	0.3	Red	Chem	
535	Glucose Synovial Fluid	Synovial fluid	Body Fluid	Syringe	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Red	Chem	
536	Glucose Urine 24Hr	Urine	Urine 24hr	Urine	(ALL)	0 Minutes	150 Years	(ALL)	2.0	U24H	Urine Chem	Refrigerate
537	Glucose, Serum, 1.0Hour (1.0Hr)	Blood	Blood	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Grey	Chem	
538	Glucose, Serum, 2.0Hour (GLUS 2.0Hr)	Blood	Blood	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Grey	Chem	
539	Glucose, Serum, 3.0Hour (GLUS 3.0Hr)	Blood	Blood	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.1	Grey	Chem	
540	Glucose, Serum, 4.0Hour (GLUS 4.0Hr)	Blood	Blood	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.1	Grey	Chem	
541	Glucose, Serum, 5.0Hour (GLUS 5.0Hr)	Blood	Blood	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.1	Grey	Chem	
542	Glucose, Serum, 6.0Hour (GLUS 6.0Hr)	Blood	Blood	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Grey	Chem	
543	Glucose, Serum, Fasting (GLUS 0Hr)	Blood	Blood	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Grey	Chem	
544	Glutamic Acid Decarboxylase Autoantibod	Blood	Chem Sendout Frozen	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Red	Ref Frozen	Freeze
545	Glycomark	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Red	Ref Room	Room temperate
546	Growth Hormone	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	3.0	SST	Ref Refrig	Refrigerate
547	Growth Hormone #1	Serum	Blood	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.4	Red	Chem	
548	Growth Hormone #2	Serum	Blood	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.4	Red	Chem	
549	Growth Hormone #3	Serum	Blood	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.4	Red	Chem	
550	Growth Hormone #4	Serum	Blood	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.4	Red	Chem	
551	Growth Hormone #5	Serum	Blood	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.4	Red	Chem	
552	Growth Hormone #6	Serum	Blood	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.4	Red	Chem	
553	Growth Hormone 5 Spec	Serum	Blood	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	3.0	Red	Chem	
554	Growth Hormone 6 Spec	Serum	Blood	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	3.0	Red	Chem	
555	H. Pylori Ab (IgG,A)	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	SST	Ref Refrig	Refrigerate
556	H. Pylori Ab (IgM)	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	SST	Ref Refrig	Refrigerate
557	H. Pylori Ag Det,EIA,Stool	Stool	Stool	Stool	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Sterile Plastic	Ref Frozen	Freeze
558	H.PYLORI AB (IGG)	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	SST	Ref Refrig	Refrigerate
559	Haptoglobin	Blood	Blood	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.5	SST	Chem	
560	HBV Genotype	Blood	Chem Sendout Frozen	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Lav	Ref Frozen	Freeze

	Primary Synonym	Specimen Type	Accession Class	Collection Method	Service Resource	Age From	Age To	Coll Priority	Min Vol	Container	Collection Class	Special Handling
561	HCV RNA Genotype, Lipa	Blood	Chem Sendout Frozen	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	3.0	Lav	Ref Frozen	Freeze
562	HCV RNA, QN, BDNA W/RFL TMA	Blood	Chem Sendout Frozen	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Lav	Ref Frozen	Freeze
563	HCV RNA, QN,BDNA/TMA	Blood	Chem Sendout Frozen	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	2.0	Lav	Ref Frozen	Freeze
564	HDL Cholesterol	Blood	Blood	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)	0.3	SST (micro)	Chem	
565	HDL Cholesterol	Blood	Blood	Venous Draw	(ALL)	31 Days	150 Years	(ALL)	0.3	SST	Chem	
566	Heinz Body Stain	Blood	Blood Hem	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)	0.2	Lav (micro)	Hem	
567	Heinz Body Stain	Blood	Blood Hem	Venous Draw	(ALL)	1 Months	150 Years	(ALL)	0.5	Lav	Hem	
568	Helicobacter Pylori IgA	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	SST	Ref Refrig	Refrigerate
569	Helicobacter Pylori Urease Test	Biopsy	Special Collectio	Biopsy	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Contact Laboratory	MISC	Obtain Collection Kit from I
570	Hem Consult Smear Review	Blood	Blood Hem	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.1	Lav	Hem	
571	Hematocrit	Blood	Blood Hem	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)	0.2	Lav (micro)	Hem	
572	Hematocrit	Blood	Blood Hem	Venous Draw	(ALL)	1 Months	150 Years	(ALL)	0.5	Lav	Hem	
573	Hematocrit Arterial Blood	Blood Heparin	Blood Gases Arterial	Syringe	(ALL)	0 Minutes	150 Years	(ALL)	0.1	Hep Syringe	Chem	Deliver Immediately
574	Hematocrit Cell Saver	Blood	Blood Hem	Cell Saver	(ALL)	0 Minutes	150 Years	(ALL)	0.5	Lav	Hem	Deliver Immediately
575	Hematocrit Fluid	Synovial fluid	Body Fluid	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)	0.5	Lav	Hem	
576	Hematocrit Fluid	Body Fluid	Body Fluid	Syringe	(ALL)	0 Minutes	150 Years	(ALL)	0.5	Lav	Hem	Deliver Immediately
577	Hematocrit Fluid	Pleural Fluid	Body Fluid	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)	0.5	Lav	Hem	
578	Hematocrit Fluid	Pericardial Fluid	Body Fluid	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)	0.5	Lav	Hem	
579	Hematocrit Fluid	Abcess Drainage	Body Fluid	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)	0.5	Lav	Hem	
580	Hematocrit Fluid	Dialysis Fluid	Body Fluid	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)	0.5	Lav	Hem	
581	Hematocrit Fluid	J-Tube Drainage	Body Fluid	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)	0.5	Lav	Hem	
582	Hematocrit Fluid	Peritoneal Fluid	Body Fluid	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)	0.5	Lav	Hem	
583	Hematocrit Abscess Drainage	Abcess Drainage	Body Fluid	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)	0.2	Lav	Hem	
584	Hematocrit CSF	CSF	CSF	Syringe	(ALL)	0 Minutes	150 Years	(ALL)	0.2	Sterile Plastic	Hem	
585	Hematocrit Dialysate Fluid	Dialysis Fluid	Body Fluid	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)	0.2	Lav	Hem	
586	Hematocrit J Tube Drainage	J-Tube Drainage	Body Fluid	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)	0.2	Lav	Hem	
587	Hematocrit Pericardial Fluid	Pericardial Fluid	Body Fluid	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)	0.2	Lav	Hem	
588	Hematocrit Peritoneal Fluid	Peritoneal Fluid	Blood Gases Venous	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)	0.2	Lav	Hem	
589	Hematocrit Pleural Fluid	Pleural Fluid	Body Fluid	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)	0.2	Lav	Hem	
590	Hematocrit Synovial Fluid	Synovial fluid	Body Fluid	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)	0.2	Lav	Hem	
591	Hemoglobin A1c	Blood	CHEM LAV	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Lav	Chem	
592	Hemosiderin Urine	Urine	Urine	Urine	(ALL)	0 Minutes	150 Years	(ALL)	5.0	UA	Urinalysis	
593	Heparin, Low Molecular WT	Blood	Hem Blue Sendouts	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Blue	Ref Frozen	Freeze
594	Heparin, Unfractionated	Blood	Blood Coag	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Blue	Ref Frozen	Freeze
595	Hepatic Function Panel	Blood	Blood	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)	0.2	SST (micro)	Chem	
596	Hepatic Function Panel	Blood	Blood	Venous Draw	(ALL)	31 Days	150 Years	(ALL)	0.5	SST	Chem	
597	Hepatitis A Ab Total	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	3.0	SST	Ref Refrig	Refrigerate
598	Hepatitis A Antibody IgM	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	3.0	SST	Ref Refrig	Refrigerate
599	Hepatitis B Core IgM	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	3.0	SST	Ref Refrig	Refrigerate
600	Hepatitis B Core Antibody Total	Blood	Blood	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	SST	SP Chem	
601	Hepatitis B DNA Qualitative	Blood	Chem Sendout Frozen	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	3.0	SST	Ref Frozen	Freeze
602	Hepatitis B DNA Quantitative	Blood	Chem Sendout Frozen	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	2.0	Lav	Ref Frozen	Freeze
603	Hepatitis B Surface Antibody	Blood	Blood	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	SST	SP Chem	
604	Hepatitis B Surface Antigen	Blood	Blood	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	SST	SP Chem	
605	Hepatitis Be Antibody	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	4.0	SST	Ref Refrig	Refrigerate
606	Hepatitis Be Antigen	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	4.0	SST	Ref Refrig	Refrigerate
607	Hepatitis C Antibody	Blood	Chem Architect	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	4.0	SST	Chem	Refrigerate
608	Hepatitis C Confirming by RIBA	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	4.0	SST	Ref Refrig	Refrigerate
609	Hepatitis C RNA by PCR Qualitative	Blood	Chem lav sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	3.0	Lav	Ref Frozen	Freeze
610	Hepatitis C RNA by PCR Quantitative	Blood	Chem Sendout Frozen	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	3.0	Lav	Ref Frozen	Freeze
611	Hepatitis D IgM Ab	Serum	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	4.0	SST	Ref Refrig	Refrigerate
612	Hepatitis Delta Antibody	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	4.0	SST	Ref Room	Room temperate
613	Heptimax (TM) HCV RNA	Blood	Chem Sendout Frozen	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	3.0	Lav	Ref Frozen	Freeze
614	Herpes Simplex IgM	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	5.0	SST	Ref Refrig	Refrigerate
615	Herpes Type 1&2 DNA, PCR (CSF)	CSF	Chem Sendout Frozen	Syringe	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Sterile T	Ref Frozen	Freeze
616	Herpesvirus 8 IGG, AB, IFA	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	2.0	SST	Ref Refrig	Refrigerate

	Primary Synonym	Specimen Type	Accession Class	Collection Method	Service Resource	Age From	Age To	Coll Priority	Min Vol	Container	Collection Class	Special Handling
617	Hexoseamidase A	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	4.0	SST	Ref Frozen	Freeze
618	Hgb A2 (Lab Order Only)	Blood	Blood Hem	Venous Draw	(ALL)	1 Months	150 Years	(ALL)	1.0	Lav	Hem	
619	Hgb A2 (Lab Order Only)	Blood	Blood Hem	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)	0.2	Lav (micro)	Hem	
620	Hgb Electrophoresis Acid (Lab Order Only)	Blood	Blood Hem	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.5	Lav	Hem	
621	Hgb Electrophoresis Alkaline (Lab Order Only)	Blood	Blood Hem	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)	0.2	Lav (micro)	Hem	
622	Hgb Electrophoresis Alkaline (Lab Order Only)	Blood	Blood Hem	Venous Draw	(ALL)	1 Months	150 Years	(ALL)	1.0	Lav	Hem	
623	Hgb Electrophoresis HPLC	Blood	Blood Hem	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)	0.2	Lav (micro)	Hem	
624	Hgb Electrophoresis HPLC	Blood	Blood Hem	Venous Draw	(ALL)	1 Months	150 Years	(ALL)	1.0	Lav	Hem	
625	Hgb F (Lab Order Only)	Blood	Blood Hem	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)	0.2	Lav (micro)	Hem	
626	Hgb F (Lab Order Only)	Blood	Blood Hem	Venous Draw	(ALL)	1 Months	150 Years	(ALL)	1.0	Lav	Hem	
627	Histamine Plasma	Blood	Chem Sendout Frozen	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	2.0	Lav	Ref Frozen	Freeze
628	Histoplasma Ab Panel	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Red	Ref Refrig	Refrigerate
629	Histoplasma AB, ID	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	2.0	Red	Ref Refrig	Refrigerate
630	Histoplasma Ag	CSF	Chem Sendout	Syringe	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Sterile T	Ref Refrig	Refrigerate
631	Histoplasma Ag	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	5.0	Red	Ref Refrig	Refrigerate
632	Histoplasma Ag	Urine	Urine	Urine	(ALL)	0 Minutes	150 Years	(ALL)	5.0	UA	Ref Refrig	Refrigerate
633	HIV 1 Antibody Screen	Blood	VH	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	6.0	SST	HIV	Room temperate
634	HIV 1 RNA Viral Load	Blood	VL	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	5.0	PPT	Virol	Refrigerate
635	HIV Retroviral Genotype	Blood	Chem Sendout Frozen	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	3.0	Lav	Ref Frozen	Freeze
636	HIV-1 Virpheno W/INT (NY)	Blood	Chem Sendout Frozen	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Lav	Ref Frozen	Freeze
637	HLA-B27 Antigen	Blood ACD	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	6.0	Yellow (ACD)	Ref Room	Room temperate
638	HLA-B5701 Typing	Blood ACD	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	10.0	Yellow (ACD)	Ref Refrig	Refrigerate
639	Homocysteine	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	4.0	SST	Ref Refrig	Refrigerate
640	Homovanillic Acid Urine 24Hr	Urine	Urine	Urine	(ALL)	0 Minutes	150 Years	(ALL)	30.0	U24H HCL	Ref Refrig	Refrigerate
641	Homovanillic Acid Urine Random	Urine	Urine	Urine	(ALL)	0 Minutes	150 Years	(ALL)	10.0	U Aliquot	Chem	
642	HSV 1 & 2 IgG Herpeselect Ab	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	5.0	SST	Ref Refrig	Refrigerate
643	HTLV-I/II AB,EIA W/RFL WB	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	2.0	Red	Ref Refrig	Refrigerate
644	Huntington Disease, NY	Blood	Chem lav sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	3.0	Lav	Ref Room	Room temperate
645	Hydroxyproline Free Quantitative Urine 2	Urine	Urine	Urine	(ALL)	0 Minutes	150 Years	(ALL)	30.0	U24H HCL	Ref Refrig	Refrigerate
646	Hydroxyproline Total Quantitative Urine 2	Urine	Urine	Urine	(ALL)	0 Minutes	150 Years	(ALL)	30.0	U24H HCL	Ref Refrig	Refrigerate
647	Hydroxyproline, Free, Rand	Urine	Urine	Urine	(ALL)	0 Minutes	150 Years	(ALL)	20.0	UA	Ref Refrig	Refrigerate
648	IAVI Repo Exposure Panel	Blood	VS	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Red	Serology	Freeze
649	IGF 1	Blood	Chem Sendout Frozen	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	2.0	SST	Ref Frozen	Freeze
650	IGF Binding Protein-1	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	2.0	Red	Ref Room	Room temperate
651	IGF-II	Blood	Blood	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Red	Ref Refrig	Refrigerate
652	IgG Subclasses 1 to 4	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Red	Ref Refrig	Refrigerate
653	Imipramine+Desipramine	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	3.0	Red	Ref Refrig	Refrigerate
654	Immune Complex, C1Q	Blood	Chem Sendout Frozen	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Red	Ref Frozen	Freeze
655	Immunoglobulin A Serum	Blood	Blood	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.5	SST	Chem	
656	Immunoglobulin E Serum	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	SST	Immune	
657	Immunoglobulin G	Blood	Blood	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.5	SST	Chem	
658	Immunoglobulin G CSF	CSF	CSF	Special Collection	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Sterile Plastic	Ref Refrig	
659	Immunoglobulin M	Blood	Blood	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.5	SST	Chem	
660	Immunoglobulin M Pediatric	Blood	Blood	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.5	SST	Chem	
661	Immunoglobulin M Pediatric	Blood	Blood	Venous Draw	(ALL)	0 Minutes	5 Years	(ALL)	0.5	SST	Chem	
662	Immunoglobulins Quantative	Blood	Blood	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.3	SST	Immune	
663	Infectious Mononucleosis Serum	Blood	VS	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)	0.3	SST (micro)	Serology	
664	Infectious Mononucleosis Serum	Blood	VS	Venous Draw	(ALL)	31 Days	150 Years	(ALL)	0.3	SST	Serology	
665	Inhibin A	Blood	Chem Sendout Frozen	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Red	Ref Frozen	Freeze
666	Inhibin B	Blood	Chem Sendout Frozen	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Red	Ref Frozen	Freeze
667	Insulin	Blood	Chem E170	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	2.0	SST	Chem	Freeze
668	Insulin #1	Serum	Blood	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	SST	Chem	Freeze
669	Insulin #2	Serum	Blood	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	SST	Chem	Freeze
670	Insulin #3	Serum	Blood	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	SST	Chem	Freeze
671	Insulin #4	Serum	Blood	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	SST	Chem	Freeze
672	Insulin #5	Serum	Blood	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	SST	Chem	Freeze

	Primary Synonym	Specimen Type	Accession Class	Collection Method	Service Resource	Age From	Age To	Coll Priority	Min Vol	Container	Collection Class	Special Handling
673	Insulin #6	Serum	Blood	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	SST	Chem	Freeze
674	Insulin Autoantibody	Blood	Chem Sendout Frozen	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	5.0	Red	Ref Frozen	Freeze
675	Insulin Like Growth Factor Binding Protein	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	3.0	Red	Ref Refrig	Refrigerate
676	Intrinsic Factor Blocking Antibodies	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	2.0	Red	Ref Room	Room temperate
677	IQ	Urine	Urinalysis	Urine	(ALL)	0 Minutes	150 Years	(ALL)	1.0	UA	Urinalysis	
678	Iron	Blood	Blood	Venous Draw	(ALL)	1 Months	150 Years	(ALL)	1.0	SST	Chem	
679	Iron	Blood	Blood	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)	0.3	SST (micro)	Chem	
680	Iron and TIBC	Blood	Blood	Venous Draw	(ALL)	2 Months	150 Years	(ALL)	3.0	SST	Chem	
681	Iron and TIBC	Blood	Blood	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)	1.0	SST (micro)	Chem	
682	Iron Stain Bone Marrow	Bone Marrow Asp	Blood Hem	Slide	(ALL)	0 Minutes	150 Years	(ALL)	2.0	Slides	Hem	Deliver Immediately
683	Iron Tissue	Tissue	Special Collectio	Tissue	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Biopsy Bottle	Chem	
684	Islet Cell Cytoplasmic Antibody	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	4.0	Red	Ref Refrig	Refrigerate
685	Isopropanol Stability Test	Blood	Blood Hem	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Lav	Hem	
686	ISTAT ABG Art	Blood	Blood Gases Arterial	Arterial Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	ISTAT	Chem	
687	ISTAT ABG Art	Arterial	Blood Gases Arterial	Arterial Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	ISTAT	Chem	
688	ISTAT ABG Art	Capillary	Blood Gases Arterial	Capillary	(ALL)	0 Minutes	150 Years	(ALL)	1.0	ISTAT	Chem	
689	ISTAT ABG Ven	Venous	Blood Gases Venous	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	ISTAT	Chem	
690	ISTAT ACT	Blood	Blood Heparin	Arterial Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	ISTAT	Chem	
691	ISTAT Arterial	Blood	Blood Gases Arterial	Arterial Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Hep Syringe	Chem	
692	ISTAT Chem8	OTHER	Blood	Fingerstick	(ALL)	0 Minutes	150 Years	(ALL)	1.0	ISTAT	Chem	
693	ISTAT Chem8	Venous	Blood	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	ISTAT	Chem	
694	ISTAT Chem8	Capillary	Blood	Capillary	(ALL)	0 Minutes	150 Years	(ALL)	1.0	ISTAT	Chem	
695	ISTAT Chem8	Arterial	Blood	Arterial Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	ISTAT	Chem	
696	ISTAT Glucose	Blood	POC BLD	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	ISTAT	Chem	
697	ISTAT IONIZED CA	Blood	Blood	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	ISTAT	Chem	
698	ISTAT Lytes	Blood	Blood Gases Arterial	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	ISTAT	Chem	
699	ISTAT PT	Arterial	Blood Coag	Arterial Draw	(ALL)	0 Minutes	150 Years	(ALL)	2.0	Blue	Coag	
700	ISTAT PT	Capillary	Blood Coag	Capillary	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Blue	Coag	
701	ISTAT PT	Venous	Blood Coag	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Blue	Coag	
702	ISTAT Venous	Blood	Blood Gases Venous	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	ISTAT	Chem	
703	JAK2 Mutation, QN, Leumeta	Blood	Chem lav sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	5.0	Lav	Ref Refrig	SUBMIT SAMPLES MON-
704	Kappa Lambda Light Chain	Serum	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	SST	Ref Refrig	Refrigerate
705	Kappa Lambda Total Random Urine	Urine	Chem Sendout	Urine	(ALL)	0 Minutes	150 Years	(ALL)	2.0	UA	Ref Refrig	Refrigerate
706	Kleinhauer Betke Stain	Blood	Blood Hem	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)	0.2	Lav (micro)	Hem	
707	Kleinhauer Betke Stain	Blood	Blood Hem	Venous Draw	(ALL)	1 Months	150 Years	(ALL)	0.5	Lav	Hem	
708	KOH Prep	Vaginal Exudates	Julian Date	Wash	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Sterile Plastic	Urinalysis	
709	KRAS Mutation Analysis	Blood	Chem lav sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	7.0	Lav	Ref Room	
710	KRAS Mutation Analysis	Paraffin Block	Sendout Molecular	Tissue	(ALL)	0 Minutes	150 Years	(ALL)	1.0	NoContainer	Ref Room	
711	L/S Ratio PG Amniotic Fluid	Amniotic Fluid	Body Fluid	Special Collection	(ALL)	0 Minutes	150 Years	(ALL)	3.0	Amber	Chem	Protect from light
712	Lab Miscellaneous Test	OTHER	Julian Date	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Contact Laboratory	Chem	
713	Lab Order IFE	Blood	Blood	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	SST	Chem	
714	Lab Order IFE Urine	Urine	Urine	Urine	Electropheresis	0 Minutes	150 Years	(ALL)	5.0	U Aliquot	Chem	
715	Lactate Dehydrogenase	Blood	Blood	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)	0.3	SST (micro)	Chem	
716	Lactate Dehydrogenase	Blood	Blood	Venous Draw	(ALL)	31 Days	150 Years	(ALL)	0.3	SST	Chem	
717	Lactic Acid	Blood	Blood Lactate	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.8	Grey	Chem	Deliver Immediately
718	Lactic Acid (Blood Gas)	Blood Heparin	Blood Gases Venous	Syringe	(ALL)	0 Minutes	150 Years	(ALL)	0.1	Hep Syringe	Chem	Deliver Immediately
719	Lamotrigine	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Red	Ref Refrig	Refrigerate
720	LAP	Blood	Blood Hem	Slide Prep	(ALL)	0 Minutes	150 Years	(ALL)	5.0	Slides	Hem	
721	LD Isoenzymes	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	6.0	SST	Ref Room	Room temperate
722	LD Pericardial Fluid	Pericardial Fluid	Body Fluid	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)	0.3	Red	Chem	
723	LD Pleural Fluid	Pleural Fluid	Body Fluid	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)	0.3	Red	Chem	
724	LD Renal Perfusate	Perfusate	Body Fluid	Syringe	(ALL)	0 Minutes	150 Years	(ALL)	0.5	Hep Syringe	Chem	
725	LE Latex	Blood	Hem Red	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	SST	Hem	
726	Lead	Venous	CHEM LAV	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	2.0	Tan	Chem	
727	Lead Urine	Urine	Urine	Urine	(ALL)	0 Minutes	150 Years	(ALL)	10.0	U24H	Ref Refrig	Refrigerate
728	Leflunomide as Metabolite	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	2.0	Red	Ref Refrig	Refrigerate

	Primary Synonym	Specimen Type	Accession Class	Collection Method	Service Resource	Age From	Age To	Coll Priority	Min Vol	Container	Collection Class	Special Handling
729	Leptin	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Red	Ref Refrig	Refrigerate
730	Leukemia/Lymph,Select Abs	Lymph node	Chem Sendout	Biopsy	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Sterile Screw	Ref Room	Room temperate
731	Leukemia/Lymph,Select Abs	CSF	Chem Sendout	Syringe	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Yellow (ACD)	Ref Room	Room temperate
732	Leukemia/Lymph,Select Abs	Blood	Chem lav sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	3.0	Lav	Ref Room	Room temperate
733	Leukemia/Lymph,Select Abs	Blood	Chem lav sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	3.0	Green Sodium	Ref Room	Room temperate
734	Leukemia/Lymph,Select Abs	Blood	Chem lav sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	2.0	Slides	Ref Room	Room temperate
735	Leukemia/Lymph,Select Abs	Bone Marrow	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	3.0	Green Sodium	Ref Room	Room temperate
736	Leukemia/Lymph,Select Abs	Bone Marrow	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	2.0	Slides	Ref Room	Room temperate
737	Leukemia/Lymphoma Evaluation	Blood	Blood	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Green Sodium	Chem	
738	Leukemia/Lymphoma Evaluation	Biopsy	Chem Sendout	Biopsy	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Sterile Plastic	Chem	Room temperate
739	Leukemia/Lymphoma Evaluation	Bone Marrow	Special Collectio	Slide	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Slides	Chem	
740	Leukemia/Lymphoma Evaluation	Blood	Blood	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	2.0	Slides	Chem	
741	Leukemia/Lymphoma Evaluation	Blood	Blood	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Lav	Chem	
742	Leukocytes Stool	Stool	Special Collectio	Slide Prep	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Slides	Urinalysis	
743	Levetiracetam (Keppra)	Blood	Chem Sendout Frozen	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Lav	Ref Frozen	Freeze
744	Lidocaine	Blood	Chem Sendout	Venous Draw	(ALL)	2 Minutes	150 Years	(ALL)	4.0	Red	Ref Refrig	Refrigerate
745	Lidocaine	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	1 Years	(ALL)	1.0	Red (micro)	Ref Refrig	Refrigerate
746	Lipase	Blood	Blood	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)	0.3	SST (micro)	Chem	
747	Lipase	Blood	Blood	Venous Draw	(ALL)	31 Days	150 Years	(ALL)	0.3	SST	Chem	
748	Lipid Profile	Blood	Blood	Venous Draw	(ALL)	31 Days	150 Years	(ALL)	1.0	SST	Chem	
749	Lipid Profile	Blood	Blood	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)	0.5	SST (micro)	Chem	
750	Lipoprotein (A)	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Red	Ref Refrig	Refrigerate
751	Lithium	Blood	Blood	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)	0.3	SST (micro)	Chem	Draw prior to next dose
752	Lithium	Blood	Blood	Venous Draw	(ALL)	31 Days	150 Years	(ALL)	0.3	SST	Chem	Draw prior to next dose
753	Liver Kidney Microsomal Antibody	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	3.0	Red	Ref Refrig	Refrigerate
754	Lupus Anticoagulant	Blood	Hem Blue Sendouts	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	3.0	Blue	Ref Frozen	Freeze
755	Luteinizing Hormone	Blood	Chem Architect	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	SST	SP Chem	
756	Lyme Disease IGG, IGM, WB	Blood	Blood	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	SST	Ref Refrig	
757	Lyme Disease,PCR,CSF/SYN	Synovial fluid	Special Collectio	Syringe	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Sterile Screw	Ref Refrig	
758	Lyme Disease,PCR,CSF/SYN	CSF	CSF	Syringe	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Sterile T	Ref Refrig	
759	Lysozyme	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	3.0	SST	Ref Refrig	Refrigerate
760	M Pneumoniae Antibody	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	4.0	Red	Ref Refrig	Refrigerate
761	M. Tuberculos, PCR,Non-Resp	Body Fluid	Body Fluid	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)	3.0	Sterile Screw	Ref Refrig	
762	M. Tuberculos, PCR,Non-Resp	Urine	Urine	Urine	(ALL)	0 Minutes	150 Years	(ALL)	5.0	UA	Ref Refrig	Refrigerate
763	M. Tuberculos, PCR,Non-Resp	CSF	CSF	Syringe	(ALL)	0 Minutes	150 Years	(ALL)	3.0	Sterile Screw	Ref Refrig	Refrigerate
764	MAG, Western Blot W/RFX	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Red	Ref Refrig	Refrigerate
765	Magnesium	Blood	Blood	Venous Draw	(ALL)	31 Days	150 Years	(ALL)	0.3	SST	Chem	
766	Magnesium	Blood	Blood	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)	0.3	SST (micro)	Chem	
767	Magnesium Urine 24Hr	Urine	Urine 24hr	Urine	(ALL)	0 Minutes	150 Years	(ALL)	2.0	U24H	Urine Chem	Refrigerate
768	Magnesium Urine Random	Urine	Urine	Urine	(ALL)	0 Minutes	150 Years	(ALL)	10.0	UA	Urine Chem	Refrigerate
769	Malarial Stain	Blood	Blood Hem	Venous Draw	(ALL)	1 Months	150 Years	(ALL)	0.5	Lav	Hem	
770	Malarial Stain	Blood	Blood Hem	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)	0.2	Lav (micro)	Hem	
771	Maternal Serum Quad Screen	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	8.0	SST	Ref Refrig	Refrigerate
772	Measles Antibody Immune Status IgG	Blood	VS	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	SST	Serology	
773	Mercury Blood	Blood	Chem Royal	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	4.0	Royal (EDTA)	Ref Refrig	Refrigerate
774	Mercury Urine	Urine	Urine	Urine	(ALL)	0 Minutes	150 Years	(ALL)	20.0	U Aliquot	Ref Refrig	Refrigerate
775	Metabolic Screen Neonatal	Capillary - heelstick	Special Collectio	Capillary	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Filter Paper	SP Chem	Fill Holes Completely
776	Metanepharines Urine 24Hr Fractionated	Urine	Urine	Urine	(ALL)	0 Minutes	150 Years	(ALL)	25.0	U24H HCL	Ref Refrig	Refrigerate
777	Metanephrines, Fract, Plas	Blood	Chem Sendout Frozen	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Lav	Ref Frozen	Freeze
778	Methadone Screen Urine	Urine	Urine	Urine	(ALL)	0 Minutes	150 Years	(ALL)	25.0	UA	Ref Room	Room temperate
779	Methemoglobin	Blood Heparin	Blood Gases Arterial	Arterial Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.3	Hep Syringe	Chem	Deliver Immediately
780	Methotrexate	Blood	Blood	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)	0.3	SST (micro)	Chem	
781	Methotrexate	Blood	Blood	Venous Draw	(ALL)	1 Months	150 Years	(ALL)	0.3	SST	Chem	Draw prior to next dose
782	Methotrexate CSF	CSF	CSF	Syringe	(ALL)	0 Minutes	150 Years	(ALL)	0.5	Sterile T	Chem	
783	Methsuximide	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	4.0	Red	Ref Refrig	Refrigerate
784	Methylmalonic Acid	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	3.0	SST	Ref Refrig	

	Primary Synonym	Specimen Type	Accession Class	Collection Method	Service Resource	Age From	Age To	Coll Priority	Min Vol	Container	Collection Class	Special Handling
785	Microalbumin 24Hr Urine	Urine	Urine 24hr	Urine	(ALL)	0 Minutes	150 Years	(ALL)	2.0	U24H	Urine Chem	
786	Microalbumin:Creatinine Ratio	Urine	Urine	Urine	(ALL)	0 Minutes	150 Years	(ALL)	1.0	UA	Urine Chem	
787	Microfilariae Detection	Blood	Chem lav sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	5.0	Lav	Chem	Room temperate
788	Mixing Studies APTT	Blood	Blood Coag	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	2.0	Blue	Coag	Mix Immediately
789	Mixing Studies PT	Blood	Blood Coag	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	2.0	Blue	Coag	Mix Immediately
790	Mol Dx Director Review	Blood	GC Genetics Cyxtic Fibrosis	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Lav	MolecularC	
791	Monosaccharides Urine	Urine	Urine	Urine	(ALL)	0 Minutes	150 Years	(ALL)	5.0	UA	Chem	Freeze
792	Motor Sensory Neuropathy Evaluation	Blood	Blood	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	10.0	SST	Chem	Refrigerate
793	MSKCC	Blood	Blood	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	20.0	Green	Chem	
794	MSKCC	Blood	Blood	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	20.0	SST	Chem	
795	MSKCC	Blood	Blood	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	3.0	Lav	Chem	
796	Mucchopolysaccharide Identification Urine	Urine	Urine	Urine	(ALL)	0 Minutes	150 Years	(ALL)	20.0	U24H	Chem	Freeze
797	Mucolipidosis Type IV, NY	Blood	Chem lav sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	3.0	Lav	Ref Room	Room temperate
798	Mucopolysaccharide SCR UR	Urine	Urine	Urine	(ALL)	0 Minutes	150 Years	(ALL)	20.0	U Aliquot	Ref Frozen	Freeze
799	Mumps Antibody Immune Status IgG	Blood	VS	Venous Draw	(ALL)	31 Days	150 Years	(ALL)	1.0	SST	Serology	
800	Mumps Antibody Immune Status IgG	Blood	VS	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)	1.0	SST	Serology	
801	Musk Antibody	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	2.0	SST	Ref Refrig	Refrigerate
802	Myeline Basic Protein CSF	CSF	CSF	Special Collection	(ALL)	0 Minutes	150 Years	(ALL)	2.0	Transport	Ref Refrig	Refrigerate
803	Myocardial Ab W/RFX	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Red	Ref Refrig	Refrigerate
804	Myoglobin Urine	Urine	Urine	Urine	(ALL)	0 Minutes	150 Years	(ALL)	10.0	UA	Urinalysis	
805	Natural Killer Cells	Blood	Flow Lav	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Lav	Flow	Room temperate
806	Neisseria gonorrhoeae DNA by PCR	Urethral Swab	VM	Dacron Swab	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Vir/Chlam	Virol	Refrigerate
807	Neisseria gonorrhoeae DNA by PCR	Cervical Swab	VM	Dacron Swab	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Vir/Chlam	Virol	Refrigerate
808	Neonatal T Bili Blood	Blood Heparin	Blood Gases Arterial	Syringe	(ALL)	0 Minutes	150 Years	(ALL)	0.3	Hep Syringe	Chem	Deliver Immediately
809	Neuromyelitis Optica Autoantibody, IgG, S	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	2.0	Red	Ref Refrig	Refrigerate
810	Neuronal Nucl(HU)Ab W/RFL	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Red	Ref Room	Room temperate
811	Neuronal Nucl(RI)Ab W/RFL	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Red	Ref Room	Room temperate
812	Nortriptyline	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	5.0	Red	Ref Refrig	Refrigerate
813	NRBC - LAB ORDER ONLY	Blood	Blood Hem	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Lav	Hem	
814	Nuclear Matrix Protein	Urine	Urine	Special Collection	(ALL)	0 Minutes	150 Years	(ALL)	10.0	Contact Laboratory	Ref Refrig	Refrigerate
815	Occult Blood Stool	Stool	Stool	Slide Prep	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Occult	Urinalysis	
816	Oligoclonal Banding CSF	OTHER	CSF	CSF and Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	2.0	Sterile Plastic	Chem	
817	Oligoclonal Banding CSF	OTHER	CSF	CSF and Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.5	SST	Chem	
818	Oligosaccharide Urine	Urine	Urine	Urine	(ALL)	0 Minutes	150 Years	(ALL)	20.0	U Aliquot	Ref Frozen	Freeze
819	Oncotic Pressure	Blood Heparin	Blood Gases Arterial	Syringe	(ALL)	0 Minutes	150 Years	(ALL)	0.1	Hep Syringe	Chem	Deliver Immediately
820	Oraquick HIV-1	Blood	VH	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	SST	Chem	
821	Organic Acid Qualitative Urine	Urine	Urine	Urine	(ALL)	0 Minutes	150 Years	(ALL)	20.0	U Aliquot	Chem	Freeze
822	Organic Acid, QN,UR,NY	Urine	Urine	Urine	(ALL)	0 Minutes	150 Years	(ALL)	20.0	U Aliquot	Chem	Freeze
823	Orotic Acid	Urine	Urine	Urine	(ALL)	0 Minutes	150 Years	(ALL)	10.0	U Aliquot	Chem	Freeze
824	Osmolality Renal Perfusate	Perfusate	Body Fluid	Syringe	(ALL)	0 Minutes	150 Years	(ALL)	0.3	Hep Syringe	Chem	
825	Osmolality Serum	Blood	Blood	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)	0.3	SST (micro)	Chem	
826	Osmolality Serum	Blood	Blood	Venous Draw	(ALL)	31 Days	150 Years	(ALL)	1.0	SST	Chem	
827	Osmolality Urine	Urine	Urine	Urine	(ALL)	0 Minutes	150 Years	(ALL)	2.0	UA	Urine Chem	
828	Oxalic Acid 24Hr Ur,W/Crea	Urine	Urine HCL	Urine	(ALL)	0 Minutes	150 Years	(ALL)	30.0	U24H HCL	Ref Refrig	Refrigerate
829	Oxalic Acid, 24Hr W/O Creat	Urine	Urine	Urine	(ALL)	0 Minutes	150 Years	(ALL)	30.0	U24H HCL	Ref Refrig	Refrigerate
830	Oxcarbazepine Metabolite	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Red	Ref Refrig	Refrigerate
831	Pancreatic Elastase-1	Stool	Stool	Stool	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Stool Container	Ref Frozen	Freeze
832	Parathyroid Hormone Intact	Blood	Blood PTH	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	5.0	SST	SP Chem	Collect prior to 10 AM fast
833	Parietal Cell Antibody	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	SST	Ref Refrig	Refrigerate
834	Paroxetine	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	5.0	Red	Ref Refrig	Refrigerate
835	Parvovirus B19 IgG IgM	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	2.0	SST	Ref Refrig	Refrigerate
836	Parvovirus B19 DNA, QL,PCR	Tissue	Chem Sendout Frozen	Tissue	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Sterile Plastic	Ref Frozen	Freeze
837	Parvovirus B19 DNA, QL,PCR	Blood	Chem Sendout Frozen	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Yellow (ACD)	Ref Frozen	Freeze
838	Parvovirus B19 DNA, QL,PCR	Bone Marrow	Chem Sendout	Special Collection	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Yellow (ACD)	Ref Refrig	Refrigerate
839	Pediatric Allergy Food Panel	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	12 Years	(ALL)	2.0	SST	Immune	
840	pH Body Fluid	Body Fluid	Body Fluid	Syringe	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Hep Syringe	Chem	

	Primary Synonym	Specimen Type	Accession Class	Collection Method	Service Resource	Age From	Age To	Coll Priority	Min Vol	Container	Collection Class	Special Handling
841	pH Stool	Stool	Stool	Special Collection	(ALL)	0 Minutes	150 Years	(ALL)	0.3	Stool Cx	Ref Frozen	Freeze
842	pH Arterial Blood	Arterial	Blood	Syringe	(ALL)	0 Minutes	150 Years	(ALL)	3.0	Hep Syringe	Chem	
843	pH Capillary Blood	Capillary - heelstick	Blood	Capillary	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Capillary	Chem	
844	pH Renal Perfusate	Perfusate	Special Collectio	Syringe	(ALL)	0 Minutes	150 Years	(ALL)	0.5	Hep Syringe	Chem	
845	pH Venous Blood	Blood	Blood	Syringe	(ALL)	0 Minutes	150 Years	(ALL)	3.0	Hep Syringe	Chem	
846	Phenobarbital	Blood	Blood	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)	0.3	SST (micro)	Chem	
847	Phenobarbital	Blood	Blood	Venous Draw	(ALL)	1 Months	150 Years	(ALL)	0.3	SST	Chem	Draw prior to next dose
848	Phenosense GT (TM), NY	Blood	Chem Sendout Frozen	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	3.0	PPT	Chem	Freeze
849	Phenosense HIV Comp, NY	Blood	Chem Sendout Frozen	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	3.0	PPT	Chem	Freeze
850	Phenylalanine, NY	Blood	Chem Sendout Frozen	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	5.0	Green	Ref Frozen	Freeze
851	Phenylalanine, NY	Blood Heparin	Chem Sendout Frozen	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	5.0	Green	Ref Frozen	Freeze
852	Phenytoin	Blood	Blood	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)	0.3	SST (micro)	Ref Refrig	
853	Phenytoin	Blood	Blood	Venous Draw	(ALL)	31 Days	150 Years	(ALL)	0.3	SST	Chem	Draw prior to next dose
854	Phenytoin Free	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	5.0	Red	Ref Refrig	Refrigerate
855	Phospholipids	Blood	Blood	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	3.0	Red	Ref Refrig	Refrigerate
856	Phosphorus	Blood	Blood	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)	0.2	SST (micro)	Chem	
857	Phosphorus	Blood	Blood	Venous Draw	(ALL)	31 Days	150 Years	(ALL)	0.2	SST	Chem	
858	Phosphorus Urine 24Hr	Urine	Urine 24hr	Urine	(ALL)	0 Minutes	150 Years	(ALL)	2.0	U24H	Urine Chem	Refrigerate
859	Phosphorus Urine Random	Urine	Urine	Urine	(ALL)	0 Minutes	150 Years	(ALL)	2.0	UA	Urine Chem	
860	PI-Linked Antigen, Blood	Blood ACD	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	7.0	Yellow (ACD)	Ref Room	Room temperate
861	Plasminogen	Blood	Blood	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	4.5	Blue	Ref Frozen	Freeze
862	Platelet Ab (IgG), Indirect	Blood	Hem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Red	Ref Refrig	Refrigerate
863	Platelet Ab Heparin Induced	Blood	Hem Red	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	2.0	Red	Ref Frozen	Freeze
864	Platelet Function Aspirin	Blood	Blood Coag	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Contact Laboratory	Coag	Initials/D&T:
865	Platelet Function P2Y12	Blood	Blood Coag	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Contact Laboratory	Coag	Initials/D&T:
866	Platelet Glycoprotein Ab	Blood	Hem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	6.0	Lav	Ref Room	Room temperate
867	POC ACT LR	Blood	Blood Coag	Syringe	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Plastic Syringe	Chem	
868	POC ACT PLUS	Blood	Blood Coag	Syringe	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Plastic Syringe	Chem	
869	POC Glucose	Blood	POC BLD	Capillary	(ALL)	0 Minutes	150 Years	(ALL)	0.2	Capillary	Chem	
870	POC Ilb_Illa	Blood	Blood	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Green	Ref Refrig	
871	POC Occult Blood	Stool	Stool	Stool	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Occult	Urinalysis	
872	POC P2Y12	Blood	POC BLD	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Blue	Chem	
873	POC Plat Funct Aspirin	Blood	POC BLD	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Blue	Chem	
874	POC Pregnancy Urine	Urine	POC Ur	Urine	(ALL)	0 Minutes	150 Years	(ALL)	1.0	UA	Urinalysis	
875	POC Urinalysis	Urine	Urinalysis	Urine	(ALL)	0 Minutes	150 Years	(ALL)	3.0	UA	Urinalysis	
876	Poliovirus Antibody	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	3.0	Red	Ref Refrig	Refrigerate
877	Porphobilinogen Quantitative 24Hour Urine	Urine	Urine 24hr	Urine	(ALL)	0 Minutes	150 Years	(ALL)	10.0	Amber	Ref Refrig	Protect from light
878	Porphobilinogen Urine Random	Urine	Urine	Urine	(ALL)	0 Minutes	150 Years	(ALL)	10.0	Amber	Ref Frozen	Freeze
879	Porphyrins Urine 24Hour	Urine	Urine 24hr	Urine	(ALL)	0 Minutes	150 Years	(ALL)	10.0	Amber	Ref Refrig	Protect from light
880	Post BUN	Blood	Post	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)	0.2	SST (micro)	Chem	
881	Post BUN	Blood	Post	Venous Draw	(ALL)	1 Minutes	150 Years	(ALL)	0.2	SST	Chem	
882	Post Creatinine	Blood	Post	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)	0.2	SST (micro)	Chem	
883	Post Creatinine	Blood	Post	Venous Draw	(ALL)	1 Months	150 Years	(ALL)	0.2	SST	Chem	
884	Post Dialysis Care Set	Blood	Post	Venous Draw	(ALL)	1 Minutes	150 Years	(ALL)	0.2	SST	Chem	
885	Post Dialysis Care Set	Blood	Post	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)	0.2	SST (micro)	Chem	
886	Post Dialysis Care Set	Blood	Post	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)	0.2	SST (micro)	Chem	
887	Post Dialysis Care Set	Blood	Post	Venous Draw	(ALL)	1 Minutes	150 Years	(ALL)	0.2	SST	Chem	
888	Potassium	Blood	Blood	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)	0.2	SST (micro)	Chem	
889	Potassium	Blood	Blood	Venous Draw	(ALL)	1 Months	150 Years	(ALL)	0.2	SST	Chem	
890	Potassium Dialysate	DIALY	Special Collectio	Special Collection	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Red	Chem	
891	Potassium Whole Blood	Blood Heparin	Blood Gases Arterial	Syringe	(ALL)	0 Minutes	150 Years	(ALL)	0.1	Hep Syringe	Chem	Deliver Immediately
892	Potassium Cardioplegia	Cardioplegia	Body Fluid	Pharmacy	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Red	Chem	
893	Potassium Renal Perfusate	Perfusate	Body Fluid	Syringe	(ALL)	0 Minutes	150 Years	(ALL)	0.3	Hep Syringe	Chem	
894	Potassium Urine 24Hr	Urine	Urine 24hr	Urine	(ALL)	0 Minutes	150 Years	(ALL)	2.0	U24H	Urine Chem	Refrigerate
895	Potassium TPN Pharmacy	TPN	Special Collectio	Pharmacy	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Red	Chem	
896	Potassium Urine Random	Urine	Urine	Urine	(ALL)	0 Minutes	150 Years	(ALL)	2.0	UA	Urine Chem	

	Primary Synonym	Specimen Type	Accession Class	Collection Method	Service Resource	Age From	Age To	Coll Priority	Min Vol	Container	Collection Class	Special Handling
897	Prealbumin	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	2.0	SST	Ref Refrig	Refrigerate
898	Pregnancy Test Urine	Urine	Urinalysis	Urine	(ALL)	0 Minutes	150 Years	(ALL)	5.0	UA	Urinalysis	
899	Pregnenolone	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Red	Ref Refrig	Refrigerate
900	Primidone	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)	0.3	SST (micro)	Ref Refrig	Refrigerate
901	Primidone	Blood	Chem Sendout	Venous Draw	(ALL)	31 Days	150 Years	(ALL)	0.3	SST	Ref Refrig	Refrigerate
902	Procainamide & NAPA	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)	0.3	Red (micro)	Ref Refrig	
903	Procainamide & NAPA	Blood	Chem Sendout	Venous Draw	(ALL)	31 Days	150 Years	(ALL)	0.3	Red	Ref Refrig	Draw prior to next dose
904	Progesterone	Blood	Chem E170	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	SST	SP Chem	
905	Prolactin	Blood	Chem Architect	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	SST	SP Chem	
906	Prostate Specific Antigen	Blood	Chem E170	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	SST	SP Chem	
907	Protein CSF	CSF	CSF	Syringe	(ALL)	0 Minutes	150 Years	(ALL)	0.5	Sterile T	Chem	Deliver Immediately
908	Protein Urine 24Hr	Urine	Urine 24hr	Urine	(ALL)	0 Minutes	150 Years	(ALL)	2.0	U24H	Urine Chem	Refrigerate
909	Protein Urine Random	Urine	Urine	Urine	(ALL)	0 Minutes	150 Years	(ALL)	2.0	UA	Urine Chem	
910	Protein C Functional	Blood Citrated	Hem Blue Sendouts	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	2.0	Blue	Ref Frozen	Freeze
911	Protein C, Antigen	Blood	Hem Blue Sendouts	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Blue	Ref Frozen	Freeze
912	Protein Electrophoresis CSF	CSF	CSF	Syringe	(ALL)	0 Minutes	150 Years	(ALL)	2.5	Sterile T	Chem	Deliver Immediately
913	Protein Electrophoresis Serum	Blood	Blood	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	2.0	SST	Chem	
914	Protein Electrophoresis Urine	Urine	Urine	Urine	(ALL)	0 Minutes	150 Years	(ALL)	8.0	UA	Chem	
915	Protein Pericardial Fluid	Pericardial Fluid	Body Fluid	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)	0.3	Red	Chem	
916	Protein Peritoneal Fluid	Peritoneal Fluid	Body Fluid	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)	0.3	Red	Chem	
917	Protein Pleural Fluid	Pleural Fluid	Body Fluid	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)	0.3	Red	Chem	
918	Protein S Antigen	Blood	Hem Blue Sendouts	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Blue	Ref Frozen	Freeze
919	Protein S Functional	Blood Citrated	Hem Blue Sendouts	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	3.0	Blue	Ref Frozen	Freeze
920	Protein S Panel	Blood	Blood	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	6.0	Blue	Ref Frozen	Freeze
921	Protein Synovial Fluid	Synovial fluid	Body Fluid	Syringe	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Red	Chem	
922	Protein Total	Blood	Blood	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)	0.2	SST (micro)	Chem	
923	Protein Total	Blood	Blood	Venous Draw	(ALL)	1 Months	150 Years	(ALL)	0.3	SST	Chem	
924	Protein Urine Timed	Urine	Urine	Urine	(ALL)	0 Minutes	150 Years	(ALL)	2.0	UA	Urine Chem	Refrigerate
925	Proteinase-3 Ab	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	SST	Ref Refrig	
926	Prothrombin Gene, NY	Blood	Blood	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Lav	Ref Room	Room temperate
927	Prothrombin Time	Blood	Blood Coag	Venous Draw	(ALL)	0 Minutes	12 Years	(ALL)	0.5	Blue (Peds)	Coag	Mix Immediately
928	Prothrombin Time	Blood	Blood Coag	Venous Draw	(ALL)	12 Years	150 Years	(ALL)	0.5	Blue	Coag	Mix Immediately
929	PSA Free	Blood	Chem E170	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	SST	SP Chem	
930	PTH, Related Protein	Blood	Special Collectio	Special Collection	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Contact Laboratory	Ref Frozen	Freeze
931	Purkinje Cells Ab W/RFX	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Red	Ref Room	Room temperate
932	Pyruvate Kinase	Blood EDTA	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	3.0	Yellow (ACD)	Ref Refrig	Refrigerate
933	Pyruvic Acid	Blood	Chem Sendout Frozen	Special Collection	(ALL)	0 Minutes	150 Years	(ALL)	5.0	Perchloric Acid	Ref Frozen	Freeze
934	Quinidine	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)	0.3	Red (micro)	Ref Refrig	
935	Quinidine	Blood	Chem Sendout	Venous Draw	(ALL)	31 Days	150 Years	(ALL)	0.3	Red	Ref Refrig	Draw prior to next dose
936	Reducing Substances	Urine	Urinalysis	Urine	(ALL)	0 Minutes	150 Years	(ALL)	0.5	UA	Urinalysis	
937	Remove Any Test Not Needed	OTHER	Blood	Special Collection	(ALL)	0 Minutes	150 Years	(ALL)	1.0	NoContainer	Chem	
938	Renal Function Panel	Blood	Blood	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	SST	Chem	
939	Renin	Blood	Chem Sendout Frozen	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	3.0	Lav	Ref Frozen	Freeze
940	Renin Act/Aldosterone(P)	Blood	Chem Sendout Frozen	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	3.0	Lav	Ref Frozen	Freeze
941	Repo Exposure Panel	Blood	VS	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Red	Serology	Freeze
942	Reptilase Time	Blood	Blood Coag	Venous Draw	(ALL)	0 Minutes	12 Years	(ALL)	0.5	Blue	Coag	Mix Immediately
943	Reptilase Time	Blood	Blood Coag	Venous Draw	(ALL)	12 Years	150 Years	(ALL)	0.5	Blue	Coag	Mix Immediately
944	Research Phlebotomy	Blood	POC BLD	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	SST	Chem	
945	Research Processing	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	SST	Chem	Freeze
946	Reticulin Ab (IgA) W/RFX	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Red	Ref Refrig	Refrigerate
947	Reticulin Ab (IgG)	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	SST	Ref Refrig	Refrigerate
948	Reticulocyte Count	Blood	Blood Hem	Venous Draw	(ALL)	1 Months	150 Years	(ALL)	1.0	Lav	Hem	
949	Reticulocyte Count	Blood	Blood Hem	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)	0.2	Lav (micro)	Hem	
950	Reticulocyte Count Manual	Blood	Blood Hem	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.5	Lav	Hem	
951	Retinol Binding Protein	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	6.0	Red	Ref Refrig	Refrigerate
952	Rheumatoid Factor Serum	Blood	Blood	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.5	SST	Chem	

	Primary Synonym	Specimen Type	Accession Class	Collection Method	Service Resource	Age From	Age To	Coll Priority	Min Vol	Container	Collection Class	Special Handling
953	Rickettsial Ab PNL W/REFL	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	2.0	Red	Ref Refrig	Refrigerate
954	Rocky MT Spot (G,M)W/RFX	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	3.0	SST	Ref Refrig	Refrigerate
955	RPR Serum	Blood	VS	Venous Draw	(ALL)	1 Months	150 Years	(ALL)	0.2	SST	Serology	
956	RPR Serum	Blood	VS	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)	0.2	SST	Serology	
957	RPR Serum	Cord Blood	VS	Special Collection	(ALL)	0 Minutes	150 Years	(ALL)	1.0	SST	Serology	
958	Rubella Antibodies IgM	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	5.0	SST	Ref Refrig	Refrigerate
959	Rubella Antibody Immune Status IgG by E	Blood	VS	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.3	SST	Serology	Room temperate
960	S. Cerevisiae Ab (IgG)	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Red	Ref Room	Room temperate
961	S.PNEUMONIAE IGG (14)	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Red	Ref Refrig	Refrigerate
962	Salicylate	Blood	Blood	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)	0.3	SST (micro)	Chem	
963	Salicylate	Blood	Blood	Venous Draw	(ALL)	1 Months	150 Years	(ALL)	0.3	SST	Chem	Draw prior to next dose
964	Salvaged Potassium	Blood	Blood	Venous Draw	(ALL)	1 Months	150 Years	(ALL)	0.2	SST	Chem	
965	Salvaged Potassium	Blood	Blood	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)	0.2	SST	Chem	
966	Salvaged Hematocrit	Blood	Blood Hem	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)	0.2	Lav (micro)	Hem	
967	Salvaged Hematocrit	Blood	Blood Hem	Venous Draw	(ALL)	1 Months	150 Years	(ALL)	0.5	Lav	Hem	
968	Salvaged PTT	Blood	Blood Coag	Venous Draw	(ALL)	0 Minutes	12 Years	(ALL)	0.5	Blue (Peds)	Coag	
969	Salvaged PTT	Blood	Blood Coag	Venous Draw	(ALL)	12 Minutes	150 Years	(ALL)	0.5	Blue	Coag	
970	Selenium	Blood	Chem Sendout Frozen	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	2.0	Royal (Serum)	Ref Frozen	Freeze
971	Semen Analysis	Body Fluid	Body Fluid	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Sterile Plastic	Chem	Room temperate
972	Serotonin Release Assay	Blood	Chem Sendout Frozen	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	2.0	Red	Ref Frozen	Freeze
973	Serotonin, Serum	Blood	Chem Sendout Frozen	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	5.0	Red	Ref Frozen	Freeze
974	Sex Hormone Binding Globulin	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	6.0	SST	Ref Refrig	Refrigerate
975	Sickle Cell Prep	Blood	Blood Hem	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)	0.2	Lav (micro)	Hem	
976	Sickle Cell Prep	Blood	Blood Hem	Venous Draw	(ALL)	1 Months	150 Years	(ALL)	1.0	Lav	Hem	
977	Sickle Solubility Test	Blood	Blood Hem	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)	0.2	Lav (micro)	Hem	
978	Sickle Solubility Test	Blood	Blood Hem	Venous Draw	(ALL)	1 Months	150 Years	(ALL)	1.0	Lav	Hem	
979	Single Site BRAC Analysis	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	3.0	Contact Laboratory	Chem	Room temperate
980	Sirolimus	Blood EDTA	Immuno Lav	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	2.0	Lav	Chem	Draw prior to next dose
981	Sodium	Blood	Blood	Venous Draw	(ALL)	0 Minutes	30 Months	(ALL)	0.2	SST (micro)	Chem	
982	Sodium	Blood	Blood	Venous Draw	(ALL)	31 Days	150 Years	(ALL)	0.2	SST	Chem	
983	Sodium Dialysate	DIALY	Special Collectio	Special Collection	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Red	Chem	
984	Sodium Urine 24Hr	Urine	Urine 24hr	Urine	(ALL)	0 Minutes	150 Years	(ALL)	2.0	U24H	Urine Chem	Refrigerate
985	Sodium Blood	Blood Heparin	Blood Gases Arterial	Syringe	(ALL)	0 Minutes	150 Years	(ALL)	0.1	Hep Syringe	Chem	Deliver Immediately
986	Sodium Renal Perfusate	Perfusate	Body Fluid	Syringe	(ALL)	0 Minutes	150 Years	(ALL)	0.3	Hep Syringe	Chem	
987	Sodium TPN Pharmacy	TPN	Special Collectio	Pharmacy	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Red	Chem	
988	Sodium Urine Random	Urine	Urine	Urine	(ALL)	0 Minutes	150 Years	(ALL)	2.0	UA	Urine Chem	
989	Specific Gravity	Urine	Urinalysis	Urine	(ALL)	0 Minutes	150 Years	(ALL)	0.5	UA	Urinalysis	
990	Specific Gravity Fluid	Body Fluid	Body Fluid	Syringe	(ALL)	0 Minutes	150 Years	(ALL)	0.5	Lav	Urinalysis	
991	Streptomycin, HPLC,Ser	CSF	Chem Sendout Frozen	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Sterile Plastic	Ref Frozen	Freeze
992	Streptomycin, HPLC,Ser	Blood	Chem Sendout Frozen	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	2.0	Red	Ref Frozen	Freeze
993	Sugar Water Test	Blood	Hem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.5	Red	Hem	
994	Sugar Water Test	Blood	Hem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	2.0	Blue	Hem	
995	Sulfate, 24Hr Urine	Urine	Urine	Urine	(ALL)	0 Minutes	150 Years	(ALL)	10.0	U24H	Ref Refrig	Refrigerate
996	Syphilis Confirmation	Blood	VS	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	SST	Serology	
997	T and B Cell Panel	Blood	Flow Lav	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Lav	Flow	Room temperate
998	T Cell Immune Function-ATP	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	3.0	Green	Ref Room	Contact Laboratory
999	T Cell Panel	Blood	Flow Lav	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Lav	Flow	Room temperate
1000	T, B and NK Cell Panel	Blood	Flow Lav	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.5	Lav	Flow	Room temperate
1001	T. Vaginalis Culture	Genital Swab	Chem Sendout	Swab	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Contact Laboratory	Chem	Room temperate
1002	T3	Blood	Chem Architect	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.5	SST	SP Chem	
1003	T3 Free	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	3.0	SST	Ref Refrig	Refrigerate
1004	T3 Uptake	Blood	Chem Architect	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.5	SST	SP Chem	
1005	T4 Free	Blood	Chem Architect	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.5	SST	SP Chem	
1006	T4 Free by Equilibrium Dialysis	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	5.0	SST	Ref Refrig	Refrigerate
1007	TBG	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	2.0	SST	Ref Refrig	Refrigerate
1008	TCR Rearrange, QL, Cell	Bone Marrow	Chem lav sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Lav	Ref Refrig	Refrigerate

	Primary Synonym	Specimen Type	Accession Class	Collection Method	Service Resource	Age From	Age To	Coll Priority	Min Vol	Container	Collection Class	Special Handling
1009	TCR Rearrange,QN,Cell	Bone Marrow	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	3.0	Lav	Ref Refrig	Refrigerate
1010	TCR Rearrange,QN,Cell	Blood	Chem lav sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	3.0	Lav	Ref Refrig	Refrigerate
1011	Testosterone	Blood	Chem E170	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	SST	SP Chem	
1012	Testosterone Free	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	3.0	SST	Ref Refrig	Refrigerate
1013	Tetanus Toxoid Antibodies	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	3.0	SST	Ref Refrig	Refrigerate
1014	Theophylline	Blood	Blood	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.3	SST	Chem	Draw prior to next dose
1015	Thiocyanate	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.3	SST	Ref Room	Room temperate
1016	Thioridazine	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.3	SST	Ref Room	Room temperate
1017	Thrombin Time	Blood	Blood Coag	Venous Draw	(ALL)	0 Minutes	12 Years	(ALL)	0.5	Blue (Peds)	Coag	Mix Immediately
1018	Thrombin Time	Blood	Blood Coag	Venous Draw	(ALL)	12 Years	150 Years	(ALL)	0.5	Blue	Coag	Mix Immediately
1019	Thyroglobulin Ab	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	4.0	SST	Ref Refrig	Refrigerate
1020	Thyroglobulin Panel	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	2.0	SST	Ref Refrig	Refrigerate
1021	Thyroid Antibodies	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	4.0	SST	Ref Refrig	Refrigerate
1022	Thyroid Stimulating Hormone	Blood	Chem Architect	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	SST	SP Chem	
1023	Thyroid Stimulating Immunoglobulin	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	SST	Ref Refrig	Refrigerate
1024	Thyrotropin Receptor Antibody	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.3	Red	Ref Refrig	Refrigerate
1025	Thyroxine	Blood	Chem Architect	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.5	SST	SP Chem	
1026	Tiagabine(Gabitril)	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	2.0	Red	Ref Refrig	Refrigerate
1027	Tick, Other Arthropods, ID	OTHER	Chem Sendout	Special Collection	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Contact Laboratory	Ref Room	Room temperate
1028	Tissue Transglutam Ab IgA	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Red	Ref Refrig	Refrigerate
1029	Tissue Transglutam Ab IgG	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Red	Ref Refrig	Refrigerate
1030	Tobramycin Peak	Serum	Peak	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)	0.3	SST (micro)	Chem	
1031	Tobramycin Peak	Serum	Peak	Venous Draw	(ALL)	31 Days	150 Years	(ALL)	0.3	SST	Chem	
1032	Tobramycin Random	Blood	Blood	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)	0.3	SST (micro)	Chem	
1033	Tobramycin Random	Blood	Blood	Venous Draw	(ALL)	31 Days	150 Years	(ALL)	0.3	SST	Chem	
1034	Tobramycin Trough	Serum	Trough	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)	0.3	SST (micro)	Chem	
1035	Tobramycin Trough	Serum	Trough	Venous Draw	(ALL)	31 Days	150 Years	(ALL)	0.3	SST	Chem	
1036	Topiramate	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Red	Ref Refrig	Refrigerate
1037	Toxicology Screen Urine	Urine	KCH Sendout	Urine	(ALL)	0 Minutes	150 Years	(ALL)	15.0	U Aliquot	Chem	
1038	Toxoplasma Antibodies IgG & IgM	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	2.0	SST	Ref Refrig	Refrigerate
1039	Toxoplasmosis AB(IGG),CSF	CSF	CSF	Syringe	(ALL)	0 Minutes	150 Years	(ALL)	0.5	Sterile T	Ref Refrig	Deliver Immediately
1040	Toxoplasmosis gondii Antibodies IgG	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	SST	Ref Refrig	Refrigerate
1041	TPMT Genotype (NY)	Blood	Chem lav sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	3.0	Lav	Ref Room	Room temperate
1042	Transferrin	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.3	SST	Ref Refrig	
1043	Triglycerides	Blood	Blood	Venous Draw	(ALL)	1 Months	150 Years	(ALL)	0.3	SST	Chem	
1044	Triglycerides	Blood	Blood	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)	0.2	SST (micro)	Chem	
1045	Triglycerides Peritoneal Fluid	Peritoneal Fluid	Body Fluid	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)	0.3	Red	Chem	
1046	Triglycerides Pleural Fluid	Pleural Fluid	Body Fluid	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)	0.3	Red	Chem	
1047	Troponin I	Blood	Blood Troponin	Venous Draw	(ALL)	31 Days	150 Years	(ALL)	0.3	SST	Chem	
1048	Troponin I	Blood	Blood Troponin	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)	0.3	SST (micro)	Chem	
1049	Trypsin	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Red	Ref Refrig	Refrigerate
1050	Tryptase	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Red	Ref Refrig	Refrigerate
1051	Type II Collagen Ab	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	2.0	Red	Ref Refrig	Refrigerate
1052	Unigold HIV-1	Blood	VH	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	SST	Chem	
1053	Urea Nitrogen Urine 24Hr	Urine	Urine 24hr	Urine	(ALL)	0 Minutes	150 Years	(ALL)	2.0	U24H	Urine Chem	Refrigerate
1054	Urea Nitrogen Urine Random	Urine	Urine	Urine	(ALL)	0 Minutes	150 Years	(ALL)	2.0	UA	Urine Chem	
1055	Urea Nitrogen Peritoneal Dialysate	DIALY	Body Fluid	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)	0.5	Red	Chem	
1056	Uric Acid	Blood	Blood	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)	0.3	SST (micro)	Chem	
1057	Uric Acid	Blood	Blood	Venous Draw	(ALL)	31 Days	150 Years	(ALL)	0.3	SST	Chem	
1058	Uric Acid Urine 24 Hr	Urine	Urine 24hr	Urine	(ALL)	0 Minutes	150 Years	(ALL)	2.0	U24H	Urine Chem	Refrigerate
1059	Uric Acid Urine Random	Urine	Urine	Urine	(ALL)	0 Minutes	150 Years	(ALL)	2.0	UA	Urine Chem	
1060	Urinalysis	Urine	Urinalysis	Urine	(ALL)	0 Minutes	150 Years	(ALL)	3.0	UA	Urinalysis	
1061	Urinalysis with Microscopic	Urine	Urinalysis	Urine	(ALL)	0 Minutes	150 Years	(ALL)	2.0	UA	Urinalysis	
1062	Urine Microscopic	Urine	Urinalysis	Urine	(ALL)	0 Minutes	150 Years	(ALL)	5.0	UA	Urinalysis	
1063	URR	Blood	Post	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)	0.2	SST (micro)	Chem	
1064	URR	Blood	Post	Venous Draw	(ALL)	1 Months	150 Years	(ALL)	0.2	SST	Chem	

	Primary Synonym	Specimen Type	Accession Class	Collection Method	Service Resource	Age From	Age To	Coll Priority	Min Vol	Container	Collection Class	Special Handling
1065	Vaginal Wet Prep	Vaginal Exudates	Julian Date	Wash	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Sterile Plastic	Urinalysis	
1066	Valproic Acid	Blood	Blood	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)	0.3	SST (micro)	Chem	
1067	Valproic Acid	Blood	Blood	Venous Draw	(ALL)	31 Days	150 Years	(ALL)	0.3	SST	Chem	Draw prior to next dose
1068	Valproic Acid, Free	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Red	Ref Refrig	Refrigerate
1069	Vancomycin Peak	Serum	Peak	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.3	SST	Chem	
1070	Vancomycin Random	Blood	Blood	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.3	SST	Chem	
1071	Vancomycin Trough	Serum	Trough	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.3	SST	Chem	
1072	VAP (TM) Cholesterol Test	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	5.0	Red	Ref Refrig	Refrigerate
1073	Varicella Antibody Immune Status IgG	Blood	VS	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	SST	Serology	
1074	Varicella Zoster IgM	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	2.0	SST	Ref Refrig	Refrigerate
1075	VDRL CSF	CSF	VS	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)	0.5	Sterile Screw	Serology	Deliver Immediately
1076	Venous Blood Gas OR	Blood Heparin	Blood Gases Venous	Syringe	(ALL)	0 Minutes	150 Years	(ALL)	3.0	Hep Syringe	Chem	
1077	Viscosity, Serum	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	3.0	SST	Ref Refrig	Refrigerate
1078	Vitamin A	Blood	Blood	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.3	SST	Ref Refrig	Protect from light
1079	Vitamin B1, Plasma	Blood	Chem Sendout Frozen	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	2.0	Lav	Ref Frozen	Protect from light
1080	Vitamin B12	Blood	Chem E170	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	SST	SP Chem	
1081	Vitamin B6	Blood	Chem Sendout Frozen	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Lav	Ref Frozen	Protect from light
1082	Vitamin C	Blood	Chem Sendout Frozen	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Red	Ref Frozen	Protect from light
1083	Vitamin D 25 Hydroxy	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	SST	Ref Refrig	Refrigerate
1084	Vitamin D 1 25 Dihydroxy	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	3.0	SST	Ref Refrig	Refrigerate
1085	Vitamin E	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.3	Red	Ref Refrig	Protect from light
1086	Vitamin K	Plasma	Chem Sendout Frozen	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Lav	Ref Frozen	Protect from light
1087	VMA Urine 24 hr	Urine	Urine 24hr	Urine	(ALL)	0 Minutes	150 Years	(ALL)	30.0	U24H HCL	Ref Refrig	Refrigerate
1088	Von Willerbrand Factor Ag	Plasma	Hem Blue Sendouts	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	2.0	Blue	Ref Frozen	Freeze
1089	VWF Ag, Multimeric	Blood	Hem Blue Sendouts	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Blue	Ref Frozen	Freeze
1090	VWF Protease ACT w RFL INH	Plasma	Hem Blue Sendouts	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Blue	Ref Frozen	Freeze
1091	VWF, Ristocetin Co-Factor	Blood	Hem Blue Sendouts	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	2.0	Blue	Ref Frozen	Freeze
1092	WBC Man Diff	Blood	Blood Hem	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.5	Lav	Hem	
1093	West Nile Virus	CSF	VC CSF	Syringe	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Sterile Screw	Serology	
1094	West Nile Virus	Blood	VC	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Red	Serology	
1095	Whole Blood Glucose	Blood Heparin	Blood Gases Arterial	Syringe	(ALL)	0 Minutes	150 Years	(ALL)	0.1	Hep Syringe	Chem	Deliver Immediately
1096	Whole Blood Lytes	Blood Heparin	Blood Gases Arterial	Arterial Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.3	Hep Syringe	Chem	Deliver Immediately
1097	wwABID Ref Lab	Blood	Blood Bank	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	6.0	Pink	BldBank	
1098	XX_CF Carrier	Blood	GC Genetics Cyxtic Fibrosis	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	5.0	Lav	MolecularC	
1099	xxKit/V	Blood	Post	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	SST	Chem	
1100	Xylose Tolerance 5 hr	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	5.0	Grey	Ref Refrig	Refrigerate
1101	Xylose,Blood,1Hour Post	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	5.0	Grey	Ref Refrig	Refrigerate
1102	Xylose,Blood,2Hour Post	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	5.0	Grey	Ref Refrig	Refrigerate
1103	Zinc Protoporphyrin (ZPP)	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Lav	Ref Refrig	Protect from light
1104	Zinc Urine	Urine	Urine	Urine	(ALL)	0 Minutes	150 Years	(ALL)	10.0	Contact Laboratory	Ref Refrig	
1105	Zinc, Plasma	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Royal (EDTA)	Ref Room	Room temperate
1106	ZZCreat	Urine	Urine 24hr	Urine	(ALL)	0 Minutes	150 Years	(ALL)	1.0	U24H	Urine Chem	Refrigerate



BLOOD BANK

	Primary Synonym	Specimen Type	Accession Class	Collection Method	Service Resource	Age From	Age To	Coll Priority	Min Vol	Container	Collection Class	Special Handling
1	BB Cord Information	Cord Blood	Blood Bank	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	NoContainer	BldBank	
2	BB Requested ABO	Blood	Blood Bank	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	6.0	Pink	BldBank	Initials/D&T:
3	BB Requested ABO	Blood	Blood Bank	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	BB Form	BldBank	Initials/D&T:
4	Cryoprecipitate	BB Product	Blood Bank	Special Collection	(ALL)	0 Minutes	150 Years	(ALL)	1.0	BB Form	BldBank	Initials & Date:
5	Derivatives	BB Product	Blood Bank	Special Collection	(ALL)	0 Minutes	150 Years	(ALL)	1.0	BB Form	BldBank	Initials & Date:
6	FFP	BB Product	Blood Bank	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Pink	BldBank	Initials/D&T:
7	Granulocytes	BB Product	Blood Bank	Special Collection	(ALL)	0 Minutes	150 Years	(ALL)	1.0	BB Form	BldBank	Initials & Date:
8	Platelets	BB Product	Blood Bank	Special Collection	(ALL)	0 Minutes	150 Years	(ALL)	1.0	BB Form	BldBank	Initials & Date:
9	Red Blood Cell	BB Product	Blood Bank	Special Collection	(ALL)	0 Minutes	150 Years	(ALL)	1.0	BB Form	BldBank	Initials & Date:
10	Rhogam	BB Product	Blood Bank	Special Collection	(ALL)	0 Minutes	150 Years	(ALL)	1.0	BB Form	BldBank	Initials & Date:
11	Transfusion Reaction	Blood	Blood Bank	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.1	Pink	BldBank	Initials/D&T:
12	wwABO/Rh	Blood	Blood Bank	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	6.0	Pink	BldBank	Initials/D&T:
13	wwAntibody Identification	Blood	Blood Bank	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.1	Pink	BldBank	Initials/D&T:
14	wwAntibody Screen	Blood	Blood Bank	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	6.0	Pink	BldBank	Initials/D&T:
15	wwAntigen Testing	Blood	Blood Bank	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.1	Pink	BldBank	Initials/D&T:
16	wwBlood Type ABO	Blood	Blood Bank	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.1	Pink	BldBank	Initials/D&T:
17	wwCoombs Crossmatch	Blood	Blood Bank	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.1	Pink	BldBank	Initials/D&T:
18	wwCord Blood ABO/Rh	Blood	Blood Bank	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.1	Pink	BldBank	Initials/D&T:
19	wwCord Blood ABO/Rh	Cord Blood	Blood Bank	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.1	Pink	BldBank	Initials/D&T:
20	wwCord Blood Direct Coombs	Cord Blood	Blood Bank	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	6.0	Pink	BldBank	Initials/D&T:
21	wwCord Elution	Cord Blood	Blood Bank	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)	0.1	Pink	BldBank	Initials/D&T:
22	wwData Only Trans Rxn	Blood	Blood Bank	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.1	BB Form	BldBank	Initials/D&T:
23	wwDifferential Direct Coombs	Blood	Blood Bank	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.1	Pink	BldBank	Initials/D&T:
24	wwDirect Coombs	Blood	Blood Bank	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.1	Pink	BldBank	Initials/D&T:
25	wwEluate Crossmatch	Blood	Blood Bank	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.1	Pink	BldBank	Initials/D&T:
26	wwElution	Blood	Blood Bank	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.1	Pink	BldBank	Initials/D&T:
27	wwExtra Antigen Charge	Blood	Blood Bank	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.1	Pink	BldBank	Initials/D&T:
28	wwFetal Screen	Blood	Blood Bank	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.1	Pink	BldBank	Initials/D&T:
29	wwIS Crossmatch	Blood	Blood Bank	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.1	Pink	BldBank	Initials/D&T:
30	wwInhibition Test	Blood	Blood Bank	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.1	Pink	BldBank	Initials/D&T:
31	wwLab Only - Data for Hemo TXRXN	Blood	Blood Bank	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.1	BB Form	BldBank	Initials/D&T:
32	wwLab Only - Hemolytic TXRXN	Blood	Blood Bank	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.1	BB Form	BldBank	Initials/D&T:
33	wwLectin Identification Test	Blood	Blood Bank	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.1	Pink	BldBank	Initials/D&T:
34	wwNewborn ABO/Rh	Blood	Blood Bank	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.1	Lav (micro)	BldBank	Initials/D&T:
35	wwNewborn Blood DAT	Blood	Blood Bank	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.1	Lav (micro)	BldBank	Initials/D&T:
36	wwPhenotype	Blood	Blood Bank	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.1	Pink	BldBank	Initials/D&T:
37	wwPost Txn Rxn ABO/Rh	Blood	Blood Bank	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.1	Pink	BldBank	Initials/D&T:
38	wwPost Txn Rxn ABSC	Blood	Blood Bank	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.1	Pink	BldBank	Initials/D&T:
39	wwPost Txn Rxn Crossmatch	Blood	Blood Bank	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.1	Pink	BldBank	Initials/D&T:
40	wwPost Txn Rxn DAT	Blood	Blood Bank	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.1	Pink	BldBank	Initials/D&T:
41	wwPost Txn Rxn Differential	Blood	Blood Bank	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.1	Pink	BldBank	Initials/D&T:
42	wwPost Txn Rxn Elution	Blood	Blood Bank	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.1	Pink	BldBank	Initials/D&T:
43	wwPre Txn Rxn ABO/Rh	Blood	Blood Bank	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.1	Pink	BldBank	Initials/D&T:
44	wwPre Txn Rxn ABSC	Blood	Blood Bank	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.1	Pink	BldBank	Initials/D&T:
45	wwPre Txn Rxn Crossmatch	Blood	Blood Bank	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.1	Pink	BldBank	Initials/D&T:
46	wwPre Txn Rxn DAT	Blood	Blood Bank	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.1	Pink	BldBank	Initials/D&T:
47	wwPre Warm Crossmatch	Blood	Blood Bank	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.1	Pink	BldBank	Initials/D&T:
48	wwRetype	Blood	Blood Bank	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	5.0	Pink	BldBank	Initials/D&T:
49	wwRetype	Blood	Blood Bank	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	BB Form	BldBank	Initials/D&T:
50	wwRh Phenotype	Blood	Blood Bank	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.1	Pink	BldBank	Initials/D&T:
51	wwTXRXN Interp	Blood	Blood Bank	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.1	BB Form	BldBank	Initials/D&T:
52	wwTiter	Blood	Blood Bank	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.1	Pink	BldBank	Initials/D&T:
53	wwTrans Rxn Unit Confirm	Blood	Blood Bank	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.1	Pink	BldBank	Initials/D&T:
54	wwUnit Confirmation	Blood	Blood Bank	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.1	Pink	BldBank	Initials/D&T:
55	wwUnit Retype	Blood	Blood Bank	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	0.1	Pink	BldBank	Initials/D&T:



MICROBIOLOGY

	Primary Synonym	Specimen Type	Accession Class	Collection Method	Service Resource	Age From	Age To	Coll Priority	Min Vol	Container	Collection Class	Special Handling
1	AFB Culture	CSF	MB AFB CSF	Special Collection	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Sterile Plastic	Micro	Deliver Immediately
2	AFB Culture	Sputum	MB AFB Sputum	Sputum	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Sterile Plastic	Micro	
3	AFB Culture	Wound	MB AFB Wound	Micro Swab	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Micro Swab	Micro	
4	AFB Culture	Synovial fluid	MB Synovial	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Sterile T	Micro	
5	AFB Culture	Blood	MB AFB Blood	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Yellow (MAI)	Micro	
6	AFB Culture	Urine	MB AFB Urine	Urine	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Sterile Screw	Micro	
7	AFB Culture	Stool	MB AFB Stool	Stool	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Sterile Plastic	Micro	
8	AFB Culture	Nasopharyn Wash	MB AFB	Wash	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Sterile Plastic	Micro	
9	AFB Culture	Bronchial wash	MB AFB Bronchial Wash	Wash	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Sterile Plastic	Micro	
10	AFB Culture	Pleural Fluid	MB AFB Pleural	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Sterile T	Micro	
11	AFB Culture	Pericardial Fluid	MB AFB Pericardial fluid	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Sterile T	Micro	
12	AFB Culture	Bronchoalv lavage	MB AFB Bronchalevo lavage	Lavage	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Sterile Plastic	Micro	
13	AFB Culture	Ascites Fluid	MB AFB Ascites fluid	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Sterile T	Micro	
14	AFB Culture	Tracheal aspirate	MB AFB Tracheal aspirate	Aspirate	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Sterile Plastic	Micro	
15	AFB Culture	Nasophargeal Asp	MB AFB	Aspirate	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Sterile T	Micro	
16	AFB Culture	Joint Aspirate	MB AFB Joint asp	Aspirate	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Sterile T	Micro	
17	AFB Culture	Brain Bx	MB AFB Brain bx	Tissue	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Sterile Plastic	Micro	
18	AFB Culture	Bone Marrow Asp	MB AFB Bone marrow asp	Aspirate	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Sterile T	Micro	
19	AFB Culture	Lung Bx	MB AFB	Tissue	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Sterile Plastic	Micro	
20	AFB Culture	Lesion	MB AFB	Micro Swab	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Micro Swab	Micro	
21	AFB Culture	Lung asp	MB AFB Lung asp	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Sterile Plastic	Micro	
22	AFB Culture	Bronchial bx	MB AFB Bronchial bx	Tissue	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Sterile Plastic	Micro	
23	AFB Culture	Transtrach asp	MB AFB Transtracheal asp	Sputum	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Sterile Plastic	Micro	
24	AFB Culture	Abcess Drainage	MB AFB Abscess drainage	Micro Swab	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Micro Swab	Micro	
25	AFB Culture	Peritoneal Fluid	MB AFB Peritoneal fluid	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Sterile T	Micro	
26	AFB Culture	OTHER	MB AFB Other	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Sterile Plastic	Micro	
27	Abscess Culture	Swab	MB Abscess	Micro Swab	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Micro Swab	Micro	
28	Abscess Culture	Abcess Drainage	MB Abscess	Micro Swab	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Micro Swab	Micro	
29	Abscess Culture	OTHER	MB Abscess	Micro Swab	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Micro Swab	Micro	
30	Amniotic Fluid Culture	Amniotic Fluid	MB Wound	Micro Swab	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Sterile Plastic	Micro	
31	Ascites Fluid Culture	Ascites Fluid	MB Ascites	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)	2.0	Bld Cx (2 Bottles)	Micro	
32	Autopsy Culture	Tissue	MB	Tissue	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Biopsy Bottle	Micro	Deliver on Wet Ice
33	Autopsy Culture	Blood	MB	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	5.0	Bld Cx (2 Bottles)	Micro	
34	Autopsy Culture	Autopsy	MB	Micro Swab	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Micro Swab	Micro	
35	Bacterial Antigens CSF	CSF	MB CSF	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Plastic Cone T	Micro	
36	Bile Culture	Bile	MB Bile	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)	2.0	Sterile Plastic	Micro	
37	Biopsy Culture	Tissue	MB Tissue	Tissue	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Sterile Screw	Micro	
38	Blood Culture	Blood	MB Blood	Venous Draw	(ALL)	0 Minutes	10 Years	(ALL)	2.0	Bld Cx (Peds)	Micro	
39	Blood Culture	Salvage Blood	MB Blood	Venous Draw	(ALL)	0 Minutes	10 Years	(ALL)	2.0	Bld Cx (Peds)	Micro	
40	Blood Culture	Blood	MB Blood	Venous Draw	(ALL)	10 Years	150 Years	(ALL)	2.0	Bld Cx (2 Bottles)	Micro	
41	Blood Culture	Salvage Blood	MB Blood	Venous Draw	(ALL)	10 Years	150 Years	(ALL)	2.0	Bld Cx (2 Bottles)	Micro	
42	Bone Culture	Bone	MB Tissue	Tissue	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Sterile Plastic	Micro	
43	Bordetella Culture	Nasopharyn Swab	MB Naso	Special Collection	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Micro Swab	Micro	Obtain Collection Kit from
44	Brain Biopsy	Brain Bx	MB Wound	Biopsy	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Sterile Screw	Micro	
45	Bronchial Brush	OTHER	MB BB	Special Collection	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Sterile Screw	Micro	
46	Bronchial Lavage	Bronchoalv lavage	MB BRL	Lavage	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Sterile Plastic	Micro	
47	Bronchial Secretions	Bronchial wash	MB BRW	Wash	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Sterile Screw	Micro	
48	Bronchial Wash	Bronchial wash	MB BRW	Wash	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Sterile Plastic	Micro	
49	C. difficile Toxin A&B	Stool	MB C. DIFF	Stool	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Sterile Plastic	Micro	
50	CMV Antigenemia	Blood EDTA	VC BLOOD EDTA	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	3.0	Lav	Virol	Deliver Immediately
51	CSF Culture	CSF	MB CSF	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Sterile T	Micro	Deliver Immediately
52	Cervical Culture	Cervical Swab	MB Cerv	Micro Swab	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Micro Swab	Micro	
53	Chlamydia Culture	Vaginal Swab	VC	Dacron Swab	(ALL)	0 Minutes	150 Years	(ALL)	3.0	Vir/Chlam	Virol	Deliver Immediately on We
54	Chlamydia Culture	Swab	VC	Dacron Swab	(ALL)	0 Minutes	150 Years	(ALL)	3.0	Vir/Chlam	Virol	Deliver Immediately on We
55	Chlamydia Culture	Eye Swab	VC EYE	Dacron Swab	(ALL)	0 Minutes	150 Years	(ALL)	3.0	Vir/Chlam	Virol	Deliver Immediately on We
56	Chlamydia Culture	Urethral Swab	VC	Dacron Swab	(ALL)	0 Minutes	150 Years	(ALL)	3.0	Vir/Chlam	Virol	Deliver Immediately on We

	Primary Synonym	Specimen Type	Accession Class	Collection Method	Service Resource	Age From	Age To	Coll Priority	Min Vol	Container	Collection Class	Special Handling
57	Chlamydia Culture	Rectal Swab	VC RECTAL	Dacron Swab	(ALL)	0 Minutes	150 Years	(ALL)	3.0	Vir/Chlam	Virol	Deliver Immediately on We
58	Chlamydia Culture	Cervical Swab	VC	Dacron Swab	(ALL)	0 Minutes	150 Years	(ALL)	3.0	Vir/Chlam	Virol	Deliver Immediately on We
59	Chlamydia Culture	NPA	VC	Aspirate	(ALL)	0 Minutes	150 Years	(ALL)	3.0	Vir/Chlam	Virol	Deliver Immediately on We
60	Comprehensive Viral Culture	CSF	VC CSF	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Sterile Screw	Virol	Deliver Immediately on We
61	Comprehensive Viral Culture	Urine	VC URINE	Urine	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Sterile Screw	Virol	Deliver Immediately on We
62	Comprehensive Viral Culture	Body Fluid	VC FLUID	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Sterile T	Virol	Deliver Immediately on We
63	Comprehensive Viral Culture	Nasopharyn Wash	VC FLUID	Wash	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Vir/Chlam	Virol	Deliver Immediately on We
64	Comprehensive Viral Culture	Bronchial wash	VC BW	Wash	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Sterile Screw	Virol	Deliver Immediately on We
65	Comprehensive Viral Culture	Bronchoalv lavage	VC BAL	Lavage	(ALL)	0 Minutes	150 Years	(ALL)	2.0	Sterile Screw	Virol	Deliver Immediately on We
66	Comprehensive Viral Culture	Blood EDTA	VC BLOOD EDTA	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	3.0	Lav	Virol	Deliver Immediately on We
67	Comprehensive Viral Culture	Stomach Bx	VC STOMACH BX	Tissue	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Vir/Chlam	Virol	Deliver Immediately on We
68	Comprehensive Viral Culture	Liver Bx	VC LIVER BX	Tissue	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Vir/Chlam	Virol	Deliver Immediately on We
69	Comprehensive Viral Culture	GI Bx	VC GI BX	Tissue	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Vir/Chlam	Virol	Deliver Immediately on We
70	Comprehensive Viral Culture	Spleen Aut.	VC	Tissue	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Vir/Chlam	Virol	Deliver Immediately on We
71	Comprehensive Viral Culture	Pancr. Aut.	VC	Tissue	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Vir/Chlam	Virol	Deliver Immediately on We
72	Comprehensive Viral Culture	Lip Swab	VC LIP	Dacron Swab	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Vir/Chlam	Virol	Deliver Immediately on We
73	Comprehensive Viral Culture	Corneal Swab	VC	Tissue	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Vir/Chlam	Virol	Deliver Immediately on We
74	Comprehensive Viral Culture	Urethral Swab	VC URETHRAL	Dacron Swab	(ALL)	0 Minutes	150 Years	(ALL)	2.0	Vir/Chlam	Virol	Deliver Immediately on We
75	Comprehensive Viral Culture	Perianal Swab	VC PERIANAL	Dacron Swab	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Vir/Chlam	Virol	Deliver Immediately on We
76	Comprehensive Viral Culture	Bone Marrow Asp	VC BONE MARROW	Aspirate	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Vir/Chlam	Virol	Deliver Immediately on We
77	Comprehensive Viral Culture	PERTNL	VC	Tissue	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Vir/Chlam	Virol	Deliver Immediately on We
78	Comprehensive Viral Culture	NPA	VC NPA	Aspirate	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Vir/Chlam	Virol	Deliver Immediately on We
79	Comprehensive Viral Culture	Lesion	VC	Dacron Swab	(ALL)	0 Minutes	150 Years	(ALL)	2.0	Sterile Screw	Virol	Deliver Immediately on We
80	Comprehensive Viral Culture	Lesion	VC	Dacron Swab	(ALL)	0 Minutes	150 Years	(ALL)	2.0	Sterile Screw	Virol	Deliver Immediately on We
81	Comprehensive Viral Culture	Amniotic Fluid	VC AMNIOTIF FL	Body Fluid	(ALL)	10 Years	150 Years	(ALL)	2.0	Sterile Screw	Virol	Deliver Immediately on We
82	Comprehensive Viral Culture	Vaginal Swab	VC VAGINAL	Dacron Swab	Viral	0 Minutes	150 Years	(ALL)	1.0	Vir/Chlam	Virol	Deliver Immediately on We
83	Comprehensive Viral Culture	Esophageal Bx	VC ESOPH BX	Tissue	Viral	0 Minutes	150 Years	(ALL)	1.0	Vir/Chlam	Virol	Deliver Immediately on We
84	Comprehensive Viral Culture	Duodenal Bx	VC DUODENAL BX	Tissue	Viral	0 Minutes	150 Years	(ALL)	1.0	Vir/Chlam	Virol	Deliver Immediately on We
85	Comprehensive Viral Culture	Colon Bx	VC COLON BX	Tissue	Viral	0 Minutes	150 Years	(ALL)	1.0	Vir/Chlam	Virol	Deliver Immediately on We
86	Comprehensive Viral Culture	Brain Bx	VC BRAIN BX	Tissue	Viral	0 Minutes	150 Years	(ALL)	1.0	Vir/Chlam	Virol	Deliver Immediately on We
87	Comprehensive Viral Culture	Lung Aut.	VC LUNG AUT	Tissue	Viral	0 Minutes	150 Years	(ALL)	1.0	Vir/Chlam	Virol	Deliver Immediately on We
88	Comprehensive Viral Culture	Liver Aut.	VC LIVER AUT	Tissue	Viral	0 Minutes	150 Years	(ALL)	1.0	Vir/Chlam	Virol	Deliver Immediately on We
89	Comprehensive Viral Culture	Brain Aut.	VC BRAIN AUT	Tissue	Viral	0 Minutes	150 Years	(ALL)	1.0	Vir/Chlam	Virol	Deliver Immediately on We
90	Comprehensive Viral Culture	Kidney Aut.	VC KIDNEY AUT	Tissue	Viral	0 Minutes	150 Years	(ALL)	1.0	Vir/Chlam	Virol	Deliver Immediately on We
91	Comprehensive Viral Culture	BW	VC BW	Wash	Viral	0 Minutes	150 Years	(ALL)	1.0	Vir/Chlam	Virol	Deliver Immediately on We
92	Comprehensive Viral Culture	Lip Swab	VC LIP	Dacron Swab	Viral	0 Minutes	150 Years	(ALL)	1.0	Vir/Chlam	Virol	Deliver Immediately on We
93	Comprehensive Viral Culture	Nasopharyn Swab	VC NP SWAB	Dacron Swab	Viral	0 Minutes	150 Years	(ALL)	1.0	Vir/Chlam	Virol	Deliver Immediately on We
94	Comprehensive Viral Culture	Nasopharyn Swab	VC NP SWAB	Dacron Swab	Viral	0 Minutes	150 Years	(ALL)	1.0	Vir/Chlam	Virol	Deliver Immediately on We
95	Comprehensive Viral Culture	Nasal Swab	VC NP SWAB	Dacron Swab	Viral	0 Minutes	150 Years	(ALL)	1.0	Vir/Chlam	Virol	Deliver Immediately on We
96	Comprehensive Viral Culture	Nasal Swab	VC NP SWAB	Dacron Swab	Viral	0 Minutes	150 Years	(ALL)	1.0	Vir/Chlam	Virol	Deliver Immediately on We
97	Comprehensive Viral Culture	Swab	VC	Dacron Swab	Viral	0 Minutes	150 Years	(ALL)	1.0	Vir/Chlam	Virol	Deliver Immediately on We
98	Comprehensive Viral Culture	Swab	VC	Dacron Swab	Viral	0 Minutes	150 Years	(ALL)	1.0	Vir/Chlam	Virol	Deliver Immediately on We
99	Comprehensive Viral Culture	Conj Swab	VC EYE	Dacron Swab	Viral	0 Minutes	150 Years	(ALL)	1.0	Vir/Chlam	Virol	Deliver Immediately on We
100	Comprehensive Viral Culture	Conj Swab	VC EYE	Dacron Swab	Viral	0 Minutes	150 Years	(ALL)	1.0	Vir/Chlam	Virol	Deliver Immediately on We
101	Comprehensive Viral Culture	Eye Swab	VC EYE	Dacron Swab	Viral	0 Minutes	150 Years	(ALL)	1.0	Vir/Chlam	Virol	Deliver Immediately on We
102	Comprehensive Viral Culture	Esophageal Swab	VC ESOPH SWAB	Dacron Swab	Viral	0 Minutes	150 Years	(ALL)	1.0	Vir/Chlam	Virol	Deliver Immediately on We
103	Comprehensive Viral Culture	Ear Swab	VC EAR	Dacron Swab	Viral	0 Minutes	150 Years	(ALL)	1.0	Vir/Chlam	Virol	Deliver Immediately on We
104	Comprehensive Viral Culture	Ear Swab	VC EAR	Dacron Swab	Viral	0 Minutes	150 Years	(ALL)	1.0	Vir/Chlam	Virol	Deliver Immediately on We
105	Comprehensive Viral Culture	Oral Swab	VC ORAL	Dacron Swab	Viral	0 Minutes	150 Years	(ALL)	1.0	Vir/Chlam	Virol	Deliver Immediately on We
106	Comprehensive Viral Culture	Oral Swab	VC ORAL	Dacron Swab	Viral	0 Minutes	150 Years	(ALL)	1.0	Vir/Chlam	Virol	Deliver Immediately on We
107	Comprehensive Viral Culture	Scrotal Swab	VC SCROTAL	Dacron Swab	Viral	0 Minutes	150 Years	(ALL)	1.0	Vir/Chlam	Virol	Deliver Immediately on We
108	Comprehensive Viral Culture	Scrotal Swab	VC SCROTAL	Dacron Swab	Viral	0 Minutes	150 Years	(ALL)	1.0	Vir/Chlam	Virol	Deliver Immediately on We
109	Comprehensive Viral Culture	Stool Swab	VC STOOL	Dacron Swab	Viral	0 Minutes	150 Years	(ALL)	1.0	Vir/Chlam	Virol	Deliver Immediately on We
110	Comprehensive Viral Culture	Sacral Swab	VC SACRAL	Dacron Swab	Viral	0 Minutes	150 Years	(ALL)	1.0	Vir/Chlam	Virol	Deliver Immediately on We
111	Comprehensive Viral Culture	Sacral Swab	VC SACRAL	Dacron Swab	Viral	0 Minutes	150 Years	(ALL)	1.0	Vir/Chlam	Virol	Deliver Immediately on We
112	Comprehensive Viral Culture	Rectal Swab	VC RECTAL	Dacron Swab	Viral	0 Minutes	150 Years	(ALL)	1.0	Vir/Chlam	Virol	Deliver Immediately on We

	Primary Synonym	Specimen Type	Accession Class	Collection Method	Service Resource	Age From	Age To	Coll Priority	Min Vol	Container	Collection Class	Special Handling
113	Comprehensive Viral Culture	Rectal Swab	VC RECTAL	Dacron Swab	Viral	0 Minutes	150 Years	(ALL)	1.0	Vir/Chlam	Virol	Deliver Immediately on We
114	Comprehensive Viral Culture	Perineal Swab	VC PERINEAL	Dacron Swab	Viral	0 Minutes	150 Years	(ALL)	1.0	Vir/Chlam	Virol	Deliver Immediately on We
115	Comprehensive Viral Culture	Labial Swab	VC LABIAL	Dacron Swab	Viral	0 Minutes	150 Years	(ALL)	1.0	Vir/Chlam	Virol	Deliver Immediately on We
116	Comprehensive Viral Culture	Genital Swab	VC GENITAL	Dacron Swab	Viral	0 Minutes	150 Years	(ALL)	1.0	Vir/Chlam	Virol	Deliver Immediately on We
117	Comprehensive Viral Culture	Vulva Swab	VC	Dacron Swab	Viral	0 Minutes	150 Years	(ALL)	1.0	Vir/Chlam	Virol	Deliver Immediately on We
118	Comprehensive Viral Culture	Cervical Swab	VC CERVICAL	Dacron Swab	Viral	0 Minutes	150 Years	(ALL)	1.0	Vir/Chlam	Virol	Deliver Immediately on We
119	Comprehensive Viral Culture	Clitoral Swab	VC CLITORIAL	Dacron Swab	Viral	0 Minutes	150 Years	(ALL)	1.0	Vir/Chlam	Virol	Deliver Immediately on We
120	Comprehensive Viral Culture	Anal Swab	VC ANAL	Dacron Swab	Viral	0 Minutes	150 Years	(ALL)	3.0	Vir/Chlam	Virol	Deliver Immediately on We
121	Comprehensive Viral Culture	Brain Autopsy	VC	Tissue	Viral	0 Minutes	150 Years	(ALL)	1.0	Vir/Chlam	Virol	Deliver Immediately on We
122	Comprehensive Viral Culture	Vesicle Asp.	VC	Aspirate	Viral	0 Minutes	150 Years	(ALL)	1.0	Vir/Chlam	Virol	Deliver Immediately on We
123	Comprehensive Viral Culture	Throat Swab	VC THROAT	Dacron Swab	Viral	0 Minutes	150 Years	(ALL)	1.0	Vir/Chlam	Virol	Deliver Immediately on We
124	Comprehensive Viral Culture	Lung Bx	VC LUNG AUT	Tissue	Viral	0 Minutes	150 Years	(ALL)	1.0	Vir/Chlam	Virol	Deliver Immediately on We
125	Cryptococcal Antigen	CSF	MB CSF	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Sterile Plastic	Micro	
126	Cryptococcal Antigen	Blood	MB Blood	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	1.0	SST	Micro	
127	Cryptosporidium Smear	Stool	MB	Stool	(ALL)	0 Minutes	150 Years	(ALL)	1.0	O&P kit	Micro	
128	Cytomegalovirus Culture	CSF	VC CSF	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)	2.0	Sterile Screw	Virol	Deliver Immediately on We
129	Cytomegalovirus Culture	Tissue	VC EAR	Tissue	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Vir/Chlam	Virol	Deliver Immediately on We
130	Cytomegalovirus Culture	Sputum	VC FLUID	Sputum	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Vir/Chlam	Virol	Deliver Immediately on We
131	Cytomegalovirus Culture	Blood	VC BLOOD EDTA	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	3.0	Lav	Virol	Deliver Immediately
132	Cytomegalovirus Culture	Urine	VC URINE	Urine	(ALL)	0 Minutes	150 Years	(ALL)	2.0	Sterile Screw	Virol	Deliver Immediately on We
133	Cytomegalovirus Culture	Saliva fluid	VC ORAL	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Vir/Chlam	Virol	Deliver Immediately on We
134	Cytomegalovirus Culture	Body Fluid	VC FLUID	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Sterile Screw	Virol	Deliver Immediately on We
135	Cytomegalovirus Culture	Bronchial wash	VC BAL	Lavage	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Sterile Screw	Virol	Deliver Immediately on We
136	Cytomegalovirus Culture	Pericardial Fluid	VC FLUID	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Sterile Screw	Virol	Deliver Immediately on We
137	Cytomegalovirus Culture	Bronchoalv lavage	VC BAL	Lavage	(ALL)	0 Minutes	150 Years	(ALL)	2.0	Sterile Screw	Virol	Deliver Immediately on We
138	Cytomegalovirus Culture	Bone Marrow	VC BONE MARROW	Aspirate	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Vir/Chlam	Virol	Deliver Immediately on We
139	Cytomegalovirus Culture	Blood EDTA	VC BLOOD EDTA	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	3.0	Lav	Virol	Deliver Immediately
140	Cytomegalovirus Culture	Stomach Bx	VC STOMACH BX	Tissue	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Vir/Chlam	Virol	
141	Cytomegalovirus Culture	Liver Bx	VC LIVER BX	Tissue	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Vir/Chlam	Virol	Deliver Immediately on We
142	Cytomegalovirus Culture	GI Bx	VC GI BX	Tissue	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Vir/Chlam	Virol	Deliver Immediately on We
143	Cytomegalovirus Culture	Esophageal Bx	VC ESOPH BX	Tissue	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Vir/Chlam	Virol	Deliver Immediately on We
144	Cytomegalovirus Culture	Duodenal Bx	VC DUODENAL BX	Tissue	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Vir/Chlam	Virol	Deliver Immediately on We
145	Cytomegalovirus Culture	Brain Bx	VC BRAIN BX	Tissue	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Vir/Chlam	Virol	Deliver Immediately on We
146	Cytomegalovirus Culture	Lung Aut.	VC LUNG AUT	Tissue	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Vir/Chlam	Virol	Deliver Immediately on We
147	Cytomegalovirus Culture	Liver Aut.	VC LIVER AUT	Tissue	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Vir/Chlam	Virol	Deliver Immediately on We
148	Cytomegalovirus Culture	Kidney Aut.	VC KIDNEY AUT	Tissue	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Vir/Chlam	Virol	Deliver Immediately on We
149	Cytomegalovirus Culture	Lip Swab	VC LIP	Dacron Swab	(ALL)	0 Minutes	150 Years	(ALL)	2.0	Vir/Chlam	Virol	Deliver Immediately on We
150	Cytomegalovirus Culture	Nasopharyn Swab	VC NP SWAB	Dacron Swab	(ALL)	0 Minutes	150 Years	(ALL)	2.0	Vir/Chlam	Virol	Deliver Immediately on We
151	Cytomegalovirus Culture	Nasal Swab	VC	Dacron Swab	(ALL)	0 Minutes	150 Years	(ALL)	2.0	Vir/Chlam	Virol	Deliver Immediately on We
152	Cytomegalovirus Culture	Eye Swab	VC EYE	Dacron Swab	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Vir/Chlam	Virol	Deliver Immediately on We
153	Cytomegalovirus Culture	Esophageal Swab	VC ESOPH SWAB	Dacron Swab	(ALL)	0 Minutes	150 Years	(ALL)	2.0	Vir/Chlam	Virol	Deliver Immediately on We
154	Cytomegalovirus Culture	Oral Swab	VC ORAL	Dacron Swab	(ALL)	0 Minutes	150 Years	(ALL)	2.0	Vir/Chlam	Virol	Deliver Immediately on We
155	Cytomegalovirus Culture	Scrotal Swab	VC	Dacron Swab	(ALL)	0 Minutes	150 Years	(ALL)	2.0	Vir/Chlam	Virol	Deliver Immediately on We
156	Cytomegalovirus Culture	Sacral Swab	VC	Dacron Swab	(ALL)	0 Minutes	150 Years	(ALL)	2.0	Vir/Chlam	Virol	Deliver Immediately on We
157	Cytomegalovirus Culture	Rectal Swab	VC RECTAL	Dacron Swab	(ALL)	0 Minutes	150 Years	(ALL)	2.0	Vir/Chlam	Virol	Deliver Immediately on We
158	Cytomegalovirus Culture	Perineal Swab	VC	Dacron Swab	(ALL)	0 Minutes	150 Years	(ALL)	2.0	Vir/Chlam	Virol	Deliver Immediately on We
159	Cytomegalovirus Culture	Labial Swab	VC	Dacron Swab	(ALL)	0 Minutes	150 Years	(ALL)	2.0	Vir/Chlam	Virol	Deliver Immediately on We
160	Cytomegalovirus Culture	Genital Swab	VC GENITAL	Dacron Swab	(ALL)	0 Minutes	150 Years	(ALL)	2.0	Vir/Chlam	Virol	Deliver Immediately on We
161	Cytomegalovirus Culture	Vulva Swab	VC GENITAL	Dacron Swab	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Vir/Chlam	Virol	Deliver Immediately on We
162	Cytomegalovirus Culture	Cervical Swab	VC	Dacron Swab	(ALL)	0 Minutes	150 Years	(ALL)	2.0	Vir/Chlam	Virol	Deliver Immediately on We
163	Cytomegalovirus Culture	Anal Swab	VC	Dacron Swab	(ALL)	0 Minutes	150 Years	(ALL)	2.0	Vir/Chlam	Virol	Deliver Immediately on We
164	Cytomegalovirus Culture	Brain Autopsy	VC BRAIN AUT	Tissue	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Vir/Chlam	Virol	Deliver Immediately on We
165	Cytomegalovirus Culture	Bone Marrow Asp	VC BONE MARROW	Aspirate	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Vir/Chlam	Virol	Deliver Immediately
166	Cytomegalovirus Culture	Throat Swab	VC THROAT	Dacron Swab	(ALL)	0 Minutes	150 Years	(ALL)	2.0	Vir/Chlam	Virol	Deliver Immediately on We
167	Cytomegalovirus Culture	Lung Bx	VC LUNG AUT	Tissue	(ALL)	0 Minutes	150 Years	(ALL)	2.0	Vir/Chlam	Virol	Deliver Immediately on We
168	Cytomegalovirus Culture	NPA	VC NPA	Aspirate	(ALL)	0 Minutes	150 Years	(ALL)	3.0	Vir/Chlam	Virol	Deliver Immediately on We

	Primary Synonym	Specimen Type	Accession Class	Collection Method	Service Resource	Age From	Age To	Coll Priority	Min Vol	Container	Collection Class	Special Handling
169	Cytomegalovirus Culture	Lesion	VC LES OTHER	Dacron Swab	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Vir/Chlam	Virol	Deliver Immediately on We
170	Cytomegalovirus Culture	OTHER	VC LES OTHER	Dacron Swab	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Vir/Chlam	Virol	Deliver Immediately on We
171	Cytomegalovirus Culture	Amniotic Fluid	VC AMNIOTIF FL	Body Fluid	(ALL)	10 Years	150 Years	(ALL)	2.0	Sterile Screw	Virol	Deliver Immediately on We
172	Ear Culture	Ear Swab	MB Ear	Micro Swab	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Micro Swab	Micro	
173	Eye Culture	Eye Swab	MB Eye	Micro Swab	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Cx Swab	Micro	
174	Fungal Culture	CSF	MB CSF	Special Collection	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Sterile T	Micro	Deliver Immediately
175	Fungal Culture	Sputum	MB Sputum	Sputum	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Sterile Plastic	Micro	
176	Fungal Culture	Wound	MBFUNGAL	Micro Swab	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Micro Swab	Micro	
177	Fungal Culture	Synovial fluid	MB Synovial	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Sterile T	Micro	
178	Fungal Culture	Blood	MB Blood	Venous Draw	(ALL)	0 Minutes	7 Years	(ALL)	1.0	Bld Cx (Peds)	Micro	
179	Fungal Culture	Urine	MB Urine	Urine	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Midstream	Micro	
180	Fungal Culture	Bronchial wash	MB BRW	Wash	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Sterile Plastic	Micro	
181	Fungal Culture	Pleural Fluid	MB Pleural	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Sterile T	Micro	
182	Fungal Culture	Pericardial Fluid	MB Pericardial	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Sterile T	Micro	
183	Fungal Culture	Bronchoalv lavage	MB BRL	Lavage	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Sterile Plastic	Micro	
184	Fungal Culture	Nasal Swab	MB Nasal	Micro Swab	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Cx Swab	Micro	
185	Fungal Culture	Eye Swab	MB Eye	Micro Swab	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Cx Swab	Micro	
186	Fungal Culture	Ear Swab	MB Ear	Micro Swab	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Cx Swab	Micro	
187	Fungal Culture	Oral Swab	MB Oral	Micro Swab	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Cx Swab	Micro	
188	Fungal Culture	Urethral Swab	MB Urethral	Micro Swab	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Cx Swab	Micro	
189	Fungal Culture	Genital Swab	MB Genital	Micro Swab	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Cx Swab	Micro	
190	Fungal Culture	Bone Marrow Asp	MB BM	Syringe	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Yellow (MAI)	Micro	Mix carefully
191	Fungal Culture	Vesicle Asp.	MB Vesicle	Syringe	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Sterile T	Micro	
192	Fungal Culture	Lung Bx	MB Lung	Tissue	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Sterile Plastic	Micro	
193	Fungal Culture	IV Cath	MB IV Cath	Micro Swab	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Sterile T	Micro	
194	Fungal Culture	Abcess Drainage	MB Abcess	Micro Swab	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Micro Swab	Micro	
195	Fungal Culture	Peritoneal Fluid	MB Peritoneal	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Sterile T	Micro	
196	Fungal Culture	OTHER	MB Other	Micro Swab	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Sterile Plastic	Micro	
197	Fungal Culture	Blood	MB Blood	Venous Draw	(ALL)	7 Years	150 Years	(ALL)	2.0	Bld Cx (2 Bottles)	Micro	
198	Fungal Smear	CSF	MB CSF	Special Collection	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Sterile T	Micro	Deliver Immediately
199	Fungal Smear	Sputum	MB Stool	Sputum	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Sterile Plastic	Micro	
200	Fungal Smear	Wound	MBFUNGAL	Micro Swab	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Micro Swab	Micro	
201	Fungal Smear	Synovial fluid	MB Synovial	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Sterile T	Micro	
202	Fungal Smear	Blood	MB Blood	Venous Draw	(ALL)	0 Minutes	6 Years	(ALL)	1.0	Bld Cx (Peds)	Micro	
203	Fungal Smear	Urine	MB Urine	Urine	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Midstream	Micro	
204	Fungal Smear	Bronchial wash	MB BRW	Wash	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Sterile Plastic	Micro	
205	Fungal Smear	Pleural Fluid	MB Pleural	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Sterile T	Micro	
206	Fungal Smear	Pericardial Fluid	MB Pericardial	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Sterile T	Micro	
207	Fungal Smear	Bronchoalv lavage	MB BRL	Lavage	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Sterile Plastic	Micro	
208	Fungal Smear	Vesicle Asp	MB Vesicle	Syringe	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Sterile T	Micro	
209	Fungal Smear	Nasal Swab	MB Nasal	Micro Swab	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Cx Swab	Micro	
210	Fungal Smear	Eye Swab	MB Eye	Micro Swab	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Cx Swab	Micro	
211	Fungal Smear	Ear Swab	MB Ear	Micro Swab	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Cx Swab	Micro	
212	Fungal Smear	Oral Swab	MB Oral	Micro Swab	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Cx Swab	Micro	
213	Fungal Smear	Urethral Swab	MB Urethral	Micro Swab	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Cx Swab	Micro	
214	Fungal Smear	Genital Swab	MB Genital	Micro Swab	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Cx Swab	Micro	
215	Fungal Smear	Bone Marrow Asp	MB BM	Syringe	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Yellow (MAI)	Micro	
216	Fungal Smear	Lung Bx	MB Lung	Tissue	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Sterile Plastic	Micro	
217	Fungal Smear	IV Cath	MB IV Cath	Micro Swab	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Sterile T	Micro	
218	Fungal Smear	Abcess Drainage	MB Abcess	Micro Swab	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Micro Swab	Micro	
219	Fungal Smear	Peritoneal Fluid	MB Peritoneal	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Sterile T	Micro	
220	Fungal Smear	OTHER	MB Other	Micro Swab	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Sterile Plastic	Micro	
221	Fungal Smear	Blood	MB Blood	Venous Draw	(ALL)	7 Years	150 Years	(ALL)	2.0	Bld Cx (2 Bottles)	Micro	
222	GC Culture	Vaginal Swab	MB Vaginal	Micro Swab	(ALL)	0 Minutes	150 Years	(ALL)	1.0	JEMBEC	Micro	
223	GC Culture	Urethral Swab	MB Urethral	Micro Swab	(ALL)	0 Minutes	150 Years	(ALL)	1.0	JEMBEC	Micro	
224	GC Culture	Cervical Swab	MB Cerv	Micro Swab	(ALL)	0 Minutes	150 Years	(ALL)	1.0	JEMBEC	Micro	

	Primary Synonym	Specimen Type	Accession Class	Collection Method	Service Resource	Age From	Age To	Coll Priority	Min Vol	Container	Collection Class	Special Handling
225	GC Culture	Anal Swab	MB Anal	Micro Swab	(ALL)	0 Minutes	150 Years	(ALL)	1.0	JEMBEC	Micro	
226	GC Culture	Throat Swab	MB Throat	Micro Swab	(ALL)	0 Minutes	150 Years	(ALL)	1.0	JEMBEC	Micro	
227	Gram Stain	Gram Stain	MB	Slide	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Slides	Micro	
228	Groin Culture	Groin	MB AFB Wound	Swab	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Micro Swab	Micro	
229	HSV Cx Neonate	CSF	VC CSF	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Sterile Screw	Virol	Deliver Immediately on We
230	HSV Cx Neonate	Penile swab	VC LES PENILE	Dacron Swab	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Vir/Chlam	Virol	Deliver Immediately on We
231	HSV Cx Neonate	Vaginal Swab	VC LES GENITAL	Dacron Swab	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Vir/Chlam	Virol	Deliver Immediately
232	HSV Cx Neonate	Lip Swab	VC LES ORAL	Dacron Swab	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Vir/Chlam	Virol	Deliver Immediately on We
233	HSV Cx Neonate	Eye Swab	VC EYE	Dacron Swab	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Vir/Chlam	Virol	Deliver Immediately on We
234	HSV Cx Neonate	Oral Swab	VC LES ORAL	Dacron Swab	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Vir/Chlam	Virol	Deliver Immediately on We
235	HSV Cx Neonate	Perianal Swab	VC PERIANAL	Dacron Swab	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Vir/Chlam	Virol	Deliver Immediately on We
236	HSV Cx Neonate	Labial Swab	VC LABIAL	Dacron Swab	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Vir/Chlam	Virol	Deliver Immediately on We
237	HSV Cx Neonate	Genital Swab	VC LES GENITAL	Dacron Swab	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Vir/Chlam	Virol	Deliver Immediately on We
238	HSV Cx Neonate	Vulva Swab	VC LES GENITAL	Dacron Swab	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Vir/Chlam	Virol	Deliver Immediately on We
239	HSV Cx Neonate	Cervical Swab	VC CERVICAL	Dacron Swab	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Vir/Chlam	Virol	Deliver Immediately on We
240	HSV Cx Neonate	Throat Swab	VC ORAL	Dacron Swab	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Vir/Chlam	Virol	Deliver Immediately on We
241	HSV Cx Neonate	Lesion	VC LES OTHER	Dacron Swab	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Vir/Chlam	Virol	Deliver Immediately on We
242	HSV VZ Neonate	Tissue	VC	Tissue	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Vir/Chlam	Virol	Deliver Immediately on We
243	HSV VZ Neonate	Vesicle Asp	VC FLUID	Aspirate	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Vir/Chlam	Virol	Deliver Immediately on We
244	HSV VZ Neonate	Lesion	VC LES OTHER	Dacron Swab	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Vir/Chlam	Virol	Deliver Immediately on We
245	Herpes Simplex Culture	CSF	VC CSF	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Sterile Screw	Virol	Deliver Immediately on We
246	Herpes Simplex Culture	Penile swab	VC PENILE	Dacron Swab	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Vir/Chlam	Virol	Deliver Immediately on We
247	Herpes Simplex Culture	Vaginal Swab	VC LES GENITAL	Dacron Swab	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Vir/Chlam	Virol	Deliver Immediately on We
248	Herpes Simplex Culture	Lip Swab	VC LES ORAL	Dacron Swab	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Vir/Chlam	Virol	Deliver Immediately on We
249	Herpes Simplex Culture	Eye Swab	VC EYE	Dacron Swab	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Vir/Chlam	Virol	Deliver Immediately on We
250	Herpes Simplex Culture	Oral Swab	VC LES ORAL	Dacron Swab	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Vir/Chlam	Virol	Deliver Immediately on We
251	Herpes Simplex Culture	Perineal Swab	VC PERINEAL	Dacron Swab	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Vir/Chlam	Virol	Deliver Immediately
252	Herpes Simplex Culture	Labial Swab	VC LABIAL	Dacron Swab	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Vir/Chlam	Virol	Deliver Immediately on We
253	Herpes Simplex Culture	Genital Swab	VC LES GENITAL	Dacron Swab	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Vir/Chlam	Virol	Deliver Immediately on We
254	Herpes Simplex Culture	Vulva Swab	VC VULVA	Dacron Swab	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Vir/Chlam	Virol	Deliver Immediately on We
255	Herpes Simplex Culture	Cervical Swab	VC CERVICAL	Dacron Swab	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Vir/Chlam	Virol	Deliver Immediately
256	Herpes Simplex Culture	Throat Swab	VC ORAL	Dacron Swab	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Vir/Chlam	Virol	Deliver Immediately on We
257	Herpes Simplex Culture	Lesion	VC LES OTHER	Dacron Swab	(ALL)	0 Minutes	150 Years	(ALL)	2.0	Vir/Chlam	Virol	Deliver Immediately on We
258	Herpes Simplex and Varicella Culture	Tissue	VC	Tissue	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Vir/Chlam	Virol	Deliver Immediately on We
259	Herpes Simplex and Varicella Culture	Vesicle Asp	VC FLUID	Aspirate	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Vir/Chlam	Virol	Deliver Immediately on We
260	Herpes Simplex and Varicella Culture	Lesion	VC LES OTHER	Dacron Swab	(ALL)	0 Minutes	150 Years	(ALL)	2.0	Vir/Chlam	Virol	Deliver Immediately on We
261	IV Catheter Tip Culture	IV Cath	MB	Catheter	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Sterile Plastic	Micro	
262	IV Site Culture	Swab	MB	Micro Swab	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Sterile Screw	Micro	
263	Influenza Antigens	Nasopharyn Wash	VC	Wash	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Vir/Chlam	Virol	Deliver Immediately on We
264	Influenza Antigens	Bronchial wash	VC BW	Wash	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Sterile Screw	Virol	Deliver Immediately on We
265	Influenza Antigens	Bronchoalv lavage	VC BAL	Lavage	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Vir/Chlam	Virol	
266	Influenza Antigens	Tracheal aspirate	VC TRACH ASP	Aspirate	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Vir/Chlam	Virol	
267	Influenza Antigens	Nasopharyn Swab	VC NP SWAB	Dacron Swab	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Vir/Chlam	Virol	Deliver Immediately on We
268	Influenza Antigens	Throat Swab	VC THROAT	Dacron Swab	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Vir/Chlam	Virol	Deliver Immediately on We
269	Influenza Antigens	NPA	VC NPA	Aspirate	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Vir/Chlam	Virol	Deliver Immediately on We
270	Influenza Antigens	Tracheal Wash	VC TRACH ASP	Wash	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Vir/Chlam	Virol	
271	Legionella Culture	Sputum	MB	Sputum	(ALL)	0 Minutes	150 Years	(ALL)	2.0	Sterile Screw	Micro	History Form Required-Cal
272	Legionella Culture	Bronchial wash	MB	Wash	(ALL)	0 Minutes	150 Years	(ALL)	2.0	Sterile Screw	Micro	History Form Required-Cal
273	Legionella Culture	Bronchoalv lavage	MB	Lavage	(ALL)	0 Minutes	150 Years	(ALL)	2.0	Sterile Screw	Micro	History Form Required-Cal
274	Legionella DFA	Sputum	MB Sputum	Sputum	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Sterile Plastic	Micro	History Form Required-Cal
275	Legionella Urinary Antigen	Urine	MB Urine	Urine	(ALL)	0 Minutes	150 Years	(ALL)	5.0	Sterile Plastic	Micro	History Form Required-Cal
276	Lung Biopsy	Lung Bx	MB Lung	Tissue	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Sterile Screw	Micro	
277	Lung Aspirate	Lung asp	MB Lung	Aspirate	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Sterile Screw	Micro	
278	Miscellaneous Culture	OTHER	MB Other	Swab	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Micro Swab	Micro	
279	Mycoplasma pneumoniae Culture	Sputum	MB Sputum	Sputum	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Sterile Screw	Micro	
280	Nose Culture	Nasal Swab	MB Nasal	Micro Swab	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Micro Swab	Micro	

	Primary Synonym	Specimen Type	Accession Class	Collection Method	Service Resource	Age From	Age To	Coll Priority	Min Vol	Container	Collection Class	Special Handling
281	Organ Transplant Culture	Tissue	MB Other	Syringe	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Sterile Screw	Micro	
282	Organ Transplant Culture	Perfusate	MB Other	Syringe	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Sterile Screw	Micro	
283	Organ Transplant Culture	Pre Flush Perfusate	MB Other	Syringe	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Sterile Screw	Micro	
284	Organ Transplant Culture	Post Flush Perfusate	MB Other	Syringe	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Sterile Screw	Micro	
285	Organ Transplant Culture	Peritoneal	MB AFB Other	Syringe	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Sterile Screw	Micro	
286	Organ Transplant Culture	Pre Pump Perfusate	MB AFB Other	Syringe	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Sterile Screw	Micro	
287	Organ Transplant Culture	1Hr Pump Perfusate	MB Other	Syringe	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Sterile Screw	Micro	
288	Organ Transplant Culture	Post Left Perfusate	MB Other	Syringe	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Sterile Screw	Micro	
289	Organ Transplant Culture	Post Right Perfusate	MB Other	Syringe	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Sterile Screw	Micro	
290	Organ Transplant Culture	Transport Solution	MB Other	Syringe	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Sterile Screw	Micro	
291	Ova & Parasites	Urine	MB Urine	Urine	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Sterile Screw	Micro	
292	Ova & Parasites	Stool	MB Stool	Stool	(ALL)	0 Minutes	150 Years	(ALL)	1.0	O&P kit	Micro	
293	Pericardial Fluid Culture	Pericardial Fluid	MB Pericardial	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Sterile Plastic	Micro	
294	Peritoneal Fluid Culture	Peritoneal Fluid	MB Peritoneal	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Bld Cx (2 Bottles)	Micro	
295	Pertussis DFA	Nasopharyn Swab	MB Naso	Nasopharyngeal	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Contact Laboratory	Micro	Obtain Collection Kit from I
296	Pleural Fluid Culture	Pleural Fluid	MB Pleural	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Sterile T	Micro	
297	ROTA VIRUS	Stool Swab	VC STOOL	Dacron Swab	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Vir/Chlam	Virol	Deliver Immediately on We
298	RSV Antigen	Nasopharyn Wash	VC	Wash	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Vir/Chlam	Virol	Deliver Immediately on We
299	RSV Antigen	Bronchial wash	VC BW	Wash	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Vir/Chlam	Virol	Deliver Immediately on We
300	RSV Antigen	Nasopharyn Swab	VC NP SWAB	Dacron Swab	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Vir/Chlam	Virol	Deliver Immediately on We
301	RSV Antigen	Throat Swab	VC THROAT	Dacron Swab	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Vir/Chlam	Virol	Deliver Immediately on We
302	RSV Antigen	NPA	VC NPA	Aspirate	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Vir/Chlam	Virol	Deliver Immediately on We
303	Rectal Swab	Rectal Swab	MB Rectal Swab	Micro Swab	(ALL)	0 Minutes	150 Years	(ALL)	0.1	Micro Swab	Micro	
304	Respiratory Screen Viral Culture	Nasopharyn Wash	VC FLUID	Vial	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Sterile Screw	Virol	Deliver Immediately on We
305	Respiratory Screen Viral Culture	Bronchial wash	VC BW	Wash	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Sterile Screw	Virol	Deliver Immediately on We
306	Respiratory Screen Viral Culture	Bronchoalv lavage	VC BAL	Lavage	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Sterile Screw	Virol	Deliver Immediately on We
307	Respiratory Screen Viral Culture	Tracheal aspirate	VC TRACH ASP	Aspirate	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Vir/Chlam	Virol	Deliver Immediately on We
308	Respiratory Screen Viral Culture	Nasopharyn Swab	VC NP SWAB	Dacron Swab	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Vir/Chlam	Virol	Deliver Immediately on We
309	Respiratory Screen Viral Culture	Nasal Swab	VC NP SWAB	Dacron Swab	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Vir/Chlam	Virol	Deliver Immediately on We
310	Respiratory Screen Viral Culture	Throat Swab	VC THROAT	Dacron Swab	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Vir/Chlam	Virol	Deliver Immediately on We
311	Respiratory Screen Viral Culture	NPA	VC NPA	Aspirate	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Vir/Chlam	Virol	Deliver Immediately on We
312	Respiratory Screen Viral Culture	Tracheal Wash	VC	Wash	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Vir/Chlam	Virol	Deliver Immediately on We
313	Sinus Aspirate	Asp.	MB Nasal	Aspirate	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Sterile Screw	Micro	
314	Sputum Culture	Sputum	MB Sputum	Sputum	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Sterile Plastic	Micro	
315	Sterility Culture	Dialysate	MB Other	Special Collection	(ALL)	0 Minutes	150 Years	(ALL)	15.0	Contact Laboratory	Micro	
316	Sterility Culture	TPN	MB TPN	Pharmacy	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Sterile Plastic	Micro	
317	Sterility Culture	ENVIRONMENT	MB Other	Special Collection	(ALL)	0 Minutes	150 Years	(ALL)	15.0	Sterile Plastic	Micro	
318	Sterility Culture	OTHER	MB Other	Micro Swab	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Cx Swab	Micro	
319	Sterility Culture	Dialysis Water	MB Other	Special Collection	(ALL)	0 Minutes	150 Years	(ALL)	15.0	Contact Laboratory	Micro	
320	Sterility Culture	Water	MB Other	Special Collection	(ALL)	0 Minutes	150 Years	(ALL)	15.0	Contact Laboratory	Micro	
321	Sterility Culture	BB Product	MB Other	Syringe	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Sterile Plastic	Micro	
322	Stool Culture	Stool	MB Stool	Stool	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Stool Cx	Micro	
323	Surgical Drain Culture	Abccess Drainage	MB Abccess	Micro Swab	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Micro Swab	Micro	
324	Synovial Fluid Culture	Synovial fluid	MB Synovial	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Sterile T	Micro	
325	Throat Culture	Throat Swab	MB Throat	Micro Swab	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Micro Swab	Micro	
326	Tissue Culture	Tissue	MB Tissue	Tissue	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Sterile Plastic	Micro	
327	Tracheal Aspirate Culture	Tracheal aspirate	MB Trach	Aspirate	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Sterile Plastic	Micro	
328	Transtracheal Aspirate Culture	Tracheal aspirate	MB	Aspirate	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Sterile Plastic	Micro	
329	Ureaplasma urealyticum Culture	Urine	MB Urine	Urine	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Sterile Screw	Micro	
330	Urethral Culture	Urethral Swab	MB Urethral	Micro Swab	(ALL)	0 Minutes	150 Years	(ALL)	1.0	JEMBEC	Micro	Incubate immediately
331	Urine Culture	U CC	MB Urine	Urine	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Midstream	Micro	
332	Urine Culture	U Cath	MB Urine	Catheter	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Sterile Screw	Micro	
333	Urine Culture	U Supra	MB Urine	Urine	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Sterile Plastic	Micro	
334	Vaginal Beta Strep Screen	Vaginal Swab	MB Wound	Swab	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Micro Swab	Micro	
335	Vaginal Culture	Vaginal Swab	MB Vaginal	Micro Swab	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Micro Swab	Micro	Deliver Immediately
336	Varicella-zoster Culture	Vesicle Asp	VC FLUID	Aspirate	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Vir/Chlam	Virol	Deliver Immediately on We

	Primary Synonym	Specimen Type	Accession Class	Collection Method	Service Resource	Age From	Age To	Coll Priority	Min Vol	Container	Collection Class	Special Handling
337	Varicella-zoster Culture	Swab	VC LES SKIN	Dacron Swab	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Vir/Chlam	Virol	Deliver Immediately on We
338	Varicella-zoster Culture	Lesion	VC LES OTHER	Dacron Swab	(ALL)	0 Minutes	150 Years	(ALL)	2.0	Vir/Chlam	Virol	Deliver Immediately on We
339	Varicella-zoster Culture	OTHER	VC	Dacron Swab	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Vir/Chlam	Virol	Deliver Immediately on We
340	Viral Cx Neonate	CSF	VC CSF	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Sterile Screw	Virol	Deliver Immediately on We
341	Viral Cx Neonate	Urine	VC URINE	Urine	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Sterile Screw	Virol	Deliver Immediately on We
342	Viral Cx Neonate	Body Fluid	VC FLUID	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Sterile T	Virol	Deliver Immediately on We
343	Viral Cx Neonate	Nasopharyn Wash	VC FLUID	Wash	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Vir/Chlam	Virol	Deliver Immediately on We
344	Viral Cx Neonate	Bronchial wash	VC BW	Wash	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Sterile Screw	Virol	Deliver Immediately on We
345	Viral Cx Neonate	Bronchoalv lavage	VC AMNIOTIF FL	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)	2.0	Sterile Screw	Virol	Deliver on Wet Ice
346	Viral Cx Neonate	Amniotic Fluid	VC AMNIOTIF FL	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)	2.0	Sterile Screw	Virol	Deliver Immediately on We
347	Viral Cx Neonate	Vaginal Swab	VC VAGINAL	Dacron Swab	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Vir/Chlam	Virol	Deliver Immediately on We
348	Viral Cx Neonate	Blood EDTA	VC BLOOD EDTA	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)	3.0	LCX	Virol	Deliver Immediately on We
349	Viral Cx Neonate	Stomach Bx	VC STOMACH BX	Tissue	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Vir/Chlam	Virol	Deliver Immediately on We
350	Viral Cx Neonate	Liver Bx	VC LIVER BX	Tissue	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Vir/Chlam	Virol	Deliver Immediately on We
351	Viral Cx Neonate	GI Bx	VC GI BX	Tissue	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Vir/Chlam	Virol	Deliver Immediately on We
352	Viral Cx Neonate	Esophageal Bx	VC ESOPH BX	Tissue	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Vir/Chlam	Virol	Deliver Immediately on We
353	Viral Cx Neonate	Duodenal Bx	VC DUODENAL BX	Tissue	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Vir/Chlam	Virol	Deliver Immediately on We
354	Viral Cx Neonate	Colon Bx	VC COLON BX	Tissue	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Vir/Chlam	Virol	Deliver Immediately on We
355	Viral Cx Neonate	Brain Bx	VC BRAIN BX	Tissue	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Vir/Chlam	Virol	Deliver Immediately on We
356	Viral Cx Neonate	Spleen Aut.	VC	Tissue	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Vir/Chlam	Virol	Deliver Immediately on We
357	Viral Cx Neonate	Pancr. Aut.	VC	Tissue	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Vir/Chlam	Virol	Deliver Immediately on We
358	Viral Cx Neonate	Lung Aut.	VC LUNG AUT	Tissue	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Vir/Chlam	Virol	Deliver Immediately on We
359	Viral Cx Neonate	Liver Aut.	VC LIVER AUT	Tissue	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Vir/Chlam	Virol	Deliver Immediately on We
360	Viral Cx Neonate	Kidney Aut.	VC KIDNEY AUT	Tissue	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Vir/Chlam	Virol	Deliver Immediately on We
361	Viral Cx Neonate	Lip Swab	VC LIP	Dacron Swab	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Vir/Chlam	Virol	Deliver Immediately on We
362	Viral Cx Neonate	Nasopharyn Swab	VC NP SWAB	Dacron Swab	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Vir/Chlam	Virol	Deliver Immediately on We
363	Viral Cx Neonate	Nasal Swab	VC NP SWAB	Dacron Swab	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Vir/Chlam	Virol	Deliver Immediately on We
364	Viral Cx Neonate	Esophageal Swab	VC ESOPH BX	Tissue	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Vir/Chlam	Virol	Deliver Immediately on We
365	Viral Cx Neonate	Brain Autopsy	VC BRAIN AUT	Tissue	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Vir/Chlam	Virol	Deliver Immediately on We
366	Wound Culture	Wound	MB Wound	Micro Swab	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Micro Swab	Micro	
367	Wound Culture	Bone Marrow Asp	MB BM	Micro Swab	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Micro Swab	Micro	
368	Wound Culture	Abcess Drainage	MB	Micro Swab	(ALL)	0 Minutes	150 Years	(ALL)	1.0	Micro Swab	Micro	