

Just PINCH-PRESS-PREP

Frepp® 1.5 mL Applicator

Approximate coverage area: 2.5" x 2.5"
 Catalog number 260299 – 20 per carton, 25 cartons per case

Instructions for use:

PINCH



- ▶ Hold the applicator as shown; do not touch sponge
- ▶ Pinch wings **once** to activate applicator and release antiseptic

PRESS



- ▶ Holding the applicator horizontally, allow the solution to partially load in sponge
- ▶ Gently press applicator against treatment area to evenly distribute solution throughout the sponge

PREP



- ▶ Once solution is visible on the skin, use gentle back-and-forth strokes to prep the insertion/incision site for at least 30 seconds, then continue back-and-forth strokes, progressing from the insertion/incision site to the periphery:
 - ▶ For dry sites (e.g., abdomen or arm): Use repeated back-and-forth strokes for 30 seconds. Allow area to air dry for approximately 30 seconds.
 - ▶ For moist sites (e.g., inguinal fold or axilla): Use repeated back-and-forth strokes for 2 minutes. Allow area to air dry for approximately 1 minute.
- ▶ Do not blot or wipe away
- ▶ Solution must be dry for optimal drape/dressing adhesion

Safety points for products containing alcohol:

- ▶ Do not drape or use ignition source until solution is completely dry
- ▶ Allow more time for solution to dry if it gets into hair
- ▶ Do not allow solution to pool
- ▶ Remove wet materials from prep area

Additional information:

- ▶ Discard the applicator after a single use
- ▶ Latex free
- ▶ For external use only
- ▶ Do not use on children less than 2 months of age
- ▶ Keep solution out of eyes, ears and mouth
- ▶ Store between 15-30°C (59-86°F)
- ▶ Avoid freezing and excessive heat above 40°C (104°F)

ChloroPrep representative: _____

In-service dates: _____

Time: _____

REORDER #: _____

Location of ChloroPrep applicators at this facility: _____

Before using the ChloroPrep® Frepp® applicator, please read the instructions on the package. Use in accordance with the policies and procedures of your hospital.

For more information, visit chloroprep.com or call 800-523-0502.