

	Primary Synonym	Specimen Type	Accession Class	Collection Method	Service Resource	Age From	Age To	Coll Priority	Aliq	Min Vol	Container	Collection Class	Special Handling	Add Lbl	Opt
1	11 Desoxycortisol	Blood	Chem Sendout Frozen	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		4.0	Red	Ref Frozen	Freeze	0	
2	17 Hydroxyprogesterone	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		0.5	SST	Ref Refrig	Refrigerate	0	
3	17 OHPregnenolone	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Red	Ref Refrig	Refrigerate	0	
4	17 OHPregnenolone	Blood	Chem Sendout	Venous Draw	LabCorp	0 Minutes	150 Years	(ALL)		1.0	Red	Ref Frozen	Freeze	0	
5	1ST Trimester SCR,H-HCG	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		4.0	SST	Ref Refrig		0	
6	21-Hydroxylase Ab	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Red	Ref Refrig	Refrigerate	0	
7	21-Hydroxylase Ab	Blood	Chem Sendout	Venous Draw	LabCorp	0 Minutes	150 Years	(ALL)		3.0	Red	Ref Frozen		0	
8	5 HIAA	Urine	Urine Boric	Urine	(ALL)	0 Minutes	150 Years	(ALL)		100.0	U24H Boric	Ref Refrig	Refrigerate	0	
9	5 HIAA	Urine	Urine Boric	Urine	LabCorp	0 Minutes	150 Years	(ALL)		30.0	U24H	Ref Refrig		0	
10	5 Nucleotidase	Blood	Chem Sendout Frozen	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		5.0	SST	Ref Frozen	Freeze	0	
11	5-HIAA,Random Urine	Urine	Chem Sendout	Urine	(ALL)	0 Minutes	150 Years	(ALL)		100.0	UA	Ref Refrig	Refrigerate	0	
12	A1 Antitrypsin, 24Hr Feces	Stool	Chem Sendout Frozen	Stool	(ALL)	0 Minutes	150 Years	(ALL)		2.0	Stool Container	Ref Frozen	Freeze	0	
13	ABLHB	Blood	Blood Gases Arterial	Syringe	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Hep Syringe	Chem	Deliver Immediately	0	
14	AFP Amniotic Fluid	Amniotic Fluid	Body Fluid	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)		3.0	Sterile T	Chem		0	
15	ALT	Blood	Blood	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)		0.3	SST (micro)	Chem		0	
16	ALT	Blood	Blood	Venous Draw	(ALL)	1 Months	150 Years	(ALL)		0.3	SST	Chem		0	
17	APC Resistance	Blood	Hem Blue Sendouts	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Blue	Ref Frozen	Freeze	0	
18	APTT	Blood	Blood Coag	Venous Draw	(ALL)	0 Minutes	12 Years	(ALL)		0.5	Blue (Peds)	Coag	Mix Immediately	0	
19	APTT	Blood	Blood Coag	Venous Draw	(ALL)	12 Years	150 Years	(ALL)		0.5	Blue	Coag	Mix Immediately	0	
20	AST	Blood	Blood	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)		0.3	SST (micro)	Chem		0	
21	AST	Blood	Blood	Venous Draw	(ALL)	1 Months	150 Years	(ALL)		0.3	SST	Chem		0	
22	Acetaminophen	Blood	Blood	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)		0.3	SST (micro)	Chem		0	
23	Acetaminophen	Blood	Blood	Venous Draw	(ALL)	1 Months	150 Years	(ALL)		0.5	SST	Chem	Draw prior to next dose	0	
24	Acetone Serum	Blood	Blood	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		0.3	SST	Chem		0	
25	Acetylcholine Receptor Antibodies	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		2.0	SST	Ref Refrig	Refrigerate	0	
26	Acute Hepatitis Panel	Blood	Blood	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	SST	Chem		0	
27	Acylcarnitine, Plasma (NY)	Blood	Chem Sendout Frozen	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		10.0	Green	Ref Frozen	Freeze	0	
28	Adenosine Deamin,Pleural	Pleural Fluid	Chem Sendout Frozen	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Sterile T	Ref Frozen	Freeze	0	
29	Adenosine Deaminase, Fluid	CSF	CSF	Syringe	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Sterile Screw	Ref Refrig	Refrigerate	0	
30	Adenovirus Group Antibodies	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	6 Months	(ALL)		1.0	Red (micro)	Ref Refrig	Refrigerate	0	
31	Adenovirus Group Antibodies	Blood	Chem Sendout	Venous Draw	(ALL)	6 Months	150 Years	(ALL)		2.0	Red	Ref Refrig	Refrigerate	0	
32	Adenovirus Group Antibodies	Blood	Chem Sendout	Venous Draw	LabCorp	0 Minutes	150 Years	(ALL)		3.0	SST	Ref Room		0	
33	Adrenal Ab	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		2.0	Red	Ref Refrig	Refrigerate	0	
34	Adrenal Ab	Blood	Chem Sendout	Venous Draw	LabCorp	0 Minutes	150 Years	(ALL)		3.0	SST	Ref Room		0	
35	Adrenocorticotrophic Hormone	Blood	Chem lav sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		2.0	Lav	Ref Frozen	Freeze	0	
36	Albumin	Blood	Blood	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)		0.2	SST (micro)	Chem		0	
37	Albumin	Blood	Blood	Venous Draw	(ALL)	1 Months	150 Years	(ALL)		0.3	SST	Chem		0	
38	Albumin Pericardial Fluid	Pericardial Fluid	Body Fluid	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)		0.3	Red	Chem		0	
39	Albumin Peritoneal Fluid	Peritoneal Fluid	Body Fluid	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)		0.3	Red	Chem		0	
40	Albumin Pleural Fluid	Pleural Fluid	Body Fluid	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)		0.3	Red	Chem		0	
41	Aldolase	Blood	Chem Sendout	Venous Draw	LabCorp	0 Minutes	150 Years	(ALL)		3.0	SST	Ref Refrig		0	
42	Aldosterone	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	6 Months	(ALL)		1.5	Red (micro)	Ref Refrig	Refrigerate	0	
43	Aldosterone	Blood	Chem Sendout	Venous Draw	(ALL)	6 Months	150 Years	(ALL)		5.0	Red	Ref Refrig	Refrigerate	0	
44	Aldosterone	Blood	Chem Sendout	Venous Draw	LabCorp	0 Minutes	150 Years	(ALL)		5.0	SST	Ref Refrig	Refrigerate	0	
45	Aldosterone 24hr Urine	Urine	Urine 24hr	Urine	(ALL)	0 Minutes	150 Years	(ALL)		100.0	U24H	Ref Refrig	Refrigerate	0	
46	Alk Phos Isoenzymes	Blood	Chem Red Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		4.0	Red	Ref Refrig	Refrigerate	0	
47	Alkaline Phosphatase	Blood	Blood	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)		0.2	SST (micro)	Chem		0	
48	Alkaline Phosphatase	Blood	Blood	Venous Draw	(ALL)	1 Months	150 Years	(ALL)		0.3	SST	Chem		0	
49	Allergen Almond	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		0.3	SST	Immune		0	
50	Allergen Alternaria	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		0.2	SST	Immune		0	
51	Allergen Apple	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		0.3	SST	Immune		0	
52	Allergen Barley	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		0.3	SST	Immune		0	
53	Allergen Beech Tree	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		0.3	SST	Immune		0	
54	Allergen Beef	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		0.3	SST	Immune		0	
55	Allergen Bermuda Grass	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		0.3	SST	Immune		0	
56	Allergen Birch Tree	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		0.3	SST	Immune		0	
57	Allergen Box Elder	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		0.3	SST	Immune		0	
58	Allergen Cacao	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		0.3	SST	Immune		0	
59	Allergen Candida Albicans	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		0.3	SST	Immune		0	
60	Allergen Casein	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		0.3	SST	Immune		0	
61	Allergen Casein Peds	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	12 Years	(ALL)		2.0	SST	Immune		0	
62	Allergen Cashew Nut	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		0.3	SST	Immune		0	
63	Allergen Cat Epithelium	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		0.3	SST	Immune		0	
64	Allergen Cheddar Cheese	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		0.3	SST	Immune		0	
65	Allergen Chicken	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		0.2	SST	Immune		0	
66	Allergen Clado Herb	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		0.3	SST	Immune		0	
67	Allergen Cockroach	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		0.3	SST	Immune		0	
68	Allergen Coconut	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		0.3	SST	Immune		0	
69	Allergen Cod Fish	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		0.3	SST	Immune		0	
70	Allergen Common Ragweed	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		0.3	SST	Immune		0	
71	Allergen Corn	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		0.3	SST	Immune		0	

	Primary Synonym	Specimen Type	Accession Class	Collection Method	Service Resource	Age From	Age To	Coll Priority	Aliq	Min Vol	Container	Collection Class	Special Handling	Add Lbl	Opt
72	Allergen Crab	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		0.3	SST	Immune		0	
73	Allergen Dermato Far	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		0.3	SST	Immune		0	
74	Allergen Dermato Pter	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		0.3	SST	Immune		0	
75	Allergen Dog Dander	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		0.3	SST	Immune		0	
76	Allergen Egg White	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		0.3	SST	Immune		0	
77	Allergen Egg White Peds	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	12 Years	(ALL)		2.0	SST	Immune		0	
78	Allergen Egg Yolk	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		0.3	SST	Immune		0	
79	Allergen Egg Yolk Peds	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	12 Years	(ALL)		2.0	SST	Immune		0	
80	Allergen Elm Tree	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		0.2	SST	Immune		0	
81	Allergen English Plantain	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		0.3	SST	Immune		0	
82	Allergen Giant Ragweed	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		0.3	SST	Immune		0	
83	Allergen Grape	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		0.3	SST	Immune		0	
84	Allergen Hazelnut	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		0.3	SST	Immune		0	
85	Allergen Horse Dander	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		0.3	SST	Immune		0	
86	Allergen House Dust	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		0.3	SST	Immune		0	
87	Allergen June Grass	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		0.3	SST	Immune		0	
88	Allergen Latex	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		0.3	SST	Immune		0	
89	Allergen Melon	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		0.3	SST	Immune		0	
90	Allergen Milk	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		0.3	SST	Immune		0	
91	Allergen Milk Peds	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	12 Years	(ALL)		2.0	SST	Immune		0	
92	Allergen Mouse Epithelium	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		0.3	SST	Immune		0	
93	Allergen Oak Tree	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		0.3	SST	Immune		0	
94	Allergen Oats	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		0.3	SST	Immune		0	
95	Allergen Oats Peds	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	12 Years	(ALL)		2.0	SST	Immune		0	
96	Allergen Orange	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		0.3	SST	Immune		0	
97	Allergen Orchard Grass	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		0.3	SST	Immune		0	
98	Allergen Parrot Feathers	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		0.3	SST	Immune		0	
99	Allergen Peanut	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		0.3	SST	Immune		0	
100	Allergen Peanuts Peds	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	12 Years	(ALL)		2.0	SST	Immune		0	
101	Allergen Pear	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		0.3	SST	Immune		0	
102	Allergen Peas	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		0.3	SST	Immune		0	
103	Allergen Pecan	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		0.3	SST	Immune		0	
104	Allergen Penicillium Notatum	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		0.3	SST	Immune		0	
105	Allergen Pineapple	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		0.3	SST	Immune		0	
106	Allergen Pistachio	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		0.3	SST	Immune		0	
107	Allergen Pork	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		0.3	SST	Immune		0	
108	Allergen Rabbit Epithelium	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		0.3	SST	Immune		0	
109	Allergen Rat Epithelium	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		0.3	SST	Immune		0	
110	Allergen Rice	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		0.3	SST	Immune		0	
111	Allergen Rice	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		0.3	SST	Immune		0	
112	Allergen Rye	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		0.3	SST	Immune		0	
113	Allergen Sesame	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		0.3	SST	Immune		0	
114	Allergen Shrimp	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		0.3	SST	Immune		0	
115	Allergen Shrimp Peds	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	12 Years	(ALL)		2.0	SST	Immune		0	
116	Allergen Soybean	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		0.3	SST	Immune		0	
117	Allergen Soybean Peds	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	12 Years	(ALL)		2.0	SST	Immune		0	
118	Allergen Spinach	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		0.3	SST	Immune		0	
119	Allergen Strawberry	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		0.3	SST	Immune		0	
120	Allergen Sycamore Tree	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		0.3	SST	Immune		0	
121	Allergen Timothy Grass	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		0.3	SST	Immune		0	
122	Allergen Tomato	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		0.3	SST	Immune		0	
123	Allergen Tuna	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		0.3	SST	Immune		0	
124	Allergen Walnut	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		0.3	SST	Immune		0	
125	Allergen Wheat	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		0.3	SST	Immune		0	
126	Allergen Wheat Peds	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	12 Years	(ALL)		2.0	SST	Immune		0	
127	Allergen White Bean	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		0.3	SST	Immune		0	
128	Allergy Panel Animals	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		5.0	SST	Immune		0	
129	Allergy Panel Dust Mites	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		5.0	SST	Immune		0	
130	Allergy Panel Foods	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		5.0	SST	Immune		0	
131	Allergy Panel Inhalants	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		5.0	SST	Immune		0	
132	Allergy Panel Molds	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		5.0	SST	Immune		0	
133	Alpha Fetoprotein Maternal	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		3.0	SST	Ref Refrig		0	
134	Alpha Fetoprotein Tumor Marker	Blood	Chem E170	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		0.4	SST	Chem	Refrigerate	0	
135	Alpha Fucosidase A	Blood	Chem Sendout	Venous Draw	(ALL)	1 Minutes	150 Years	(ALL)		7.0	Yellow (ACD)	Ref Refrig	Keep on Ice	0	
136	Alpha Galactosidase A	Blood	Chem Sendout Frozen	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		10.0	Red	Ref Frozen	Freeze	0	
137	Alpha L Iduronidase	Blood	Blood	Venous Draw	(ALL)	1 Minutes	150 Years	(ALL)		10.0	Green	Chem	Keep on Ice	0	
138	Alpha Subunit	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Red	Ref Refrig	Refrigerate	0	
139	Alpha Subunit	Blood	Chem Sendout	Venous Draw	LabCorp	0 Minutes	150 Years	(ALL)		3.0	Red	Ref Frozen	Freeze	0	
140	Alpha1 Antitrypsin	Blood	Chem Sendout Room	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	SST	Ref Room	Room Temperature	0	
141	Alpha1 Antitrypsin Phenotyping	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		3.0	SST	Ref Refrig	Refrigerate	0	
142	Aluminum Serum	Blood	Chem Royal	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)		1.5	Royal (Serum)	Ref Room	Room Temperature	0	

	Primary Synonym	Specimen Type	Accession Class	Collection Method	Service Resource	Age From	Age To	Coll Priority	Aliq	Min Vol	Container	Collection Class	Special Handling	Add Lbl	Opt
143	Aluminum Serum	Blood	Chem Royal	Venous Draw	(ALL)	1 Months	150 Years	(ALL)		5.0	Royal (Serum)	Ref Room	Room Temperature	0	
144	Aluminum Urine	Urine	Urine	Urine	(ALL)	0 Minutes	150 Years	(ALL)		20.0	U Aliquot	Ref Refrig		0	
145	Amikacin Peak	Serum	Peak	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)		0.2	SST (micro)	Chem		0	
146	Amikacin Peak	Serum	Peak	Venous Draw	(ALL)	1 Months	150 Years	(ALL)		0.2	SST	Chem		0	
147	Amikacin Trough	Serum	Trough	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)		0.3	SST (micro)	Chem	Draw prior to next dose	0	
148	Amikacin Trough	Serum	Trough	Venous Draw	(ALL)	1 Months	150 Years	(ALL)		0.3	SST	Chem	Draw prior to next dose	0	
149	Amino Acids Quant. Urine	Urine	Urine 24hr	Urine	(ALL)	0 Minutes	1 Years	(ALL)		2.0	U Aliquot	Ref Frozen	Freeze	0	
150	Amino Acids Quant. Urine	Urine	Urine 24hr	Urine	(ALL)	1 Years	150 Years	(ALL)		10.0	U Aliquot	Ref Frozen	Freeze	0	
151	Amino Acids Quant. Plasma	Blood	Blood Heparin	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		3.0	Green	Ref Frozen	Freeze	0	
152	Amiodarone	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)		1.0	Red (micro)	Ref Refrig	Refrigerate	0	
153	Amiodarone	Blood	Chem Sendout	Venous Draw	(ALL)	1 Months	150 Years	(ALL)		3.0	Red	Ref Refrig	Refrigerate	0	
154	Amitriptyline	Blood	Chem Sendout Room	Venous Draw	(ALL)	0 Minutes	2 Months	(ALL)		1.0	Red (micro)	Ref Room	Room Temperature	0	
155	Amitriptyline	Blood	Chem Sendout Room	Venous Draw	(ALL)	61 Days	150 Years	(ALL)		3.0	Red	Ref Room	Room Temperature	0	
156	Ammonia	Blood Heparin	Blood Heparin	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)		1.0	Green	Chem	Deliver on ice	0	
157	Ammonia	Blood Heparin	Blood Heparin	Venous Draw	(ALL)	1 Months	150 Years	(ALL)		3.0	Green	Chem	Deliver on ice	0	
158	Amphetamines (GC/MS), Blood	Blood	Chem Sendout Room	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		8.0	Grey	Ref Room	Room Temperature	0	
159	Amylase	Blood	Blood	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)		0.2	SST (micro)	Chem		0	
160	Amylase	Blood	Blood	Venous Draw	(ALL)	1 Months	150 Years	(ALL)		0.3	SST	Chem		0	
161	Amylase Isoenzymes	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		2.0	Red	Ref Refrig	Refrigerate	0	
162	Amylase Timed Urine	Urine	Urine	Urine	(ALL)	0 Minutes	150 Years	(ALL)		2.0	U Aliquot	Urine Chem	Refrigerate	0	
163	Amylase Urine	Urine	Urine	Urine	(ALL)	0 Minutes	150 Years	(ALL)		2.0	UA	Urine Chem		0	
164	Amylase, Body Fluid	Synovial fluid	Chem Sendout Fluid	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Sterile T	Ref Room	Room Temperature	0	
165	Amylase, Body Fluid	Pleural Fluid	Chem Sendout Fluid	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Sterile T	Chem	Room Temperature	0	
166	Amylase, Body Fluid	Pericardial Fluid	Chem Sendout Fluid	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Sterile T	Chem	Room Temperature	0	
167	Amylase, Body Fluid	Ascites Fluid	Chem Sendout Fluid	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Sterile T	Chem	Room Temperature	0	
168	Amylase, Body Fluid	Peritoneal Fluid	Chem Sendout Fluid	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Sterile T	Chem	Room Temperature	0	
169	Amylase, Body Fluid	Pancreatic Fluid	Chem Sendout Fluid	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Sterile T	Chem	Room Temperature	0	
170	Amylase, Body Fluid	Cyst Fluid	Chem Sendout Fluid	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Sterile T	Chem	Room Temperature	0	
171	Androstendione	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	6 Months	(ALL)		2.0	Red (micro)	Ref Refrig	Refrigerate	0	
172	Androstendione	Blood	Chem Sendout	Venous Draw	(ALL)	6 Months	150 Years	(ALL)		5.0	Red	Ref Refrig	Refrigerate	0	
173	Angiotensin Conv Enzy, CSF	CSF	Chem Sendout Fluid	Syringe	(ALL)	0 Minutes	150 Years	(ALL)		0.2	Sterile Screw	Ref Frozen	Freeze	0	
174	Angiotensin Converting Enzyme	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		2.0	SST	Ref Refrig	Refrigerate	0	
175	Anti CCP IgG	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		0.4	SST	Immune		0	
176	Anti Centromere B	Blood	Blood	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		0.5	SST	Immune		0	
177	Anti DNA Antibodies Double Stranded	Blood	Blood	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		0.1	SST	Immune		0	
178	Anti GBM Abs	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		4.0	Red	Ref Refrig	Refrigerate	0	
179	Anti GBM Abs	Blood	Chem Sendout	Venous Draw	LabCorp	0 Minutes	150 Years	(ALL)		3.0	SST	Ref Refrig		0	
180	Anti GM 1	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	SST	Ref Room	Room Temperature	0	
181	Anti Gliadin Antibodies	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		4.0	Red	Ref Refrig	Refrigerate	0	
182	Anti Gliadin Antibodies	Blood	Chem Sendout	Venous Draw	LabCorp	0 Minutes	150 Years	(ALL)		3.0	SST	Ref Refrig		0	
183	Anti JO 1	Blood	Blood	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		0.1	SST	Immune		0	
184	Anti Mitochondrial Antibodies	Blood	Blood	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	SST	Immune		0	
185	Anti Ribonucleoprotein Antibodies	Blood	Blood	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		0.1	SST	Immune		0	
186	Anti Sjogren Syndrome A Antibodies	Blood	Blood	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		0.1	SST	Immune		0	
187	Anti Sjogren Syndrome B Antibodies	Blood	Blood	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		0.1	SST	Immune		0	
188	Anti Smith Antibodies	Blood	Blood	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		0.1	SST	Immune		0	
189	Anti Smooth Muscle Antibodies	Blood	Blood	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	SST	Immune		0	
190	Anti-68 KD(HSP-70) Ab	Blood	Chem Red Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Red	Ref Refrig	Refrigerate	0	
191	Anti-Mullerian Hormone	Blood	Chem Sendout Frozen	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Red	Ref Frozen	Freeze	0	
192	Anticardiolipin Antibodies IgG IgM	Blood	Hem Red	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		3.0	SST	Ref Refrig	Refrigerate	0	
193	Anticardiolipin Antibodies IgM	Blood	Hem Red	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		4.0	SST	Ref Refrig	Refrigerate	0	
194	Antihistone Abs IgG and IgM	Blood	Blood	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		0.1	SST	Immune		0	
195	Antimicrosomal Antibody	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		2.0	SST	Ref Refrig	Refrigerate	0	
196	Antineutrophil Cytoplasmic Antibodies	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		4.0	SST	Ref Refrig	Refrigerate	0	
197	Antinuclear Antibodies	Blood	Blood	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		0.1	SST	Immune		0	
198	Antiphospholipid Ab Panel	Blood	Chem Red Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		6.0	Red	Ref Refrig		0	
199	Antiscleroderma 70 Antibodies	Blood	Blood	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		0.1	SST	Immune		0	
200	Antistreptolysin O Antibodies	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		0.3	SST	Ref Refrig	Refrigerate	0	
201	Antistriated Antibodies	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		4.0	SST	Ref Room	Room Temperature	0	
202	Antithrombin III Antigen (Immunologic)	Blood	Hem Blue Sendouts	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		5.0	Blue	Ref Frozen	Freeze	0	
203	Antithrombin III Functional	Blood	Blood Coag	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		4.5	Blue	Ref Frozen	Freeze	0	
204	Apolipoprotein A 1	Blood	Chem Red Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		4.0	Red	Ref Room	Room Temperature	0	
205	Apolipoprotein B	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		4.0	SST	Ref Refrig	Refrigerate	0	
206	Apolipoprotein B	Blood	Chem Sendout	Venous Draw	LabCorp	0 Minutes	150 Years	(ALL)		4.0	SST	Ref Room		0	
207	Arginine Vasopressin	Blood	Chem Sendout Frozen	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		3.0	Lav	Ref Frozen	Freeze	0	
208	Arsenic Exposure Profile Urine	Urine	Urine 24hr	Urine	(ALL)	0 Minutes	150 Years	(ALL)		20.0	U Aliquot	Ref Refrig		0	
209	Arsenic, Blood	Blood	Chem Royal	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		5.0	Royal (EDTA)	Ref Refrig	Refrigerate	0	
210	Arsenic, Blood	Blood	Chem Royal	Venous Draw	LabCorp	0 Minutes	150 Years	(ALL)		2.0	Royal (EDTA)	Ref Room		0	
211	Arylsulfatase A	Blood	Chem Sendout	Venous Draw	(ALL)	1 Minutes	150 Years	(ALL)		7.0	Yellow (ACD)	Ref Refrig	Refrigerate	0	
212	Arylsulfatase B	Blood	Chem Sendout	Venous Draw	(ALL)	1 Minutes	150 Years	(ALL)		10.0	Green	Ref Refrig	Refrigerate	0	
213	Aspergillus AB Profile	Blood	Chem Red Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		4.0	Red	Ref Refrig		0	

	Primary Synonym	Specimen Type	Accession Class	Collection Method	Service Resource	Age From	Age To	Coll Priority	Aliq	Min Vol	Container	Collection Class	Special Handling	Add Lbl	Opt
214	Aspergillus AB Profile	Blood	Chem Red Sendout	Venous Draw	LabCorp	0 Minutes	150 Years	(ALL)		3.0	SST	Ref Refrig		0	
215	Aspergillus Fumigatus Allergen M3	Blood	Blood	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		0.3	SST	Immune		0	
216	Automated Diff (Lab Order only)	Blood	Blood Hem	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)		0.2	Lav (micro)	Hem		0	
217	Automated Diff (Lab Order only)	Blood	Blood Hem	Venous Draw	(ALL)	1 Months	150 Years	(ALL)		1.0	Lav	Hem		0	
218	B-Cell Rearrange,QL,Cell	Tissue	Chem Sendout Frozen	Tissue	(ALL)	0 Minutes	150 Years	(ALL)		2.0	Contact Laboratory	Ref Frozen	Freeze	0	
219	B-Cell Rearrange,QL,Cell	Blood	Chem lav sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		5.0	Lav	Ref Room	Room Temperature	0	
220	B-Cell Rearrange,QL,Cell	Paraffin Block	Chem Sendout	Tissue	(ALL)	0 Minutes	150 Years	(ALL)		2.0	Slides	Ref Room	Room Temperature	0	
221	B. Pertussis Ab (IgG,M)	Blood	Chem Red Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		2.0	SST	Ref Refrig	Refrigerate	0	
222	BB Director Review	OTHER	Blood Bank	Special Collection	(ALL)	0 Minutes	150 Years	(ALL)		0.1	NoContainer	BldBank		0	
223	BCR-ABL1, CML/ALL, PCR, Quant	Blood	Chem Sendout Room	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		3.0	Lav	Ref Room	Room Temperature	0	
224	BCR-ABL1, CML/ALL, PCR, Quant	Bone Marrow	Chem Sendout Room	Special Collection	(ALL)	0 Minutes	150 Years	(ALL)		2.0	Lav	Ref Room		0	
225	BK Virus DNA, QN PCR	CSF	Chem Sendout Frozen	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Sterile T	Ref Frozen	Freeze	0	
226	BK Virus DNA, QN PCR	Blood	Chem Sendout Frozen	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Lav	Ref Frozen	Freeze	0	
227	BK Virus DNA, QN PCR	Urine	Chem Sendout Frozen	Urine	(ALL)	0 Minutes	150 Years	(ALL)		5.0	UA	Ref Frozen	Freeze	0	
228	BUN	Blood	Blood	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)		0.2	SST (micro)	Chem		0	
229	BUN	Blood	Blood	Venous Draw	(ALL)	1 Months	150 Years	(ALL)		0.2	SST	Chem		0	
230	Barbituates Urine	Urine	Urine	Urine	(ALL)	0 Minutes	150 Years	(ALL)		20.0	U Aliquot	Chem		0	
231	Barbiturates (GC/MS), Blood	Blood	Chem Sendout Room	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		8.0	Grey	Ref Room	Room Temperature	0	
232	Barbiturates ID & QN 1	Blood	Chem Red Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Red	Ref Refrig		0	
233	Bartonella AB IGG IGM	Blood	Chem Red Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Red	Ref Refrig	Refrigerate	0	
234	Basic Metabolic Panel	Blood	Blood	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)		0.5	SST (micro)	Chem		0	
235	Basic Metabolic Panel	Blood	Blood	Venous Draw	(ALL)	1 Months	150 Years	(ALL)		0.5	SST	Chem		0	
236	Benzodiazepines (GC/MS), Blood	Blood	Chem Sendout Room	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		8.0	Grey	Ref Room	Room Temperature	0	
237	Beta 2 Microglobulin Serum	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	6 Months	(ALL)		1.0	SST (micro)	Ref Refrig	Refrigerate	0	
238	Beta 2 Microglobulin Serum	Blood	Chem Sendout	Venous Draw	(ALL)	6 Months	150 Years	(ALL)		3.0	SST	Ref Refrig	Refrigerate	0	
239	Beta 2 Microglobulin Urine	Urine	Urine	Urine	(ALL)	0 Minutes	6 Months	(ALL)		1.0	U Aliquot	Ref Refrig		0	
240	Beta 2 Microglobulin Urine	Urine	Urine	Urine	(ALL)	7 Months	150 Years	(ALL)		10.0	U Aliquot	Ref Refrig		0	
241	Beta 2-Glycoprot (G,A,M)	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		2.0	Red	Ref Refrig	Refrigerate	0	
242	Beta Galactocerebrosidase	Blood	Blood	Venous Draw	(ALL)	1 Minutes	150 Years	(ALL)		10.0	Green	Chem	Keep on Ice	0	
243	Beta Galactosidase	Blood	Blood	Venous Draw	(ALL)	1 Minutes	150 Years	(ALL)		7.0	Yellow (ACD)	Ref Refrig	Keep on Ice	0	
244	Beta Glucosidase	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		10.0	Green	Ref Frozen	Freeze	0	
245	Beta Glucuronidase	Blood	Chem Sendout	Venous Draw	(ALL)	1 Minutes	150 Years	(ALL)		10.0	Green	Ref Frozen	Freeze	0	
246	Beta HCG Qualitative	Blood	Julian Date	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		0.3	SST	Chem		0	
247	Beta HCG Quantitative	Blood	Chem Architect	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	SST	Chem		0	
248	Beta-2, Transferrin	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		2.0	SST	Ref Refrig		0	
249	Beta-2, Transferrin	Body Fluid	Body Fluid	Syringe	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Sterile Screw	Ref Frozen	Freeze	0	
250	Beta-2, Transferrin	Nasal Fluid	Chem Sendout Frozen	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)		0.5	Sterile Screw	Ref Frozen	Freeze	0	
251	Bile Acids, Fractionated	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		4.0	SST	Ref Refrig	Refrigerate	0	
252	Bile Acids, Fractionated	Blood	Chem Sendout	Venous Draw	LabCorp	0 Minutes	150 Years	(ALL)		3.0	SST	Ref Frozen	Freeze	0	
253	Bile Acids, Urine	Urine	Chem Sendout Frozen	Urine	(ALL)	0 Minutes	150 Years	(ALL)		50.0	UA	Chem	Freeze	0	
254	Bilirubin Amniotic Fluid	Amniotic Fluid	Body Fluid	Special Collection	(ALL)	0 Minutes	150 Years	(ALL)		5.0	Sterile Plastic	Chem	Protect from light	0	
255	Bilirubin Direct	Blood	Blood	Venous Draw	(ALL)	15 Days	1 Months	(ALL)		0.3	SST (micro)	Chem		0	
256	Bilirubin Direct	Blood	Blood	Venous Draw	(ALL)	1 Months	150 Years	(ALL)		0.3	SST	Chem		0	
257	Bilirubin Total	Blood	Blood	Venous Draw	(ALL)	0 Minutes	2 Years	(ALL)		0.3	SST (micro)	Chem		0	
258	Bilirubin Total	Blood	Blood	Venous Draw	(ALL)	2 Years	150 Years	(ALL)		0.3	SST	Chem		0	
259	Bilirubin Total & Direct	Blood	Blood	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)		0.4	SST (micro)	Chem		0	
260	Bilirubin Total & Direct	Blood	Blood	Venous Draw	(ALL)	1 Months	150 Years	(ALL)		0.6	SST	Chem		0	
261	Bilirubin Total Peritoneal Fluid	Peritoneal Fluid	Body Fluid	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)		0.3	Red	Chem		0	
262	Blood Gases Capillary	Blood Heparin	Blood Gases Capillary	Syringe	(ALL)	0 Minutes	150 Years	(ALL)		0.2	Capillary	Chem	Deliver Immediately	0	
263	Blood Gases Intra Operative	Blood Heparin	Blood Gases Arterial	Syringe	(ALL)	0 Minutes	150 Years	(ALL)		3.0	Hep Syringe	Chem		0	
264	Blood Gases Arterial	Blood Heparin	Blood Gases Arterial	Syringe	(ALL)	0 Minutes	150 Years	(ALL)		0.1	Hep Syringe	Chem	Deliver Immediately	0	
265	Blood Gases Perfusate	Perfusate	Body Fluid	Syringe	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Hep Syringe	Chem		0	
266	Blood Gases Venous	Blood Heparin	Blood Gases Venous	Syringe	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Hep Syringe	Chem	Deliver Immediately	0	
267	Bone Marrow Interpretation	Bone Marrow Asp	Blood Hem	Slide	(ALL)	0 Minutes	150 Years	(ALL)		2.0	Slides	Hem	Deliver Immediately	0	
268	Brain Natriuretic Peptide	Blood	Hgb Alc	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		0.5	Lav	Chem		0	
269	Brucella Antibodies	Blood	Chem Red Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		4.0	Red	Ref Room	Room Temperature	0	
270	C Peptide	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		3.0	SST	Ref Refrig		0	
271	C Peptide #1	Blood	Blood	Venous Draw	(ALL)	0 Minutes	6 Months	(ALL)		1.0	SST (micro)	Chem	Freeze	0	
272	C Peptide #1	Blood	Blood	Venous Draw	(ALL)	7 Months	150 Years	(ALL)		5.0	SST	Chem	Freeze	0	
273	C Peptide #2	Blood	Blood	Venous Draw	(ALL)	0 Minutes	6 Months	(ALL)		1.0	SST (micro)	Chem	Freeze	0	
274	C Peptide #2	Blood	Blood	Venous Draw	(ALL)	7 Months	150 Years	(ALL)		5.0	SST	Chem	Freeze	0	
275	C Peptide #3	Blood	Blood	Venous Draw	(ALL)	0 Minutes	6 Months	(ALL)		1.0	SST (micro)	Chem	Freeze	0	
276	C Peptide #3	Blood	Blood	Venous Draw	(ALL)	7 Months	150 Years	(ALL)		5.0	SST	Chem	Freeze	0	
277	C Peptide #4	Blood	Blood	Venous Draw	(ALL)	0 Minutes	6 Months	(ALL)		1.0	SST (micro)	Chem	Freeze	0	
278	C Peptide #4	Blood	Blood	Venous Draw	(ALL)	7 Months	150 Years	(ALL)		5.0	SST	Chem	Freeze	0	
279	C Peptide #5	Blood	Blood	Venous Draw	(ALL)	0 Minutes	6 Months	(ALL)		1.0	SST (micro)	Chem	Freeze	0	
280	C Peptide #5	Blood	Blood	Venous Draw	(ALL)	7 Months	150 Years	(ALL)		5.0	SST	Chem	Freeze	0	
281	C Peptide #6	Blood	Blood	Venous Draw	(ALL)	0 Minutes	6 Months	(ALL)		1.0	SST (micro)	Chem	Freeze	0	
282	C Peptide #6	Blood	Blood	Venous Draw	(ALL)	7 Months	150 Years	(ALL)		5.0	SST	Chem	Freeze	0	
283	C Reactive Protein Serum	Blood	Blood	Venous Draw	(ALL)	0 Minutes	31 Days	(ALL)		0.3	SST (micro)	Chem		0	
284	C Reactive Protein Serum	Blood	Blood	Venous Draw	(ALL)	31 Days	150 Years	(ALL)		1.0	SST	Chem		0	

	Primary Synonym	Specimen Type	Accession Class	Collection Method	Service Resource	Age From	Age To	Coll Priority	Aliq	Min Vol	Container	Collection Class	Special Handling	Add Lbl	Opt
285	C1 Esterase Inhibitor	Blood	Chem Sendout Frozen	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		4.0	SST	Ref Frozen	Freeze	0	
286	C1 Esterase Inhibitor	Blood	Chem Sendout Frozen	Venous Draw	LabCorp	0 Minutes	150 Years	(ALL)		3.0	Red	Ref Frozen		0	
287	C3D Immune Complex	Blood	Chem Sendout Frozen	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Red	Ref Frozen	Freeze	0	
288	CBC	Blood	Blood Hem	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)		0.2	Lav (micro)	Hem		0	
289	CBC	Blood	Blood Hem	Venous Draw	(ALL)	1 Months	150 Years	(ALL)		1.0	Lav	Hem		0	
290	CBC with Diff	Blood	Blood Hem	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)		0.2	Lav (micro)	Hem		0	
291	CBC with Diff	Blood	Blood Hem	Venous Draw	(ALL)	1 Months	150 Years	(ALL)		1.0	Lav	Hem		0	
292	CEA	Blood	Chem Architect	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	SST	Chem	Refrigerate	0	
293	CEA, Fluid	Synovial fluid	Chem Sendout Fluid	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Sterile T	Ref Room	Room Temperature	0	
294	CEA, Fluid	Pleural Fluid	Chem Sendout Fluid	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Sterile T	Ref Room	Room Temperature	0	
295	CEA, Fluid	Pericardial Fluid	Chem Sendout Fluid	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Sterile T	Ref Room	Room Temperature	0	
296	CEA, Fluid	Ascites Fluid	Chem Sendout Fluid	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Sterile T	Ref Room	Room Temperature	0	
297	CEA, Fluid	Peritoneal Fluid	Chem Sendout Fluid	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Sterile T	Ref Room	Room Temperature	0	
298	CEA, Fluid	Pancreatic Fluid	Chem Sendout Fluid	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Sterile T	Ref Room	Room Temperature	0	
299	CEA, Fluid	Cyst Fluid	Chem Sendout Fluid	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Sterile T	Ref Room	Room Temperature	0	
300	CF Carrier Screen	Blood	GC Genetics Cyxtic Fibrosis	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		5.0	Lav	MolecularC		0	
301	CK	Blood	Blood	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)		0.3	SST (micro)	Chem		0	
302	CK	Blood	Blood	Venous Draw	(ALL)	31 Days	150 Years	(ALL)		0.3	SST	Chem		0	
303	CK MB	Blood	Blood	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)		0.3	SST (micro)	Chem		0	
304	CK MB	Blood	Blood	Venous Draw	(ALL)	31 Days	150 Years	(ALL)		0.3	SST	Chem		0	
305	CLL/Lymphoma,Diagnostic	Tissue	Chem Sendout	Tissue	(ALL)	0 Minutes	150 Years	(ALL)		5.0	Screw Capped	Ref Refrig	Refrigerate	0	
306	CLL/Lymphoma,Diagnostic	Tissue	Chem Sendout	Tissue	(ALL)	0 Minutes	150 Years	(ALL)		4.0	Slides	Ref Refrig		0	
307	CLL/Lymphoma,Diagnostic	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		5.0	Green Sodium	Ref Room	Room Temperature	0	
308	CLL/Lymphoma,Diagnostic	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		3.0	Lav	Ref Room	Room Temperature	0	
309	CLL/Lymphoma,Diagnostic	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		2.0	Slides	Ref Room	Room Temperature	0	
310	CLL/Lymphoma,Diagnostic	Body Fluid	Chem Sendout	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)		2.0	Sterile T	Ref Refrig	Refrigerate	0	
311	CLL/Lymphoma,Diagnostic	Bone Marrow	Chem lav sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		3.0	Lav	Ref Room	Room Temperature	0	
312	CLL/Lymphoma,Diagnostic	Bone Marrow	Chem lav sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		3.0	Green Sodium	Ref Room	Room Temperature	0	
313	CLL/Lymphoma,Diagnostic	Bone Marrow	Chem lav sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		2.0	Slides	Ref Room	Room Temperature	0	
314	CMV Antibodies IgM Quantitative	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		2.0	SST	Ref Refrig	Refrigerate	0	
315	CMV DNA,QL,Real-Time PCR	CSF	Chem Sendout Frozen	Syringe	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Sterile T	Ref Frozen	Freeze	0	
316	CMV DNA,QL,Real-Time PCR	Blood	Chem lav sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Lav	Ref Refrig	Refrigerate	0	
317	CMV DNA,QL,Real-Time PCR	Urine	Chem Sendout Frozen	Urine	(ALL)	0 Minutes	150 Years	(ALL)		2.0	Sterile Screw	Ref Frozen	Freeze	0	
318	CMV DNA,QL,Real-Time PCR	Amniotic Fluid	Chem Sendout Frozen	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Sterile T	Ref Frozen	Freeze	0	
319	CMV DNA,QN,Real-Time PCR	Blood	Chem Sendout Frozen	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		3.0	Lav	Ref Frozen	Freeze	0	
320	CMV Immune Status IgG by EIA	Blood	VS	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		0.3	SST	Serology		0	
321	Cadmium Blood	Blood	Chem Royal	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		2.0	Royal (EDTA)	Ref Refrig	Refrigerate	0	
322	Cadmium Urine	Urine	Urine	Urine	(ALL)	0 Minutes	150 Years	(ALL)		20.0	U Aliquot	Ref Refrig		0	
323	Caffeine	Blood	Blood	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		3.0	SST	Chem		0	
324	Calcitonin	Blood	Chem Sendout Frozen	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		5.0	SST	Ref Frozen	Freeze	0	
325	Calcium	Blood	Blood	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)		0.2	SST (micro)	Chem		0	
326	Calcium	Blood	Blood	Venous Draw	(ALL)	1 Months	150 Years	(ALL)		0.2	SST	Chem		0	
327	Calcium Urine Random	Urine	Urine	Urine	(ALL)	0 Minutes	150 Years	(ALL)		2.0	UA	Urine Chem		0	
328	Calcium Ionized Serum	Blood	Blood	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)		0.3	Red (micro)	Chem		0	
329	Calcium Ionized Serum	Blood	Blood	Venous Draw	(ALL)	31 Days	150 Years	(ALL)		0.3	Red	Chem		0	
330	Calcium Ionized Whole Blood	Blood Heparin	Blood Gases Arterial	Syringe	(ALL)	0 Minutes	150 Years	(ALL)		0.1	Hep Syringe	Chem	Deliver Immediately	0	
331	Calcium Urine 24Hr	Urine	Urine 24hr	Urine	(ALL)	0 Minutes	150 Years	(ALL)		2.0	U24H	Urine Chem	Refrigerate	0	
332	Calcium/Creatinine Clearance Ratio	Urine	Urine 24hr	Urine	(ALL)	0 Minutes	150 Years	(ALL)		1.0	U24H	24 Hr Urine	Refrigerate	0	
333	Calcium_Creatinine Clearance	Urine	Urine 24hr	Urine	(ALL)	0 Minutes	150 Years	(ALL)		1.0	U24H	24 Hr Urine	Refrigerate	0	
334	Calculi Urinary	Calculi	Urine	Special Collection	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Transport	Ref Room	Room Temperature	0	
335	Campylobacter Jejuni Ab	Blood	Chem Red Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Red	Ref Refrig	Refrigerate	0	
336	Cancer Antigen 125	Blood	Chem E170	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		5.0	SST	Chem	Freeze	0	
337	Cancer Antigen 27 29	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		3.0	SST	Ref Refrig	Refrigerate	0	
338	Cancer Antigen(CA)15-3	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	SST	Ref Refrig	Refrigerate	0	
339	Candida Antibodies	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		4.0	SST	Ref Refrig		0	
340	Cannabinoid (GC/MS), Blood	Blood	Chem Sendout Room	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		8.0	Grey	Ref Room	Room Temperature	0	
341	Carbamazapine	Blood	Blood	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)		0.2	SST (micro)	Chem		0	
342	Carbamazapine	Blood	Blood	Venous Draw	(ALL)	31 Days	150 Years	(ALL)		0.5	SST	Chem	Draw prior to next dose	0	
343	Carbohydrate Antigen 19 9	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		3.0	SST	Ref Refrig	Refrigerate	0	
344	Carbon Dioxide	Blood	Blood	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)		0.2	SST (micro)	Chem		0	
345	Carbon Dioxide	Blood	Blood	Venous Draw	(ALL)	1 Months	150 Years	(ALL)		0.2	SST	Chem		0	
346	Carbon Monoxide	Blood Heparin	Blood Gases Arterial	Arterial Draw	(ALL)	0 Minutes	150 Years	(ALL)		2.0	Hep Syringe	Chem	Deliver Immediately	0	
347	Cardio CRP	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	SST	Ref Refrig		0	
348	Cardiolipin Ab (IgG)	Blood	Hem Red	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		2.0	SST	Ref Refrig	Refrigerate	0	
349	Cardiolipin Ab (IgG,A,M)	Blood	Hem Red	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		5.0	SST	Ref Refrig	Refrigerate	0	
350	Carnitine Total and Free	Blood	Chem Red Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Red	Ref Refrig	Refrigerate	0	
351	Carnitine Total and Free	Blood	Chem Red Sendout	Venous Draw	LabCorp	0 Minutes	150 Years	(ALL)		3.0	Green	Ref Frozen	Freeze	0	
352	Carotene	Blood	Chem Sendout Frozen	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		5.0	SST	Ref Frozen	Protect from light	0	
353	Catecholamines Plasma	Blood	Chem Sendout Frozen	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		2.0	Green	Ref Frozen	Freeze	0	
354	Catecholamines Fractionated Urinary Free	Urine	Urine HCL	Urine	(ALL)	0 Minutes	150 Years	(ALL)		30.0	U24H HCL	Ref Refrig	Refrigerate	0	
355	Celiac Comp Panel , Infant (0-3 yrs)	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		5.0	Red	Ref Refrig	Refrigerate	0	

	Primary Synonym	Specimen Type	Accession Class	Collection Method	Service Resource	Age From	Age To	Coll Priority	Aliq	Min Vol	Container	Collection Class	Special Handling	Add Lbl	Opt
356	Celiac Disease Ab Screen w/Rfx	Blood	Chem Sendout Frozen	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		3.0	SST	Ref Room		0	
357	Celiac Disease Comp Panel	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		2.0	Red	Ref Refrig	Refrigerate	0	
358	Cell Count Abscess Drainage	Abcess Drainage	Body Fluid Hem	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Lav	Hem	Deliver Immediately	0	
359	Cell Count CSF	CSF	CSF Hem	Syringe	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Sterile T	Hem	Deliver Immediately	0	
360	Cell Count Dialysate Fluid	Dialysis Fluid	Body Fluid Hem	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Lav	Hem	Deliver Immediately	0	
361	Cell Count J Tube Drainage	J-Tube Drainage	Body Fluid Hem	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Lav	Hem	Deliver Immediately	0	
362	Cell Count Pericardial Fluid	Pericardial Fluid	Body Fluid Hem	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Lav	Hem	Deliver Immediately	0	
363	Cell Count Peritoneal Fluid	Peritoneal Fluid	Body Fluid Hem	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Lav	Hem	Deliver Immediately	0	
364	Cell Count Pleural Fluid	Pleural Fluid	Body Fluid Hem	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Lav	Hem	Deliver Immediately	0	
365	Cell Count Stem Cell	Stem Cell	Blood	Stem Cell	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Lav	Hem		0	
366	Cell Count Synovial Fluid	Synovial fluid	Body Fluid Hem	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)		0.1	Lav	Hem	Deliver Immediately	0	
367	Cell Count Body Fluid	Synovial fluid	Body Fluid Hem	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)		2.0	Lav	Hem		0	
368	Cell Count Body Fluid	Body Fluid	Body Fluid Hem	Aspirate	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Lav	Hem		0	
369	Cell Count Body Fluid	Pleural Fluid	Body Fluid Hem	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)		2.0	Lav	Hem		0	
370	Cell Count Body Fluid	Pericardial Fluid	Body Fluid Hem	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)		2.0	Lav	Hem		0	
371	Cell Count Body Fluid	Abcess Drainage	Body Fluid Hem	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)		2.0	Lav	Hem		0	
372	Cell Count Body Fluid	Dialysis Fluid	Body Fluid Hem	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)		2.0	Lav	Hem		0	
373	Cell Count Body Fluid	J-Tube Drainage	Body Fluid Hem	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)		2.0	Lav	Hem		0	
374	Cell Count Body Fluid	Peritoneal Fluid	Body Fluid Hem	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)		2.0	Lav	Hem		0	
375	Ceruloplasmin	Blood	Chem Sendout	Venous Draw	LabCorp	0 Minutes	150 Years	(ALL)		3.0	SST	Ref Refrig		0	
376	Chlamydia Antibodies IgG	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		3.0	SST	Ref Refrig		0	
377	Chlamydia DNA by PCR	Urethral Swab	VM	Dacron Swab	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Vir/Chlam	Virol	Refrigerate	0	
378	Chlamydia DNA by PCR	Cervical Swab	VM	Dacron Swab	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Vir/Chlam	Virol	Refrigerate	0	
379	Chlamydia Species Ab(IgG)	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Red	Ref Refrig	Refrigerate	0	
380	Chlamydia Species Ab(IgG)	Blood	Chem Sendout	Venous Draw	LabCorp	0 Minutes	150 Years	(ALL)		1.0	Red	Ref Room		0	
381	Chlamydia Species Ab(IgM)	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Red	Ref Refrig	Refrigerate	0	
382	Chlamydia/GC DNA, SDA	Urine	Chem Sendout	Urine	(ALL)	0 Minutes	150 Years	(ALL)		2.0	UA	Ref Refrig	Refrigerate	0	
383	Chloride	Blood	Blood	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)		0.2	SST (micro)	Chem		0	
384	Chloride	Blood	Blood	Venous Draw	(ALL)	1 Months	150 Years	(ALL)		0.2	SST	Chem		0	
385	Chloride Blood	Blood Heparin	Blood Gases Arterial	Syringe	(ALL)	0 Minutes	150 Years	(ALL)		0.1	Hep Syringe	Chem	Deliver Immediately	0	
386	Chloride Urine 24Hr	Urine	Urine 24hr	Urine	(ALL)	0 Minutes	150 Years	(ALL)		2.0	U24H	Urine Chem	Refrigerate	0	
387	Chloride Cardioplegia Solution	Cardioplegia	Body Fluid	Pharmacy	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Red	Chem		0	
388	Chloride Urine Random	Urine	Urine	(ALL)	0 Minutes	150 Years	(ALL)	(ALL)		2.0	UA	Urine Chem		0	
389	Chlorpheniramine	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		5.0	Red	Ref Refrig	Refrigerate	0	
390	Cholesterol	Blood	Blood	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)		0.3	SST (micro)	Chem		0	
391	Cholesterol	Blood	Blood	Venous Draw	(ALL)	31 Days	150 Years	(ALL)		0.3	SST	Chem		0	
392	Cholinesterase	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		0.5	SST	Ref Refrig	Refrigerate	0	
393	Chromatin Ab (IgG)	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Red	Ref Refrig	Refrigerate	0	
394	Chromium Plasma	Blood	Chem Royal	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		5.0	Royal (EDTA)	Ref Refrig	Refrigerate	0	
395	Chromium Urine	Urine	Urine 24hr	Urine	(ALL)	0 Minutes	150 Years	(ALL)		20.0	U Aliquot	Ref Room	Room Temperature	0	
396	Chromogranin A	Blood	Chem Red Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Red	Ref Refrig	Refrigerate	0	
397	Chromogranin A	Blood	Chem Red Sendout	Venous Draw	LabCorp	0 Minutes	150 Years	(ALL)		1.0	SST	Ref Frozen		0	
398	Chromosome Fish Technique	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		10.0	Green	Chem	Keep on Ice	0	
399	Chymotrypsin, Stool	Stool	Stool	Stool	(ALL)	0 Minutes	150 Years	(ALL)		2.0	Transport	Ref Refrig	Refrigerate	0	
400	Circulating Immune Complexes	Blood	Chem Sendout Frozen	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		0.3	Red	Ref Frozen	Freeze	0	
401	Citric Acid, 24HR UR	Urine	Urine Boric	Urine	(ALL)	0 Minutes	150 Years	(ALL)		25.0	U24H Boric	Ref Refrig	Refrigerate	0	
402	Clomipramine & Metabolite	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Royal (Serum)	Ref Refrig	Refrigerate	0	
403	Clonazepam	Blood	Chem Sendout Room	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		5.0	Red	Ref Room	Room Temperature	0	
404	Clozapine	Blood	Chem Sendout Frozen	Venous Draw	LabCorp	0 Minutes	150 Years	(ALL)		3.0	Lav	Ref Refrig		0	
405	Cocaine Metabolite GC/MS,Blood	Blood	Chem Sendout Room	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		8.0	Grey	Ref Room	Room Temperature	0	
406	Coccidioides Antibodies	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		4.0	Red	Ref Refrig	Refrigerate	0	
407	Coccidioides Antibodies CSF	CSF	CSF	Syringe	(ALL)	0 Minutes	150 Years	(ALL)		2.0	Sterile T	Ref Refrig	Refrigerate	0	
408	Cold Hemagglutinins	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		3.0	Red	Ref Room	Room Temperature	0	
409	Collagen I C-Telopep(CTX)	Blood	Chem Sendout Frozen	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		2.0	Red	Ref Frozen	Freeze	0	
410	Complement Total	Blood	Chem Sendout Frozen	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Red	Ref Frozen	Freeze	0	
411	Complement C1 Esterase Inhibitor Functional	Blood	Chem Sendout Frozen	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		5.0	Red	Ref Frozen	Freeze	0	
412	Complement C1q	Blood	Chem Sendout Frozen	Venous Draw	LabCorp	0 Minutes	150 Years	(ALL)		3.0	Red	Ref Frozen	Freeze	0	
413	Complement C2	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		5.0	Red	Ref Refrig	Refrigerate	0	
414	Complement C2	Blood	Chem Sendout	Venous Draw	LabCorp	0 Minutes	150 Years	(ALL)		5.0	Red	Ref Frozen		0	
415	Complement C3	Blood	Blood	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		0.5	SST	Chem		0	
416	Complement C3 Activator	Blood	Chem Sendout Frozen	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		5.0	Red	Ref Frozen	Freeze	0	
417	Complement C3 Activator	Blood	Chem Sendout Frozen	Venous Draw	LabCorp	0 Minutes	150 Years	(ALL)		4.0	Lav	Ref Frozen		0	
418	Complement C4	Blood	Blood	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		0.5	SST	Chem		0	
419	Comprehensive Metabolic Panel	Blood	Blood	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)		0.5	SST (micro)	Chem		0	
420	Comprehensive Metabolic Panel	Blood	Blood	Venous Draw	(ALL)	1 Months	150 Years	(ALL)		0.5	SST	Chem		0	
421	Copath_ Reflex	Thin Prep	Genetics HPV	Thin Prep	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Thin Prep	MolecularC		0	
422	Copper	Blood	Chem Royal	Venous Draw	LabCorp	0 Minutes	150 Years	(ALL)		1.0	Red	Ref Room	Room Temperature	0	
423	Copper Urine Random	Urine	Urine	Urine	(ALL)	0 Minutes	150 Years	(ALL)		0.3	U Aliquot	Ref Room	Room Temperature	0	
424	Copper Urine 24hr	Urine	Urine 24hr	Urine	(ALL)	0 Minutes	150 Years	(ALL)		7.0	Contact Laboratory	Ref Refrig	Refrigerate	0	
425	Copper Urine 24hr	Urine	Urine 24hr	Urine	LabCorp	0 Minutes	150 Years	(ALL)		5.0	U24H	Ref Room		0	
426	Cord Blood Gas	Blood Heparin	Blood Gases Cord	Syringe	(ALL)	0 Minutes	150 Years	(ALL)		0.1	Hep Syringe	Chem		0	

	Primary Synonym	Specimen Type	Accession Class	Collection Method	Service Resource	Age From	Age To	Coll Priority	Aliq	Min Vol	Container	Collection Class	Special Handling	Add Lbl	Opt
427	Cortisol AM	Blood	Chem E170	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	SST	SP Chem		0	
428	Cortisol PM	Blood	Chem E170 2	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	SST	SP Chem		0	
429	Cortisol Urinary Free	Urine	Urine	Urine	(ALL)	0 Minutes	150 Years	(ALL)		10.0	U24H	Ref Refrig	Refrigerate	0	
430	Cortisol Stimulation	Blood	Blood	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	SST	SP Chem		0	
431	Coxsackie B Abs by CF	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		5.0	Red	Ref Refrig	Refrigerate	0	
432	Creatinine	Blood	Blood	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)		0.2	SST (micro)	Chem		0	
433	Creatinine	Blood	Blood	Venous Draw	(ALL)	1 Months	150 Years	(ALL)		0.2	SST	Chem		0	
434	Creatinine Urine 24Hr	Urine	Urine 24hr	Urine	(ALL)	0 Minutes	150 Years	(ALL)		2.0	U24H	Urine Chem	Refrigerate	0	
435	Creatinine Urine Random	Urine	Urine	Urine	(ALL)	0 Minutes	150 Years	(ALL)		2.0	UA	Urine Chem		0	
436	Creatinine Peritoneal Dialysate	DIALY	Body Fluid	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)		0.5	Red	Chem		0	
437	Creatinine Urine Timed	Urine	Urine	Urine	(ALL)	0 Minutes	150 Years	(ALL)		2.0	UA	Urine Chem	Refrigerate	0	
438	Cross Linked N Telopeptide Urine	Urine	Urine	Urine	(ALL)	0 Minutes	150 Years	(ALL)		10.0	UA	Ref Frozen	Freeze	0	
439	Cryofibrinogen Screen	Blood	Blood Hem	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		2.0	Blue	Hem	Incubate at 37.C	0	
440	Cryoglobulin Screen	Blood	Hem Red	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		3.0	Red	Hem	Incubate at 37.C	0	
441	Cryptococcus Antibodies	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		5.0	Red	Ref Refrig	Refrigerate	0	
442	Cryptococcus Antibodies	Blood	Chem Sendout	Venous Draw	LabCorp	0 Minutes	150 Years	(ALL)		3.0	SST	Ref Room		0	
443	Crystal ID Body Fluids	Synovial fluid	Body Fluid	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)		0.3	Green	Hem		0	
444	Cyanide Blood	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		2.0	Grey	Ref Refrig	Refrigerate	0	
445	Cyclosporine	Blood	CHEM LAV	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		2.0	Lav	Chem		0	
446	Cysticercosis (Taenia solium)	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		2.0	SST	Ref Refrig		0	
447	Cystine, QN,24 Hour Urine	Urine	Urine 24hr	Urine	(ALL)	0 Minutes	150 Years	(ALL)		2.0	U24H HCL	Ref Refrig	Refrigerate	0	
448	Cytoch P450 2C9,VKORC1,NY	Blood	Chem lav sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		5.0	Lav	Ref Room	Room Temperature	0	
449	Cytochrome P450 2C9 (NY)	Blood	Chem lav sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		5.0	Lav	Ref Room	Room Temperature	0	
450	Cytogenetics Amniotic Fluid	Amniotic Fluid	Body Fluid	Special Collection	(ALL)	0 Minutes	150 Years	(ALL)		30.0	Sterile Plastic	Chem	Room Temperature	0	
451	Cytogenetics Bone Marrow	Bone Marrow	Blood	Special Collection	(ALL)	0 Minutes	150 Years	(ALL)		2.0	Green	Chem	Room Temperature	0	
452	Cytogenetics Fragile X	Blood	EDTA	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		9.0	Lav	Chem	Room Temperature	0	
453	Cytogenetics High Resolution	Blood	Blood	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		10.0	Green	Chem	Keep on Ice	0	
454	Cytogenetics Peripheral Blood	Blood	Blood Heparin	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		3.0	Green	Chem	Refrigerate	0	
455	Cytogenetics Tissue	Tissue	Special Collectio	Special Collection	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Sterile Plastic	Chem	Refrigerate	0	
456	D Dimer	Blood	Blood Coag	Venous Draw	(ALL)	0 Minutes	12 Years	(ALL)		0.5	Blue	Coag	Mix Immediately	0	
457	D Dimer	Blood	Blood Coag	Venous Draw	(ALL)	12 Years	150 Years	(ALL)		0.5	Blue	Coag	Mix Immediately	0	
458	DBA.44,IHC W/O Interp	Paraffin Block	Chem Sendout	Special Collection	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Contact Laboratory	Ref Room	Room Temperature	0	
459	DHEA	Blood	Chem Red Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		5.0	Red	Ref Refrig	Refrigerate	0	
460	DHEA	Blood	Chem Red Sendout	Venous Draw	LabCorp	0 Minutes	150 Years	(ALL)		3.0	SST	Ref Frozen		0	
461	DHEAS	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		3.0	SST	Ref Refrig	Refrigerate	0	
462	DMCB Influenza A & B Rapid Antigen	Nasopharyn Wash	VC	Vial	(ALL)	0 Minutes	150 Years	(ALL)		3.0	Sterile Plastic	Serology		0	
463	DMCB Influenza A & B Rapid Antigen	Nasopharyn Wash	VC	Vial	(ALL)	0 Minutes	150 Years	(ALL)		3.0	Sterile Plastic	Serology		0	
464	DMCB Influenza A & B Rapid Antigen	Nasopharyn Wash	VC	Vial	(ALL)	0 Minutes	150 Years	(ALL)		3.0	Sterile Plastic	Serology		0	
465	DMCB Influenza A & B Rapid Antigen	Nasopharyn Swab	VC	Swab	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Cx Swab	Serology		0	
466	DNA Cell Cycle Analysis	Tissue	Chem Sendout	Special Collection	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Contact Laboratory	Ref Room	Room Temperature	0	
467	DPD 5-Fluorouracil	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		3.0	Lav	Ref Room		0	
468	Dengue Fever (IgG,M)	Blood	Chem Red Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Red	Ref Room		0	
469	Deoxycorticosterone	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		2.0	SST	Ref Refrig	Refrigerate	0	
470	Desipramine	Blood	Chem Red Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		5.0	Red	Ref Refrig	Refrigerate	0	
471	Dexamethasone	Blood	Chem Red Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Red	Ref Refrig	Refrigerate	0	
472	Digoxin	Blood	Blood	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)		0.3	SST (micro)	Chem		0	
473	Digoxin	Blood	Blood	Venous Draw	(ALL)	31 Days	150 Years	(ALL)		0.3	SST	Chem	Draw prior to next dose	0	
474	Dihydrotestosterone	Blood	Chem Red Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		2.0	Red	Ref Refrig	Refrigerate	0	
475	Dihydrotestosterone	Blood	Chem Red Sendout	Venous Draw	LabCorp	0 Minutes	150 Years	(ALL)		3.0	SST	Ref Frozen		0	
476	Diphtheria Antitoxoid	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		3.0	SST	Ref Refrig	Refrigerate	0	
477	Diphtheria Antitoxoid	Blood	Chem Sendout	Venous Draw	LabCorp	0 Minutes	150 Years	(ALL)		3.0	SST	Ref Room		0	
478	Disaccharidases	Biopsy	Chem Sendout Frozen	Biopsy	(ALL)	0 Minutes	150 Years	(ALL)		5.0	Sterile Screw	Ref Frozen	Freeze	0	
479	Disialo Carbohydrate Def. Tran.	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	SST	Ref Refrig		0	
480	Disopyramide	Blood	Chem Red Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		3.0	Red	Ref Refrig	Refrigerate	0	
481	Drug Profile 7 Drugs Blood	Blood	Chem Sendout Room	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		7.0	Grey	Ref Room	Room Temperature	0	
482	Drug Screen COGA Urine	Urine	Urine	Urine	(ALL)	0 Minutes	150 Years	(ALL)		15.0	U Aliquot	Ref Refrig		0	
483	E. Histolytica (IgG), Elisa	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	SST	Ref Refrig		0	
484	EBV Acute Infection Antibodies	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		8.5	SST	Ref Refrig	Refrigerate	0	
485	EBV DNA, QN, PCR	Blood	Chem Sendout Frozen	Venous Draw	LabCorp	0 Minutes	150 Years	(ALL)		2.0	Lav	Ref Frozen	Freeze	0	
486	EBV Early Antigen IgG	Blood	Chem Sendout	Venous Draw	LabCorp	0 Minutes	150 Years	(ALL)		1.0	SST	Ref Room		0	
487	EBV Viral Capsid Antigen IgG	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		0.3	SST	Ref Refrig		0	
488	EBV Viral Capsid Antigen IgG	Blood	Chem Sendout	Venous Draw	LabCorp	0 Minutes	150 Years	(ALL)		3.0	SST	Ref Room		0	
489	ED-Rapid Elisa D-dimer	Blood	Blood Coag	Venous Draw	(ALL)	0 Minutes	12 Years	(ALL)		0.5	Blue	Coag	Mix Immediately	0	
490	ED-Rapid Elisa D-dimer	Blood	Blood Coag	Venous Draw	(ALL)	12 Years	150 Years	(ALL)		0.5	Blue	Coag	Mix Immediately	0	
491	EGFR Mutation Analysis	Paraffin Block	Chem Sendout	Special Collection	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Contact Laboratory	Ref Room	Room Temperature	0	
492	ENA I	Blood	Blood	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		0.1	SST	Immune		0	
493	ENA II	Blood	Blood	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		0.1	SST	Immune		0	
494	Echinococcus (P2) IGE	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		5.0	SST	Ref Room	Room Temperature	0	
495	Electrolyte Panel	Blood	Blood	Venous Draw	(ALL)	0 Minutes	2 Years	(ALL)		0.5	SST (micro)	Chem		0	
496	Electrolyte Panel	Blood	Blood	Venous Draw	(ALL)	2 Years	150 Years	(ALL)		0.5	SST	Chem		0	
497	Endomysial Scr (IgA)WRFL	Blood	Chem Red Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Red	Ref Refrig	Refrigerate	0	

	Primary Synonym	Specimen Type	Accession Class	Collection Method	Service Resource	Age From	Age To	Coll Priority	Aliq	Min Vol	Container	Collection Class	Special Handling	Add Lbl	Opt
498	Environmental Culture	Dialysate	MB Other	Vial	(ALL)	0 Minutes	150 Years	(ALL)		15.0	Sterile Plastic	Micro		0	
499	Environmental Culture	Water	MB Other	Vial	(ALL)	0 Minutes	150 Years	(ALL)		15.0	Sterile Plastic	Micro		0	
500	Eosinophil Urine	Urine	Urinalysis	Urine	(ALL)	0 Minutes	150 Years	(ALL)		5.0	UA	Urinalysis		0	
501	Eosinophil Nasal Smear	Nasal Swab	Special Collectio	Slide Prep	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Slides	Hem		0	
502	Epstein Barr Virus Nuclear Antigen IgG	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		3.0	SST	Ref Room	Room Temperature	0	
503	Epstein-Barr Virus IFA CSF	CSF	Chem Sendout Fluid	Syringe	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Sterile T	Ref Refrig	Refrigerate	0	
504	Erythrocyte Sedimentation Rate	Blood	Blood Hem	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Lav	Hem		0	
505	Erythropoietin	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		3.0	SST	Ref Refrig	Refrigerate	0	
506	Estradiol	Blood	Chem Red Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		5.0	Red	Ref Refrig	Refrigerate	0	
507	Estradiol, Ultra-Sensitive	Blood	Chem Red Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Red	Ref Refrig	Refrigerate	0	
508	Estriol	Blood	Chem Red Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		3.0	Red	Ref Refrig	Refrigerate	0	
509	Estriol	Blood	Chem Red Sendout	Venous Draw	LabCorp	0 Minutes	150 Years	(ALL)		3.0	SST	Ref Refrig		0	
510	Estrogens Total	Blood	Chem Red Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		5.0	Red	Ref Refrig	Refrigerate	0	
511	Estrogens Total	Blood	Chem Red Sendout	Venous Draw	LabCorp	0 Minutes	150 Years	(ALL)		5.0	SST	Ref Refrig		0	
512	Estrone	Blood	Chem Red Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		3.0	Red	Ref Refrig	Refrigerate	0	
513	Ethanol	Blood	Blood	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	SST	Chem		0	
514	Ethosuximide	Blood	Chem Red Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		5.0	Red	Ref Refrig	Refrigerate	0	
515	Ethylene Glycol	Blood	Blood	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		5.0	Grey	Chem	Refrigerate	0	
516	Ethylene Glycol	Blood	Blood	Venous Draw	LabCorp	0 Minutes	150 Years	(ALL)		7.0	Green	Ref Refrig		0	
517	Expedited Maternal HIV Ab/Ag Combo	Blood	HIV Combo	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	SST	Chem		0	
518	Expedited Needle Stick HIV-1	Blood	HIV Combo	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	SST	HIV		0	
519	Expedited Newborn HIV-1 Antibody	Venous	HIV Combo	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		2.0	Red	HIV	Deliver Immediately	0	
520	FK 506 Blood	Blood	Chem FK506	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Lav	SP Chem	Draw prior to next dose	0	
521	Factor II Assay	Blood	Blood Coag	Venous Draw	(ALL)	0 Minutes	12 Years	(ALL)		0.5	Blue (Peds)	Coag	Mix Immediately	0	
522	Factor II Assay	Blood	Blood Coag	Venous Draw	(ALL)	12 Years	150 Years	(ALL)		0.5	Blue	Coag	Mix Immediately	0	
523	Factor II, DNA Analysis	Blood	Blood Hem	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		7.0	Lav	Ref Room		0	
524	Factor IX Assay	Blood	Blood Coag	Venous Draw	(ALL)	0 Minutes	12 Years	(ALL)		0.5	Blue	Coag	Mix Immediately	0	
525	Factor IX Assay	Blood	Blood Coag	Venous Draw	(ALL)	12 Years	150 Years	(ALL)		0.5	Blue	Coag	Mix Immediately	0	
526	Factor V Assay	Blood	Blood Coag	Venous Draw	(ALL)	0 Minutes	12 Years	(ALL)		0.5	Blue (Peds)	Coag	Mix Immediately	0	
527	Factor V Assay	Blood	Blood Coag	Venous Draw	(ALL)	12 Years	150 Years	(ALL)		0.5	Blue	Coag	Mix Immediately	0	
528	Factor V Leiden Mutation	Blood	Hem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		7.0	Lav	Ref Room	Room Temperature	0	
529	Factor VII Assay	Blood	Blood Coag	Venous Draw	(ALL)	0 Minutes	12 Years	(ALL)		0.5	Blue	Coag	Mix Immediately	0	
530	Factor VII Assay	Blood	Blood Coag	Venous Draw	(ALL)	12 Years	150 Years	(ALL)		0.5	Blue	Coag	Mix Immediately	0	
531	Factor VIII Act/Inh W/RFL	Blood	Blood	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		5.0	Blue	Ref Frozen	Freeze	0	
532	Factor VIII Activity	Blood	Hem Blue Sendouts	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		3.0	Blue	Ref Frozen	Freeze	0	
533	Factor VIII Assay	Blood	Blood Coag	Venous Draw	(ALL)	0 Minutes	12 Years	(ALL)		0.5	Blue (Peds)	Coag	Mix Immediately	0	
534	Factor VIII Assay	Blood	Blood Coag	Venous Draw	(ALL)	12 Years	150 Years	(ALL)		0.5	Blue	Coag	Mix Immediately	0	
535	Factor VIII Inhibitor	Blood	Blood Coag	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		4.5	Blue	Ref Frozen	Freeze	0	
536	Factor X Assay	Blood	Blood Coag	Venous Draw	(ALL)	0 Minutes	12 Years	(ALL)		0.5	Blue (Peds)	Coag	Mix Immediately	0	
537	Factor X Assay	Blood	Blood Coag	Venous Draw	(ALL)	12 Years	150 Years	(ALL)		0.5	Blue	Coag	Mix Immediately	0	
538	Factor XI Assay	Blood	Blood Coag	Venous Draw	(ALL)	0 Minutes	12 Years	(ALL)		0.5	Blue (Peds)	Coag	Mix Immediately	0	
539	Factor XI Assay	Blood	Blood Coag	Venous Draw	(ALL)	12 Years	150 Years	(ALL)		0.5	Blue	Coag	Mix Immediately	0	
540	Factor XII Assay	Blood	Blood Coag	Venous Draw	(ALL)	0 Minutes	12 Years	(ALL)		0.5	Blue	Coag	Mix Immediately	0	
541	Factor XII Assay	Blood	Blood Coag	Venous Draw	(ALL)	12 Years	150 Years	(ALL)		0.5	Blue	Coag	Mix Immediately	0	
542	Factor XIII Screen	Blood	Blood Coag	Venous Draw	(ALL)	0 Minutes	12 Years	(ALL)		0.5	Blue	Coag	Mix Immediately	0	
543	Factor XIII Screen	Blood	Blood Coag	Venous Draw	(ALL)	12 Years	150 Years	(ALL)		0.5	Blue	Coag	Mix Immediately	0	
544	Familial Med Fever Mut, NY	Blood	Chem lav sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		3.0	Lav	Ref Room	Room Temperature	0	
545	Fatty Acd PRF,Peroxisomal	Blood	Chem Sendout Frozen	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Red	Ref Frozen	Freeze	0	
546	Fatty Acids Free	Blood	Chem Sendout Frozen	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		3.0	SST	Ref Frozen	Freeze	0	
547	Fecal Electrolytes	Stool	Stool	Stool	(ALL)	0 Minutes	150 Years	(ALL)		5.0	Transport	Ref Refrig	Refrigerate	0	
548	Fecal Fat	Stool	Stool	Stool	(ALL)	0 Minutes	150 Years	(ALL)		1.0	72H Stool	Ref Frozen	Freeze	0	
549	Fecal Fat	Stool	Stool	Stool	LabCorp	0 Minutes	150 Years	(ALL)		1.0	72H Stool	Ref Refrig		0	
550	Fecal Fat Qualitative	Stool	Stool	Stool	(ALL)	0 Minutes	150 Years	(ALL)		5.0	Sterile T	Ref Refrig	Refrigerate	0	
551	Felbamate	Blood	Chem Red Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		5.0	Red	Ref Refrig	Refrigerate	0	
552	Ferritin	Blood	Chem E170	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	SST	SP Chem		0	
553	Fibrinogen	Blood	Blood Coag	Venous Draw	(ALL)	0 Minutes	12 Years	(ALL)		0.5	Blue (Peds)	Coag	Mix Immediately	0	
554	Fibrinogen	Blood	Blood Coag	Venous Draw	(ALL)	12 Years	150 Years	(ALL)		0.5	Blue	Coag	Mix Immediately	0	
555	Fish,Her-2/Neu,Para Block	Biopsy	Chem Sendout	Slide	(ALL)	0 Minutes	150 Years	(ALL)		4.0	Slides	Ref Room	Room Temperature	0	
556	Fluoxetine	Blood	Chem Sendout Room	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		3.0	Red	Ref Room	Room Temperature	0	
557	Fluphenazine	Blood	Blood	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		5.0	Red	Chem	Refrigerate	0	
558	Folates RBC	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		3.0	Lav	Ref Frozen	Freeze	0	
559	Folates RBC	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		3.0	Lav	Ref Refrig	Submit 2 Full Lavender Tubes	0	
560	Folic Acid	Blood	Chem E170	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	SST	SP Chem		0	
561	Follicle Stimulating Hormone	Blood	Chem Architect	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	SST	SP Chem		0	
562	Free Kappa Lambda Light Chains	Serum	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	SST	Ref Refrig	Refrigerate	0	
563	Free Kappa Lambda Random Urine	Urine	Chem Sendout	Urine	(ALL)	0 Minutes	150 Years	(ALL)		2.0	UA	Ref Refrig	Refrigerate	0	
564	Fructosamine	Blood	Chem Red Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Red	Ref Room		0	
565	G6PD Quantitative	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Lav	Ref Refrig	Whole Blood	0	
566	GCT	Serum	GCT	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		0.3	SST	Chem		0	
567	GGT	Blood	Blood	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)		0.3	SST (micro)	Chem		0	
568	GGT	Blood	Blood	Venous Draw	(ALL)	31 Days	150 Years	(ALL)		0.3	SST	Chem		0	

	Primary Synonym	Specimen Type	Accession Class	Collection Method	Service Resource	Age From	Age To	Coll Priority	Aliq	Min Vol	Container	Collection Class	Special Handling	Add Lbl	Opt
569	GLIADIN AB(G,A)/RETICULIN AB (A) wRFX/TRANSGLUTAMINASE AB (A)	Blood	Chem Red Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Red	Ref Refrig	Refrigerate	0	
570	Ganglioside Ab (IGG,IGM)	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	SST	Ref Refrig	Refrigerate	0	
571	Gastrin	Blood	Chem Sendout Frozen	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		4.0	SST	Ref Frozen	Freeze	0	
572	Gene Alter,Postnatal,CGH	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		5.0	Green Sodium	Ref Room	Room Temperature	0	
573	Gentamicin Peak	Serum	Peak	Venous Draw	(ALL)	0 Minutes	31 Months	(ALL)		0.3	SST (micro)	Chem		0	
574	Gentamicin Peak	Serum	Peak	Venous Draw	(ALL)	31 Days	150 Years	(ALL)		0.3	SST	Chem		0	
575	Gentamicin Trough	Serum	Trough	Venous Draw	(ALL)	0 Minutes	31 Days	(ALL)		0.3	SST (micro)	Chem	Draw prior to next dose	0	
576	Gentamicin Trough	Serum	Trough	Venous Draw	(ALL)	31 Days	150 Years	(ALL)		0.3	SST	Chem	Draw prior to next dose	0	
577	Giardia lamblia by EIA	Stool	MB Stool	Stool	(ALL)	0 Minutes	150 Years	(ALL)		2.0	O&P kit	Micro	Room Temperature	0	
578	Glucagon Plasma	Blood	Chem Sendout Frozen	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		5.0	Lav	Ref Frozen	Freeze	0	
579	Glucose 2 Hr PP	Serum	2hr PP	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)		0.2	SST (micro)	Chem		0	
580	Glucose 2 Hr PP	Serum	2hr PP	Venous Draw	(ALL)	1 Months	150 Years	(ALL)		0.3	SST	Chem		0	
581	Glucose CSF	CSF	CSF	Syringe	(ALL)	0 Minutes	150 Years	(ALL)		0.5	Sterile T	Chem	Deliver Immediately	0	
582	Glucose Fasting	Blood	FBS	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)		0.2	SST (micro)	Chem		0	
583	Glucose Fasting	Blood	FBS	Venous Draw	(ALL)	31 Days	150 Years	(ALL)		1.0	Grey	Chem		0	
584	Glucose Random	Blood	Blood	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)		0.2	SST (micro)	Chem		0	
585	Glucose Random	Blood	Blood	Venous Draw	(ALL)	1 Months	150 Years	(ALL)		0.3	SST	Chem		0	
586	Glucose TPN Pharmacy	TPN	Special Collectio	Pharmacy	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Red	Chem		0	
587	Glucose Urine Random	Urine	Urine	Urine	(ALL)	0 Minutes	150 Years	(ALL)		2.0	UA	Urine Chem		0	
588	Glucose #1	Blood	Blood	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)		0.2	SST (micro)	Chem		0	
589	Glucose #1	Blood	Blood	Venous Draw	(ALL)	31 Days	150 Years	(ALL)		0.3	SST	Chem		0	
590	Glucose #2	Blood	Blood	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)		0.2	SST (micro)	Chem		0	
591	Glucose #2	Blood	Blood	Venous Draw	(ALL)	31 Days	150 Years	(ALL)		0.3	SST	Chem		0	
592	Glucose #3	Blood	Blood	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)		0.2	SST (micro)	Chem		0	
593	Glucose #3	Blood	Blood	Venous Draw	(ALL)	31 Days	150 Years	(ALL)		0.3	SST	Chem		0	
594	Glucose #4	Blood	Blood	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)		0.2	SST (micro)	Chem		0	
595	Glucose #4	Blood	Blood	Venous Draw	(ALL)	31 Days	150 Years	(ALL)		0.3	SST	Chem		0	
596	Glucose #5	Blood	Blood	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)		0.2	SST (micro)	Chem		0	
597	Glucose #5	Blood	Blood	Venous Draw	(ALL)	31 Days	150 Years	(ALL)		0.3	SST	Chem		0	
598	Glucose #6	Blood	Blood	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)		0.2	SST (micro)	Chem		0	
599	Glucose #6	Blood	Blood	Venous Draw	(ALL)	31 Days	150 Years	(ALL)		0.3	SST	Chem		0	
600	Glucose Load	Blood	Blood	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Grey	Chem		0	
601	Glucose Peritoneal Dialysate	DIALY	Body Fluid	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)		0.5	Red	Chem		0	
602	Glucose Peritoneal Fluid	Peritoneal Fluid	Body Fluid	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)		0.3	Red	Chem		0	
603	Glucose Pleural Fluid	Pleural Fluid	Body Fluid	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)		0.3	Red	Chem		0	
604	Glucose Synovial Fluid	Synovial fluid	Body Fluid	Syringe	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Red	Chem		0	
605	Glucose Urine 24Hr	Urine	Urine 24hr	Urine	(ALL)	0 Minutes	150 Years	(ALL)		2.0	U24H	Urine Chem	Refrigerate	0	
606	Glucose, Serum, 1.0Hour (1.0Hr)	Blood	Blood	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Grey	Chem		0	
607	Glucose, Serum, 2.0Hour (GLUS 2.0Hr)	Blood	Blood	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Grey	Chem		0	
608	Glucose, Serum, 3.0Hour (GLUS 3.0Hr)	Blood	Blood	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		0.1	Grey	Chem		0	
609	Glucose, Serum, 4.0Hour (GLUS 4.0Hr)	Blood	Blood	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		0.1	Grey	Chem		0	
610	Glucose, Serum, 5.0Hour (GLUS 5.0Hr)	Blood	Blood	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		0.1	Grey	Chem		0	
611	Glucose, Serum, 6.0Hour (GLUS 6.0Hr)	Blood	Blood	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Grey	Chem		0	
612	Glucose, Serum, Fasting (GLUS 0Hr)	Blood	Blood	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Grey	Chem		0	
613	Glutamic Acid Decarboxylase Autoantibody	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	SST	Ref Refrig		0	
614	Glycomark	Blood	Chem Red Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Red	Ref Room	Room Temperature	0	
615	Growth Hormone	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		3.0	SST	Ref Refrig	Refrigerate	0	
616	H. Influenzae Type B (IgG)	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	SST	Ref Refrig	Refrigerate	0	
617	H. Pylori Ab (IgM)	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	SST	Ref Refrig	Refrigerate	0	
618	H. Pylori Ab, IGA, Serum	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	SST	Ref Refrig	Refrigerate	0	
619	H. Pylori Ag Det,EIA,Stool	Stool	Stool	Stool	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Sterile Plastic	Ref Frozen	Freeze	0	
620	H.PYLORI AB (IGG)	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	SST	Ref Refrig	Refrigerate	0	
621	HBV Genotype	Blood	Chem Sendout Frozen	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Lav	Ref Frozen	Freeze	0	
622	HBV,DNA,QN,PCR W/REFL	Blood	Chem Sendout Frozen	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		2.0	SST	Ref Frozen	Freeze	0	
623	HCV RNA Genotype, Lipa	Blood	Chem Sendout Frozen	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		3.0	Lav	Ref Frozen	Freeze	0	
624	HCV Viral Load RT-PCR	Blood	GV HCV Viral Load	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	SST	MolecularVL		0	
625	HDL Cholesterol	Blood	Blood	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)		0.3	SST (micro)	Chem		0	
626	HDL Cholesterol	Blood	Blood	Venous Draw	(ALL)	31 Days	150 Years	(ALL)		0.3	SST	Chem		0	
627	HIV 1 RNA Viral Load	Blood	VL	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		5.0	PPT	Virol	Refrigerate	0	
628	HIV 1Antibody Screen <24mos	Blood	HIV Combo	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		6.0	SST	HIV	Room Temperature	0	
629	HIV Ag/Ab Combo	Blood	HIV Combo	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	SST	Chem		0	
630	HIV GenoSure(TM) MG Genotyping	Blood	Chem Sendout Frozen	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		5.0	Lav	Ref Frozen	Freeze	0	
631	HIV Retroviral Genotype	Blood	Chem Sendout Frozen	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		3.0	Lav	Ref Frozen	Freeze	0	
632	HIV-1 Virtpeno W/INT (NY)	Blood	Chem Sendout Frozen	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Lav	Ref Frozen	Freeze	0	
633	HIV-2 AB,EIA,W/IB RFL(NY)	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		2.0	SST	Ref Refrig	Refrigerate	0	
634	HLA-B27 Antigen	Blood ACD	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		6.0	Yellow (ACD)	Ref Room	Room Temperature	0	
635	HLA-B27 Antigen	Blood ACD	Chem Sendout	Venous Draw	LabCorp	0 Minutes	150 Years	(ALL)		4.0	Lav	Ref Room		0	
636	HLA-B5701 Typing	Blood	Chem lav sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		7.0	Lav	Ref Room		0	
637	HPV 16_18	Thin Prep	Genetics HPV	Thin Prep	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Thin Prep	MolecularC		0	
638	HPV HR Screen	Thin Prep	Genetics HPV	Thin Prep	(ALL)	0 Minutes	150 Years	(ALL)		5.0	Thin Prep	MolecularC		0	
639	HPV,Low/High Risk,DNA,ISH	Tissue	Chem Sendout	Special Collection	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Contact Laboratory	Ref Room	Room Temperature	0	

	Primary Synonym	Specimen Type	Accession Class	Collection Method	Service Resource	Age From	Age To	Coll Priority	Aliq	Min Vol	Container	Collection Class	Special Handling	Add Lbl	Opt
640	HSV 1 & 2 IgG Herpeselect Ab	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		5.0	SST	Ref Refrig	Refrigerate	0	
641	HTLV I & II Antibody	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		2.0	Red	Ref Refrig	Refrigerate	0	
642	Haptoglobin	Blood	Blood	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		0.5	SST	Chem		0	
643	Heinz Body Stain	Blood	Blood Hem	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)		0.2	Lav (micro)	Hem		0	
644	Heinz Body Stain	Blood	Blood Hem	Venous Draw	(ALL)	1 Months	150 Years	(ALL)		0.5	Lav	Hem		0	
645	Helicobacter Pylori Urease Test	Biopsy	Special Collectio	Biopsy	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Contact Laboratory	MISC	Obtain Collection Kit from Micro	0	
646	Hem Consult Smear Review	Blood	Blood Hem	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		0.1	Lav	Hem		0	
647	Hematocrit	Blood	Blood Hem	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)		0.2	Lav (micro)	Hem		0	
648	Hematocrit	Blood	Blood Hem	Venous Draw	(ALL)	1 Months	150 Years	(ALL)		0.5	Lav	Hem		0	
649	Hematocrit Arterial Blood	Blood Heparin	Blood Gases Arterial	Syringe	(ALL)	0 Minutes	150 Years	(ALL)		0.1	Hep Syringe	Chem	Deliver Immediately	0	
650	Hematocrit Cell Saver	Blood	Blood Hem	Cell Saver	(ALL)	0 Minutes	150 Years	(ALL)		0.5	Lav	Hem	Deliver Immediately	0	
651	Hematocrit Fluid	Synovial fluid	Body Fluid	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)		0.5	Lav	Hem		0	
652	Hematocrit Fluid	Body Fluid	Body Fluid	Syringe	(ALL)	0 Minutes	150 Years	(ALL)		0.5	Lav	Hem	Deliver Immediately	0	
653	Hematocrit Fluid	Pleural Fluid	Body Fluid	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)		0.5	Lav	Hem		0	
654	Hematocrit Fluid	Pericardial Fluid	Body Fluid	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)		0.5	Lav	Hem		0	
655	Hematocrit Fluid	Abcess Drainage	Body Fluid	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)		0.5	Lav	Hem		0	
656	Hematocrit Fluid	Dialysis Fluid	Body Fluid	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)		0.5	Lav	Hem		0	
657	Hematocrit Fluid	J-Tube Drainage	Body Fluid	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)		0.5	Lav	Hem		0	
658	Hematocrit Fluid	Peritoneal Fluid	Body Fluid	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)		0.5	Lav	Hem		0	
659	Hematocrit Abscess Drainage	Abcess Drainage	Body Fluid	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)		0.2	Lav	Hem		0	
660	Hematocrit CSF	CSF	CSF	Syringe	(ALL)	0 Minutes	150 Years	(ALL)		0.2	Sterile Plastic	Hem		0	
661	Hematocrit Dialysate Fluid	Dialysis Fluid	Body Fluid	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)		0.2	Lav	Hem		0	
662	Hematocrit J Tube Drainage	J-Tube Drainage	Body Fluid	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)		0.2	Lav	Hem		0	
663	Hematocrit Pericardial Fluid	Pericardial Fluid	Body Fluid	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)		0.2	Lav	Hem		0	
664	Hematocrit Peritoneal Fluid	Peritoneal Fluid	Blood Gases Venous	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)		0.2	Lav	Hem		0	
665	Hematocrit Pleural Fluid	Pleural Fluid	Body Fluid	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)		0.2	Lav	Hem		0	
666	Hematocrit Synovial Fluid	Synovial fluid	Body Fluid	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)		0.2	Lav	Hem		0	
667	Hemoglobin A1c	Blood	CHEM LAV	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Lav	Chem		0	
668	Hemosiderin Urine	Urine	Urine	Urine	(ALL)	0 Minutes	150 Years	(ALL)		5.0	UA	Urinalysis		0	
669	Heparin LMW Anti-Xa Assay	Blood	Blood Coag	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		0.5	Blue	Coag	Mix Immediately	0	
670	Heparin, Unfractionated	Blood	Blood Coag	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Blue	Ref Frozen	Freeze	0	
671	Hepatic Function Panel	Blood	Blood	Venous Draw	(ALL)	0 Minutes	31 Days	(ALL)		0.2	SST (micro)	Chem		0	
672	Hepatic Function Panel	Blood	Blood	Venous Draw	(ALL)	31 Days	150 Years	(ALL)		0.5	SST	Chem		0	
673	Hepatitis A Ab Total	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		3.0	SST	Ref Refrig	Refrigerate	0	
674	Hepatitis A Antibody IgM	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		2.0	SST	Ref Refrig	Refrigerate	0	
675	Hepatitis B Core IgM	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		2.0	SST	Ref Refrig	Refrigerate	0	
676	Hepatitis B Core Antibody Total	Blood	Chem Hep	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	SST	Chem	Refrigerate	0	
677	Hepatitis B Surface Antibody	Blood	Chem E170	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	SST	SP Chem		0	
678	Hepatitis B Surface Antigen	Blood	Chem E170	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	SST	SP Chem		0	
679	Hepatitis Be Antibody	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		4.0	SST	Ref Refrig	Refrigerate	0	
680	Hepatitis Be Antigen	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		4.0	SST	Ref Refrig	Refrigerate	0	
681	Hepatitis C Antibody	Blood	Chem Hep	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		2.0	SST	Chem	Refrigerate	0	
682	Hepatitis C Confirming by RIBA	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		4.0	SST	Ref Refrig	Refrigerate	0	
683	Hepatitis D IgM Ab	Serum	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		4.0	SST	Ref Refrig	Refrigerate	0	
684	Hepatitis Delta Antibody	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		4.0	SST	Ref Room	Room Temperature	0	
685	Hepatitis E Ab (IGG)	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		2.0	SST	Ref Frozen	Freeze	0	
686	Hepatitis E Ab (IGM)	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		2.0	SST	Ref Frozen	Freeze	0	
687	Hereditary Hemochromat,NY	Blood	Chem lav sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		3.0	Lav	Ref Room	Room Temperature	0	
688	Herpes Simplex IgM I/II Combo	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		5.0	SST	Ref Refrig		0	
689	Herpes Type 1&2 DNA, PCR	Blood	Chem Sendout Frozen	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		2.0	SST	Ref Frozen	Freeze	0	
690	Herpes Type 1&2 DNA, PCR	Body Fluid	Chem Sendout Frozen	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Sterile T	Ref Frozen	Freeze	0	
691	Herpes Type 1&2 DNA, PCR	Swab	Chem Sendout	Swab	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Contact Laboratory	Ref Refrig	Refrigerate	0	
692	Herpes Type 1&2 DNA, PCR	CSF	Chem Sendout Fluid	Syringe	LabCorp	0 Minutes	150 Years	(ALL)		0.5	Sterile T	Ref Refrig	Refrigerate	0	
693	Herpesvirus 6 (IGG,IGM)	Blood	Chem Red Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		2.0	SST	Ref Room	Room Temperature	0	
694	Herpesvirus 8 IGG, AB, IFA	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		2.0	SST	Ref Refrig	Refrigerate	0	
695	Hexoseamidase A	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		4.0	SST	Ref Frozen	Freeze	0	
696	Hgb A2 (Lab Order Only)	Blood	Blood Hem	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)		0.2	Lav (micro)	Hem		0	
697	Hgb A2 (Lab Order Only)	Blood	Blood Hem	Venous Draw	(ALL)	1 Months	150 Years	(ALL)		1.0	Lav	Hem		0	
698	Hgb Electrophoresis Acid (Lab Order Only)	Blood	Blood Hem	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		0.5	Lav	Hem		0	
699	Hgb Electrophoresis Alkaline (Lab Order Only)	Blood	Blood Hem	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)		0.2	Lav (micro)	Hem		0	
700	Hgb Electrophoresis Alkaline (Lab Order Only)	Blood	Blood Hem	Venous Draw	(ALL)	1 Months	150 Years	(ALL)		1.0	Lav	Hem		0	
701	Hgb Electrophoresis HPLC	Blood	Blood Hem	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)		0.2	Lav (micro)	Hem		0	
702	Hgb Electrophoresis HPLC	Blood	Blood Hem	Venous Draw	(ALL)	1 Months	150 Years	(ALL)		1.0	Lav	Hem		0	
703	Hgb F (Lab Order Only)	Blood	Blood Hem	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)		0.2	Lav (micro)	Hem		0	
704	Hgb F (Lab Order Only)	Blood	Blood Hem	Venous Draw	(ALL)	1 Months	150 Years	(ALL)		1.0	Lav	Hem		0	
705	Histamine Plasma	Blood	Chem Sendout Frozen	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		2.0	Lav	Ref Frozen	Freeze	0	
706	Histamine Release (CU)	Blood	Chem Red Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		2.0	Red	Ref Room	Room Temperature	0	
707	Histoplasma AB, ID	Blood	Chem Red Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		2.0	Red	Ref Refrig	Refrigerate	0	
708	Histoplasma AB, ID	Blood	Chem Red Sendout	Venous Draw	LabCorp	0 Minutes	150 Years	(ALL)		3.0	SST	Ref Refrig		0	
709	Histoplasma Ab Panel	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Red	Ref Refrig	Refrigerate	0	
710	Histoplasma Ag	Blood	Chem Red Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		5.0	Red	Ref Refrig	Refrigerate	0	

	Primary Synonym	Specimen Type	Accession Class	Collection Method	Service Resource	Age From	Age To	Coll Priority	Aliq	Min Vol	Container	Collection Class	Special Handling	Add Lbl	Opt
711	Histoplasma Ag	Blood	Chem Red Sendout	Venous Draw	LabCorp	0 Minutes	150 Years	(ALL)		5.0	SST	Ref Frozen	Freeze	0	
712	Histoplasma Ag, Urine	Urine	Chem Sendout Frozen	Urine	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Midstream	Ref Frozen		0	
713	Histoplasma Antigen, Serum	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		2.0	SST	Ref Frozen		0	
714	Homocysteine	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		4.0	SST	Ref Refrig	Refrigerate	0	
715	Homovanillic Acid Urine 24Hr	Urine	Urine	Urine	(ALL)	0 Minutes	150 Years	(ALL)		30.0	U24H HCL	Ref Refrig	Refrigerate	0	
716	Homovanillic Acid Urine Random	Urine	Urine	Urine	(ALL)	0 Minutes	150 Years	(ALL)		10.0	U Aliquot	Ref Refrig	Refrigerate	0	
717	Human Granulocytic Ehrlich-HGE	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		2.0	SST	Ref Refrig		0	
718	Huntington Disease, NY	Blood	Chem lav sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		3.0	Lav	Ref Room	Room Temperature	0	
719	Hydroxyproline, Free, Rand	Urine	Urine	Urine	(ALL)	0 Minutes	150 Years	(ALL)		20.0	UA	Ref Refrig	Refrigerate	0	
720	IAVI Repo Exposure Panel	Blood	VS	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Red	Serology	Freeze	0	
721	IGF 1	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		2.0	Red	Ref Refrig	Refrigerate	0	
722	IGF Binding Protein-1	Blood	Chem Sendout Frozen	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		2.0	Red	Ref Frozen	Freeze	0	
723	IGF-II	Blood	Chem Red Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Red	Ref Frozen	Freeze	0	
724	IQ	Urine	Urinalysis	Urine	(ALL)	0 Minutes	150 Years	(ALL)		1.0	UA	Urinalysis		0	
725	ISTAT ABG Art	Blood	Blood Gases Arterial	Arterial Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	ISTAT	Chem		0	
726	ISTAT ABG Art	Arterial	Blood Gases Arterial	Arterial Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	ISTAT	Chem		0	
727	ISTAT ABG Art	Capillary	Blood Gases Arterial	Capillary	(ALL)	0 Minutes	150 Years	(ALL)		1.0	ISTAT	Chem		0	
728	ISTAT ABG Ven	Venous	Blood Gases Venous	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	ISTAT	Chem		0	
729	ISTAT ACT	Blood	Blood Heparin	Arterial Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	ISTAT	Chem		0	
730	ISTAT Arterial	Blood	Blood Gases Arterial	Arterial Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Hep Syringe	Chem		0	
731	ISTAT CG8	Blood	Blood Gases Arterial	Arterial Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	ISTAT	Chem		0	
732	ISTAT Chem8	Arterial	Blood	Arterial Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	ISTAT	Chem		0	
733	ISTAT Chem8	Capillary	Blood	Capillary	(ALL)	0 Minutes	150 Years	(ALL)		1.0	ISTAT	Chem		0	
734	ISTAT Chem8	Venous	Blood	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	ISTAT	Chem		0	
735	ISTAT Chem8	OTHER	Blood	Fingerstick	(ALL)	0 Minutes	150 Years	(ALL)		1.0	ISTAT	Chem		0	
736	ISTAT Glucose	Blood	POC BLD	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	ISTAT	Chem		0	
737	ISTAT IONIZED CA	Blood	Blood	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	ISTAT	Chem		0	
738	ISTAT Lytes	Blood	Blood Gases Arterial	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	ISTAT	Chem		0	
739	ISTAT PT	Arterial	Blood Coag	Arterial Draw	(ALL)	0 Minutes	150 Years	(ALL)		2.0	Blue	Coag		0	
740	ISTAT PT	Capillary	Blood Coag	Capillary	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Blue	Coag		0	
741	ISTAT PT	Venous	Blood Coag	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Blue	Coag		0	
742	ISTAT Venous	Blood	Blood Gases Venous	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	ISTAT	Chem		0	
743	IgG Subclasses 1 to 4	Blood	Chem Red Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Red	Ref Refrig	Refrigerate	0	
744	Imipramine+Desipramine	Blood	Chem Red Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		3.0	Red	Ref Refrig	Refrigerate	0	
745	Immune Complex, C1Q	Blood	Chem Sendout Frozen	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Red	Ref Frozen	Freeze	0	
746	Immunoglobulin A Serum	Blood	Blood	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		0.5	SST	Chem		0	
747	Immunoglobulin E Serum	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	SST	Immune		0	
748	Immunoglobulin G	Blood	Blood	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		0.5	SST	Chem		0	
749	Immunoglobulin G CSF	CSF	CSF	Special Collection	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Sterile Plastic	Ref Refrig		0	
750	Immunoglobulin M	Blood	Blood	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		0.5	SST	Chem		0	
751	Immunoglobulin M Pediatric	Blood	Blood	Venous Draw	(ALL)	0 Minutes	5 Years	(ALL)		0.5	SST	Chem		0	
752	Immunoglobulin M Pediatric	Blood	Blood	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		0.5	SST	Chem		0	
753	Immunoglobulins Quantative	Blood	Blood	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		0.3	SST	Immune		0	
754	Infectious Mononucleosis Serum	Blood	VS	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)		0.3	SST (micro)	Serology		0	
755	Infectious Mononucleosis Serum	Blood	VS	Venous Draw	(ALL)	31 Days	150 Years	(ALL)		0.3	SST	Serology		0	
756	Inhibin A	Blood	Chem Sendout Frozen	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Red	Ref Frozen	Freeze	0	
757	Inhibin A	Blood	Chem Sendout Frozen	Venous Draw	LabCorp	0 Minutes	150 Years	(ALL)		1.0	SST	Ref Frozen		0	
758	Inhibin B	Blood	Chem Sendout Frozen	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Red	Ref Frozen	Freeze	0	
759	Inhibin B	Blood	Chem Sendout Frozen	Venous Draw	LabCorp	0 Minutes	150 Years	(ALL)		2.0	SST	Ref Frozen		0	
760	Insulin	Blood	Chem E170	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		2.0	SST	Chem	Freeze	0	
761	Insulin #1	Serum	Blood	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	SST	Chem	Freeze	0	
762	Insulin #2	Serum	Blood	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	SST	Chem	Freeze	0	
763	Insulin #3	Serum	Blood	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	SST	Chem	Freeze	0	
764	Insulin #4	Serum	Blood	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	SST	Chem	Freeze	0	
765	Insulin #5	Serum	Blood	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	SST	Chem	Freeze	0	
766	Insulin #6	Serum	Blood	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	SST	Chem	Freeze	0	
767	Insulin Autoantibody	Blood	Chem Sendout Frozen	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		5.0	Red	Ref Frozen	Freeze	0	
768	Insulin Like Growth Factor Binding Protein 3	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		3.0	SST	Ref Refrig	Refrigerate	0	
769	Intrinsic Factor Blocking Antibodies	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		2.0	Red	Ref Room	Room Temperature	0	
770	Intrinsic Factor Blocking Antibodies	Blood	Chem Sendout	Venous Draw	LabCorp	0 Minutes	150 Years	(ALL)		4.0	SST	Ref Refrig		0	
771	Iron	Blood	Blood	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)		0.3	SST (micro)	Chem		0	
772	Iron	Blood	Blood	Venous Draw	(ALL)	1 Months	150 Years	(ALL)		1.0	SST	Chem		0	
773	Iron Stain Bone Marrow	Bone Marrow Asp	Blood Hem	Slide	(ALL)	0 Minutes	150 Years	(ALL)		2.0	Slides	Hem	Deliver Immediately	0	
774	Iron Tissue	Tissue	Special Collectio	Tissue	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Biopsy Bottle	Chem		0	
775	Iron and TIBC	Blood	Blood	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)		1.0	SST (micro)	Chem		0	
776	Iron and TIBC	Blood	Blood	Venous Draw	(ALL)	1 Months	150 Years	(ALL)		3.0	SST	Chem		0	
777	Islet Cell Cytoplasmic Antibody	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		4.0	SST	Ref Refrig	Refrigerate	0	
778	Isopropanol Stability Test	Blood	Blood Hem	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Lav	Hem		0	
779	JAK2 V617F mutation detection	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		3.0	Lav	Ref Room	Room Temperature	0	
780	JC VIRUS DNA, PCR (CSF)	CSF	Chem Sendout Room	Special Collection	(ALL)	0 Minutes	150 Years	(ALL)		0.2	Sterile T	Ref Room		0	
781	JC Virus DNA,PCR (Whole Blood)	Blood	Chem lav sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		5.0	Lav	Ref Refrig	Whole Blood	0	

	Primary Synonym	Specimen Type	Accession Class	Collection Method	Service Resource	Age From	Age To	Coll Priority	Aliq	Min Vol	Container	Collection Class	Special Handling	Add Lbl	Opt
782	KOH Prep	Vaginal Exudates	Julian Date	Wash	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Sterile Plastic	Urinalysis		0	
783	KRAS Mutation Analysis	Blood	Chem lav sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		7.0	Lav	Ref Room		0	
784	KRAS Mutation Analysis	Paraffin Block	Sendout Molecular	Tissue	(ALL)	0 Minutes	150 Years	(ALL)		1.0	No Container	Ref Room		0	
785	Kappa Lambda Light Chain	Serum	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	SST	Ref Refrig	Refrigerate	0	
786	Kappa Lambda Total Random Urine	Urine	Chem Sendout	Urine	(ALL)	0 Minutes	150 Years	(ALL)		2.0	UA	Ref Refrig	Refrigerate	0	
787	Kleinhauer Betke Stain	Blood	Blood Hem	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)		0.2	Lav (micro)	Hem		0	
788	Kleinhauer Betke Stain	Blood	Blood Hem	Venous Draw	(ALL)	1 Months	150 Years	(ALL)		0.5	Lav	Hem		0	
789	L. pneumophila Serogp 1 Ur Ag	Urine	Urine	Urine	(ALL)	0 Minutes	150 Years	(ALL)		10.0	Sterile Plastic	Ref Refrig		0	
790	U/S Ratio PG Amniotic Fluid	Amniotic Fluid	Body Fluid	Special Collection	(ALL)	0 Minutes	150 Years	(ALL)		3.0	Amber	Chem	Protect from light	0	
791	LAP	Blood	Chem Sendout Room	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		2.0	Green Sodium	Ref Room	Make Slides from Heparin Tube	0	
792	LD Isoenzymes	Blood	Chem Sendout Room	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		6.0	SST	Ref Room	Room Temperature	0	
793	LD Pericardial Fluid	Pericardial Fluid	Body Fluid	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)		0.3	Red	Chem		0	
794	LD Pleural Fluid	Pleural Fluid	Body Fluid	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)		0.3	Red	Chem		0	
795	LD Renal Perfusate	Perfusate	Body Fluid	Syringe	(ALL)	0 Minutes	150 Years	(ALL)		0.5	Hep Syringe	Chem		0	
796	LE Latex	Blood	Hem Red	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	SST	Hem		0	
797	Lab Miscellaneous Test	OTHER	Julian Date	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Contact Laboratory	Chem		0	
798	Lab Order IFE	Blood	Blood	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	SST	Chem		0	
799	Lab Order IFE Urine	Urine	Urine	Urine	Electrophoresis	0 Minutes	150 Years	(ALL)		5.0	U Aliquot	Chem		0	
800	Lactate Dehydrogenase	Blood	Blood	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)		0.3	SST (micro)	Chem		0	
801	Lactate Dehydrogenase	Blood	Blood	Venous Draw	(ALL)	31 Days	150 Years	(ALL)		0.3	SST	Chem		0	
802	Lactic Acid	Blood	Blood Lactate	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		0.8	Grey	Chem	Deliver Immediately	0	
803	Lactic Acid (Blood Gas)	Blood Heparin	Blood Gases Venous	Syringe	(ALL)	0 Minutes	150 Years	(ALL)		0.1	Hep Syringe	Chem	Deliver Immediately	0	
804	Lamotrigine	Blood	Chem Sendout Room	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Red	Ref Room	Room Temperature	0	
805	Lead	Blood	CHEM LAV	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		2.0	Tan	Chem		0	
806	Lead Urine	Urine	Urine	Urine	(ALL)	0 Minutes	150 Years	(ALL)		10.0	U24H	Ref Refrig	Refrigerate	0	
807	Leflunomide as Metabolite	Blood	Chem Sendout Frozen	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		2.0	Red	Ref Frozen	Freeze	0	
808	Leptin	Blood	Chem Red Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Red	Ref Refrig	Refrigerate	0	
809	Leptospira IgM	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		3.0	SST	Ref Refrig		0	
810	Leukemia/Lymph, Select Abs	CSF	Chem Sendout	Syringe	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Yellow (ACD)	Ref Room	Room Temperature	0	
811	Leukemia/Lymph, Select Abs	Blood	Chem lav sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		3.0	Lav	Ref Room	Room Temperature	0	
812	Leukemia/Lymph, Select Abs	Blood	Chem lav sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		3.0	Green Sodium	Ref Room	Room Temperature	0	
813	Leukemia/Lymph, Select Abs	Blood	Chem lav sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		2.0	Slides	Ref Room	Room Temperature	0	
814	Leukemia/Lymph, Select Abs	Bone Marrow	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		3.0	Green Sodium	Ref Room	Room Temperature	0	
815	Leukemia/Lymph, Select Abs	Bone Marrow	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		2.0	Slides	Ref Room	Room Temperature	0	
816	Leukemia/Lymph, Select Abs	Lymph node	Chem Sendout	Biopsy	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Sterile Screw	Ref Room	Room Temperature	0	
817	Leukemia/Lymphoma Evaluation	Blood	Blood	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Lav	Chem		0	
818	Leukemia/Lymphoma Evaluation	Blood	Blood	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		3.0	Green Sodium	Chem		0	
819	Leukemia/Lymphoma Evaluation	Blood	Blood	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		2.0	Slides	Chem		0	
820	Leukemia/Lymphoma Evaluation	Bone Marrow	Special Collectio	Slide	(ALL)	0 Minutes	150 Years	(ALL)		2.0	Slides	Chem		0	
821	Leukemia/Lymphoma Evaluation	Bone Marrow	Special Collectio	Slide	(ALL)	0 Minutes	150 Years	(ALL)		3.0	Green Sodium	Chem		0	
822	Leukemia/Lymphoma Evaluation	Biopsy	Chem Sendout	Biopsy	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Sterile Plastic	Chem	Room Temperature	0	
823	Leukocytes Stool	Stool	Special Collection	Slide Prep	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Slides	Urinalysis		0	
824	Levetiracetam (Keppra)	Blood	Chem Sendout Room	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Lav	Ref Room		0	
825	Lidocaine	Blood	Chem Red Sendout	Venous Draw	(ALL)	0 Minutes	1 Years	(ALL)		1.0	Red (micro)	Ref Refrig	Refrigerate	0	
826	Lidocaine	Blood	Chem Red Sendout	Venous Draw	(ALL)	2 Minutes	150 Years	(ALL)		4.0	Red	Ref Refrig	Refrigerate	0	
827	Lipase	Blood	Blood	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)		0.3	SST (micro)	Chem		0	
828	Lipase	Blood	Blood	Venous Draw	(ALL)	31 Days	150 Years	(ALL)		0.3	SST	Chem		0	
829	Lipase, Peritoneal Fluid	Peritoneal Fluid	Chem Sendout Fluid	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)		2.0	Sterile Plastic	Ref Frozen	Freeze	0	
830	Lipid Profile	Blood	Blood	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)		0.5	SST (micro)	Chem		0	
831	Lipid Profile	Blood	Blood	Venous Draw	(ALL)	31 Days	150 Years	(ALL)		1.0	SST	Chem		0	
832	Lipoprotein (A)	Blood	Chem Red Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Red	Ref Refrig	Refrigerate	0	
833	Lithium	Blood	Blood	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)		0.3	SST (micro)	Chem	Draw prior to next dose	0	
834	Lithium	Blood	Blood	Venous Draw	(ALL)	31 Days	150 Years	(ALL)		0.3	SST	Chem	Draw prior to next dose	0	
835	Liver Kidney Microsomal Antibody	Blood	Chem Sendout	Venous Draw	LabCorp	0 Minutes	150 Years	(ALL)		3.0	SST	Ref Refrig		0	
836	Lupus Anticoagulant	Blood	Hem Blue Sendouts	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		3.0	Blue	Ref Frozen	Freeze	0	
837	Luteinizing Hormone	Blood	Chem Architect	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	SST	SP Chem		0	
838	Lyme Disease, PCR, CSF/SYN	CSF	CSF	Syringe	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Sterile T	Ref Refrig		0	
839	Lyme Disease, PCR, CSF/SYN	Synovial fluid	Body Fluid	Syringe	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Sterile Screw	Ref Refrig		0	
840	Lyme IgG/IgM Ab	Blood	Chem Sendout Room	Venous Draw	LabCorp	0 Minutes	150 Years	(ALL)		1.0	SST	Ref Room	Room Temperature	0	
841	Lymphocyte Prolif, Mitogen	Blood	Chem Sendout Room	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		5.0	Green Sodium	Ref Room	Room Temperature	0	
842	Lysozyme	Blood	Chem Sendout Frozen	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		3.0	SST	Ref Frozen	Freeze	0	
843	M. Pneumoniae Antibody	Blood	Chem Sendout Room	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		4.0	SST	Ref Room	Room Temperature	0	
844	M. Tuberculos, PCR, Non-Resp	CSF	CSF	Syringe	(ALL)	0 Minutes	150 Years	(ALL)		3.0	Sterile Screw	Ref Refrig	Refrigerate	0	
845	M. Tuberculos, PCR, Non-Resp	Urine	Urine	Urine	(ALL)	0 Minutes	150 Years	(ALL)		5.0	UA	Ref Refrig	Refrigerate	0	
846	M. Tuberculos, PCR, Non-Resp	Body Fluid	Body Fluid	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)		3.0	Sterile Screw	Ref Refrig		0	
847	MAG, Western Blot W/RFX	Blood	Chem Red Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Red	Ref Refrig	Refrigerate	0	
848	MEN2/FMTC, EX10,11,13-16NY	Blood	Chem lav sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		2.0	Lav	Ref Room	Room Temperature	0	
849	MEN2/FMTC, EX10,11,13-16NY	Amniotic Fluid	Chem Sendout	Special Collection	(ALL)	0 Minutes	150 Years	(ALL)		2.0	Sterile Plastic	Ref Room	Room Temperature	0	
850	MSKCC	Blood	Blood	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		20.0	SST	Chem		0	
851	MSKCC	Blood	Blood	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		20.0	Green	Chem		0	
852	MSKCC	Blood	Blood	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		3.0	Lav	Chem		0	

	Primary Synonym	Specimen Type	Accession Class	Collection Method	Service Resource	Age From	Age To	Coll Priority	Aliq	Min Vol	Container	Collection Class	Special Handling	Add Lbl	Opt
853	Magnesium	Blood	Blood	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)		0.3	SST (micro)	Chem		0	
854	Magnesium	Blood	Blood	Venous Draw	(ALL)	31 Days	150 Years	(ALL)		0.3	SST	Chem		0	
855	Magnesium Urine 24Hr	Urine	Urine 24hr	Urine	(ALL)	0 Minutes	150 Years	(ALL)		2.0	U24H	Urine Chem	Refrigerate	0	
856	Magnesium Urine Random	Urine	Urine	Urine	(ALL)	0 Minutes	150 Years	(ALL)		10.0	UA	Urine Chem	Refrigerate	0	
857	Malarial Stain	Blood	Blood Hem	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)		0.2	Lav (micro)	Hem		0	
858	Malarial Stain	Blood	Blood Hem	Venous Draw	(ALL)	1 Months	150 Years	(ALL)		0.5	Lav	Hem		0	
859	Mannose-Binding Lectin	Blood	Chem Sendout Frozen	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		2.0	Red	Ref Frozen	Freeze	0	
860	Maternal Serum Quad Screen	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		5.0	SST	Ref Refrig	Refrigerate	0	
861	Measles Antibody Immune Status IgG	Blood	VS	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	SST	Serology		0	
862	Mercaptopurine	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		2.0	Red	Ref Refrig	Refrigerate	0	
863	Mercury Blood	Blood	Chem Royal	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		4.0	Royal (EDTA)	Ref Refrig	Refrigerate	0	
864	Mercury Blood	Blood	Chem Royal	Venous Draw	LabCorp	0 Minutes	150 Years	(ALL)		7.0	Royal (EDTA)	Ref Room		0	
865	Mercury Urine	Urine	Urine	Urine	(ALL)	0 Minutes	150 Years	(ALL)		20.0	U Aliquot	Ref Refrig	Refrigerate	0	
866	Metabolic Screen Neonatal	Capillary - heelstick	Special Collectio	Capillary	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Filter Paper	SP Chem	Fill Holes Completely	0	
867	Metanepharines Urine 24Hr Fractionated Quantitative	Urine	Urine	Urine	(ALL)	0 Minutes	150 Years	(ALL)		25.0	U24H HCL	Ref Refrig	Refrigerate	0	
868	Metanephtrines, Fract, Plas	Blood	Chem Sendout Frozen	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Lav	Ref Frozen	Freeze	0	
869	Metanephtrines, Fract, Plas	Blood	Chem Sendout Frozen	Venous Draw	LabCorp	0 Minutes	150 Years	(ALL)		4.0	Lav	Ref Refrig		0	
870	Metanephtrines, Pheochromocyt	Urine	Chem Sendout	Urine	(ALL)	0 Minutes	150 Years	(ALL)		10.0	UA	Ref Refrig		0	
871	Methadone Screen Urine	Urine	Urine	Urine	(ALL)	0 Minutes	150 Years	(ALL)		25.0	UA	Ref Room	Room Temperature	0	
872	Methemoglobin	Blood Heparin	Blood Gases Arterial	Arterial Draw	(ALL)	0 Minutes	150 Years	(ALL)		0.3	Hep Syringe	Chem	Deliver Immediately	0	
873	Methotrexate	Blood	Blood	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)		0.3	SST (micro)	Chem		0	
874	Methotrexate	Blood	Blood	Venous Draw	(ALL)	1 Months	150 Years	(ALL)		0.3	SST	Chem	Draw prior to next dose	0	
875	Methotrexate CSF	CSF	CSF	Syringe	(ALL)	0 Minutes	150 Years	(ALL)		0.5	Sterile T	Chem		0	
876	Methsuximide	Blood	Chem Red Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		4.0	Red	Ref Refrig	Refrigerate	0	
877	Methylmalonic Acid	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		3.0	SST	Ref Refrig		0	
878	Microalbumin 24Hr Urine	Urine	Urine 24hr	Urine	(ALL)	0 Minutes	150 Years	(ALL)		2.0	U24H	Urine Chem		0	
879	Microalbumin:Creatinine Ratio	Urine	Urine	Urine	(ALL)	0 Minutes	150 Years	(ALL)		1.0	UA	Urine Chem		0	
880	Microfilariae Detection	Blood	Hem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		5.0	Lav	Hem	Room Temperature	0	
881	Microsatellite(MSI),HNPPC	Paraffin Block	Chem Sendout	Tissue	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Contact Laboratory	Ref Room	Room Temperature	0	
882	Microsatellite(MSI),HNPPC	Paraffin Block	Chem Sendout	Tissue	(ALL)	0 Minutes	150 Years	(ALL)		5.0	Lav	Ref Room	Room Temperature	0	
883	Mixing Studies APTT	Blood	Blood Coag	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		2.0	Blue	Coag	Mix Immediately	0	
884	Mixing Studies PT	Blood	Blood Coag	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		2.0	Blue	Coag	Mix Immediately	0	
885	Mol Dx Director Review	Blood	GC Genetics Cyxtic Fibrosis	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Lav	MolecularC		0	
886	Monosaccharides Urine	Urine	Urine	Urine	(ALL)	0 Minutes	150 Years	(ALL)		5.0	UA	Chem	Freeze	0	
887	Motor Sensory Neuropathy Evaluation	Blood	Blood	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		10.0	SST	Chem	Refrigerate	0	
888	Mucchopolysaccharide Identification Urine	Urine	Urine	Urine	(ALL)	0 Minutes	150 Years	(ALL)		20.0	U24H	Chem	Freeze	0	
889	Mucopolipidosis Type IV, NY	Blood	Chem lav sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		3.0	Lav	Ref Room	Room Temperature	0	
890	Mucopolysaccharide SCR UR	Urine	Urine	Urine	(ALL)	0 Minutes	150 Years	(ALL)		20.0	U Aliquot	Ref Frozen	Freeze	0	
891	Mumps Antibody Immune Status IgG	Blood	VS	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)		1.0	SST	Serology		0	
892	Mumps Antibody Immune Status IgG	Blood	VS	Venous Draw	(ALL)	31 Days	150 Years	(ALL)		1.0	SST	Serology		0	
893	Mumps Virus Ab (IGM)	Blood	Chem Red Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		2.0	Red	Ref Refrig	Refrigerate	0	
894	Musk Antibody	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		2.0	SST	Ref Refrig	Refrigerate	0	
895	Myeline Basic Protein CSF	CSF	CSF	Special Collection	(ALL)	0 Minutes	150 Years	(ALL)		2.0	Transport	Ref Refrig	Refrigerate	0	
896	Myocardial Abs	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	SST	Ref Room	Room Temperature	0	
897	Myoglobin Urine	Urine	Urine	Urine	(ALL)	0 Minutes	150 Years	(ALL)		10.0	UA	Urinalysis		0	
898	Myoglobin, Serum	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	SST	Ref Refrig		0	
899	NRBC - LAB ORDER ONLY	Blood	Blood Hem	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Lav	Hem		0	
900	Natural Killer Cells	Blood	Flow Lav	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Lav	Flow	Room Temperature	0	
901	Neisseria gonorrhoeae DNA by PCR	Urethral Swab	VM	Dacron Swab	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Vir/Chlam	Virol	Refrigerate	0	
902	Neisseria gonorrhoeae DNA by PCR	Cervical Swab	VM	Dacron Swab	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Vir/Chlam	Virol	Refrigerate	0	
903	Neonatal T Bili Blood	Blood	Blood Gases Arterial	Syringe	(ALL)	0 Minutes	150 Years	(ALL)		0.3	Hep Syringe	Chem	Deliver Immediately	0	
904	Neuromyelitis Optica Autoantibody, IgG, Serum	Blood	Chem Red Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		2.0	Red	Ref Frozen	Freeze	0	
905	Neuronal Nucl(HU)Ab W/RFL	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	SST	Ref Room	Room Temperature	0	
906	Neuronal Nucl(RI)Ab W/RFL	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	SST	Ref Room	Room Temperature	0	
907	Nortriptyline	Blood	Chem Red Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		5.0	Red	Ref Refrig	Refrigerate	0	
908	Nuclear Matrix Protein	Urine	Urine	Special Collection	(ALL)	0 Minutes	150 Years	(ALL)		10.0	Contact Laboratory	Ref Refrig	Refrigerate	0	
909	Occult Blood Stool	Stool	Stool	Slide Prep	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Occult	Urinalysis		0	
910	Oligoclonal Banding CSF	OTHER	CSF	CSF and Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		0.5	SST	Chem		0	
911	Oligoclonal Banding CSF	OTHER	CSF	CSF and Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		2.0	Sterile Plastic	Chem		0	
912	Oligosaccharide Urine	Urine	Urine	Urine	(ALL)	0 Minutes	150 Years	(ALL)		20.0	U Aliquot	Ref Frozen	Freeze	0	
913	Oncotic Pressure	Blood Heparin	Blood Gases Arterial	Syringe	(ALL)	0 Minutes	150 Years	(ALL)		0.1	Hep Syringe	Chem	Deliver Immediately	0	
914	Opiates (GC/MS), Blood	Blood	Chem Sendout Room	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		8.0	Grey	Ref Room	Room Temperature	0	
915	Oraqquick HIV-1	Blood	VH	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	SST	Chem		0	
916	Organic Acid Qualitative Urine	Urine	Chem Sendout Frozen	Urine	(ALL)	0 Minutes	150 Years	(ALL)		20.0	U Aliquot	Chem	Freeze	0	
917	Organic Acid Qualitative Urine	Urine	Chem Sendout Frozen	Urine	LabCorp	0 Minutes	150 Years	(ALL)		10.0	UA	Ref Frozen	Freeze	0	
918	Organic Acid, QN,UR,NY	Urine	Urine	Urine	(ALL)	0 Minutes	150 Years	(ALL)		20.0	U Aliquot	Chem	Freeze	0	
919	Orotic Acid	Urine	Urine	Urine	(ALL)	0 Minutes	150 Years	(ALL)		10.0	U Aliquot	Chem	Freeze	0	
920	Osmolality Renal Perfusate	Perfusate	Body Fluid	Syringe	(ALL)	0 Minutes	150 Years	(ALL)		0.3	Hep Syringe	Chem		0	
921	Osmolality Serum	Blood	Blood	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)		0.3	SST (micro)	Chem		0	
922	Osmolality Serum	Blood	Blood	Venous Draw	(ALL)	31 Days	150 Years	(ALL)		1.0	SST	Chem		0	
923	Osmolality Urine	Urine	Urine	Urine	(ALL)	0 Minutes	150 Years	(ALL)		2.0	UA	Urine Chem		0	

	Primary Synonym	Specimen Type	Accession Class	Collection Method	Service Resource	Age From	Age To	Coll Priority	Aliq	Min Vol	Container	Collection Class	Special Handling	Add Lbl	Opt
924	Osteocalcin, Serum	Blood	Chem Sendout Frozen	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		3.0	SST	Ref Frozen	Freeze	0	
925	Oxalic Acid 24Hr Ur,W/Crea	Urine	Urine HCL	Urine	(ALL)	0 Minutes	150 Years	(ALL)		30.0	U24H HCL	Ref Refrig	Refrigerate	0	
926	Oxalic Acid, 24hr W/O Creat	Urine	Urine	Urine	(ALL)	0 Minutes	150 Years	(ALL)		30.0	U24H	Ref Room	Room Temperature	0	
927	Oxcarbazepine Metabolite	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Red	Ref Refrig	Refrigerate	0	
928	PML/RARA T(15 17)QN,PCR	Blood	Chem iav sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		6.0	Lav	Ref Refrig	Refrigerate	0	
929	PNH,CD59,FC,NY	Blood	Chem iav sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		5.0	Lav	Ref Room	Room Temperature	0	
930	POC ACT LR	Blood	Blood Coag	Syringe	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Plastic Syringe	Chem		0	
931	POC ACT PLUS	Blood	Blood Coag	Syringe	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Plastic Syringe	Chem		0	
932	POC Glucose	Blood	POC BLD	Capillary	(ALL)	0 Minutes	150 Years	(ALL)		0.2	Capillary	Chem		0	
933	POC Ilb_IIIa	Blood	Blood	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Green	Ref Refrig		0	
934	POC Occult Blood	Stool	Stool	Stool	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Occult	Urinalysis		0	
935	POC P2Y12	Blood	POC BLD	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Blue	Chem		0	
936	POC Plat Funct Aspirin	Blood	POC BLD	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Blue	Chem		0	
937	POC Pregnancy Urine	Urine	POC Ur	Urine	(ALL)	0 Minutes	150 Years	(ALL)		1.0	UA	Urinalysis		0	
938	POC Urinalysis	Urine	Urinalysis	Urine	(ALL)	0 Minutes	150 Years	(ALL)		3.0	UA	Urinalysis		0	
939	PSA Free	Blood	Chem E170	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	SST	SP Chem		0	
940	PTH, Related Protein	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Green Sodium	Ref Room	Room Temperature	0	
941	PTH, Related Protein	Blood	Chem Sendout	Venous Draw	LabCorp	0 Minutes	150 Years	(ALL)		3.0	Lav	Ref Frozen		0	
942	Pancreatic Elastase-1	Stool	Stool	Stool	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Stool Container	Ref Refrig	Refrigerate	0	
943	Parathyroid Hormone Intact	Blood	Blood PTH	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		5.0	SST	SP Chem	Collect prior to 10 AM fast 8-10 hrs	0	
944	Parietal Cell Antibody	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	SST	Ref Refrig	Refrigerate	0	
945	Parietal Cell Antibody	Blood	Chem Sendout	Venous Draw	LabCorp	0 Minutes	150 Years	(ALL)		3.0	SST	Ref Frozen		0	
946	Paroxetine	Blood	Chem Red Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		5.0	Red	Ref Refrig	Refrigerate	0	
947	Parvovirus B19 IgG IgM	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		2.0	SST	Ref Refrig	Refrigerate	0	
948	Parvovirus B19 IgG IgM	Blood	Chem Sendout	Venous Draw	LabCorp	0 Minutes	150 Years	(ALL)		3.0	SST	Ref Room		0	
949	Parvovirus B19 DNA, QL,PCR	Tissue	Chem Sendout Frozen	Tissue	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Sterile Plastic	Ref Frozen	Freeze	0	
950	Parvovirus B19 DNA, QL,PCR	Blood	Chem Sendout Frozen	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Yellow (ACD)	Ref Frozen	Freeze	0	
951	Parvovirus B19 DNA, QL,PCR	Bone Marrow	Chem Sendout	Special Collection	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Yellow (ACD)	Ref Refrig	Refrigerate	0	
952	Parvovirus B19 DNA, QL,PCR	Blood	Chem Sendout Frozen	Venous Draw	LabCorp	0 Minutes	150 Years	(ALL)		1.0	Lav	Ref Room		0	
953	Pediatric Allergy Food Panel	Blood	Immunology	Venous Draw	(ALL)	0 Minutes	12 Years	(ALL)		2.0	SST	Immune		0	
954	Phencyclidine (GC/MS), Blood	Blood	Chem Sendout Room	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		8.0	Grey	Ref Room	Room Temperature	0	
955	Phenobarbital	Blood	Blood	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)		0.3	SST (micro)	Chem		0	
956	Phenobarbital	Blood	Blood	Venous Draw	(ALL)	1 Months	150 Years	(ALL)		0.3	SST	Chem	Draw prior to next dose	0	
957	Phenosense GT (TM), NY	Blood	Chem Sendout Frozen	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		3.0	PPT	Chem	Freeze	0	
958	Phenylalanine, NY	Blood	Chem Sendout Frozen	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		5.0	Green	Ref Frozen		0	
959	Phenytoin	Blood	Blood	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)		0.3	SST (micro)	Ref Refrig		0	
960	Phenytoin	Blood	Blood	Venous Draw	(ALL)	31 Days	150 Years	(ALL)		0.3	SST	Chem	Draw prior to next dose	0	
961	Phenytoin Free	Blood	Chem Red Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		5.0	Red	Ref Refrig	Refrigerate	0	
962	Phospholipids	Blood	Blood	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		3.0	Red	Ref Refrig	Refrigerate	0	
963	Phosphorus	Blood	Blood	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)		0.2	SST (micro)	Chem		0	
964	Phosphorus	Blood	Blood	Venous Draw	(ALL)	31 Days	150 Years	(ALL)		0.2	SST	Chem		0	
965	Phosphorus Urine 24Hr	Urine	Urine 24hr	Urine	(ALL)	0 Minutes	150 Years	(ALL)		2.0	U24H	Urine Chem	Refrigerate	0	
966	Phosphorus Urine Random	Urine	Urine	Urine	(ALL)	0 Minutes	150 Years	(ALL)		2.0	UA	Urine Chem		0	
967	Plasminogen	Blood	Blood	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		4.5	Blue	Ref Frozen	Freeze	0	
968	Platelet Ab (IgG), Indirect	Blood	Chem Sendout Frozen	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Red	Ref Frozen	Freeze	0	
969	Platelet Ab Heparin Induced	Blood	Hem Red	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		2.0	Red	Ref Frozen	Freeze	0	
970	Platelet Function Aspirin	Blood	Blood Coag	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Contact Laboratory	Coag	Initials/D&T: _____	0	
971	Platelet Function P2Y12	Blood	Blood Coag	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Contact Laboratory	Coag	Initials/D&T: _____	0	
972	Platelet Glycoprotein Ab	Blood	Hem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		6.0	Lav	Ref Room	Room Temperature	0	
973	Poliovirus Antibody	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		3.0	SST	Ref Refrig	Refrigerate	0	
974	Porphobilinogen Quantitative 24Hour Urine	Urine	Urine 24hr	Urine	(ALL)	0 Minutes	150 Years	(ALL)		10.0	Amber	Ref Frozen	Protect from light	0	
975	Porphobilinogen Urine Random	Urine	Urine	Urine	(ALL)	0 Minutes	150 Years	(ALL)		10.0	Amber	Ref Frozen	Protect from light	0	
976	Porphyrins Urine 24Hour	Urine	Urine 24hr	Urine	(ALL)	0 Minutes	150 Years	(ALL)		10.0	Amber	Ref Refrig	Protect from light	0	
977	Post BUN	Blood	Post	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)		0.2	SST (micro)	Chem		0	
978	Post BUN	Blood	Post	Venous Draw	(ALL)	1 Minutes	150 Years	(ALL)		0.2	SST	Chem		0	
979	Post Creatinine	Blood	Post	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)		0.2	SST (micro)	Chem		0	
980	Post Creatinine	Blood	Post	Venous Draw	(ALL)	1 Months	150 Years	(ALL)		0.2	SST	Chem		0	
981	Post Dialysis Care Set	Blood	Post	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)		0.2	SST (micro)	Chem		0	
982	Post Dialysis Care Set	Blood	Post	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)		0.2	SST (micro)	Chem		0	
983	Post Dialysis Care Set	Blood	Post	Venous Draw	(ALL)	1 Minutes	150 Years	(ALL)		0.2	SST	Chem		0	
984	Post Dialysis Care Set	Blood	Post	Venous Draw	(ALL)	1 Minutes	150 Years	(ALL)		0.2	SST	Chem		0	
985	Potassium	Blood	Blood	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)		0.2	SST (micro)	Chem		0	
986	Potassium	Blood	Blood	Venous Draw	(ALL)	1 Months	150 Years	(ALL)		0.2	SST	Chem		0	
987	Potassium Dialysate	DIALY	Special Collectio	Special Collection	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Red	Chem		0	
988	Potassium Whole Blood	Blood Heparin	Blood Gases Arterial	Syringe	(ALL)	0 Minutes	150 Years	(ALL)		0.1	Hep Syringe	Chem	Deliver Immediately	0	
989	Potassium Cardioplesia	Cardioplegia	Body Fluid	Pharmacy	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Red	Chem		0	
990	Potassium Renal Perfusate	Perfusate	Body Fluid	Syringe	(ALL)	0 Minutes	150 Years	(ALL)		0.3	Hep Syringe	Chem		0	
991	Potassium Urine 24Hr	Urine	Urine 24hr	Urine	(ALL)	0 Minutes	150 Years	(ALL)		2.0	U24H	Urine Chem	Refrigerate	0	
992	Potassium TPN Pharmacy	TPN	Special Collectio	Pharmacy	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Red	Chem		0	
993	Potassium Urine Random	Urine	Urine	Urine	(ALL)	0 Minutes	150 Years	(ALL)		2.0	UA	Urine Chem		0	
994	Prealbumin	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		2.0	SST	Ref Refrig	Refrigerate	0	

	Primary Synonym	Specimen Type	Accession Class	Collection Method	Service Resource	Age From	Age To	Coll Priority	Aliq	Min Vol	Container	Collection Class	Special Handling	Add Lbl	Opt
995	Pregnancy Test Urine	Urine	Urinalysis	Urine	(ALL)	0 Minutes	150 Years	(ALL)		5.0	UA	Urinalysis		0	
996	Pregnancy Urine POCT	Urine	Julian Date	Urine	(ALL)	0 Minutes	150 Years	(ALL)		1.0	UA	Chem		0	
997	Pregnenolone	Blood	Chem Sendout Frozen	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.1	Red	Ref Frozen	Freeze	0	
998	Primidone	Blood	Chem Red Sendout	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)		0.3	Red	Ref Refrig	Refrigerate	0	
999	Primidone	Blood	Chem Red Sendout	Venous Draw	(ALL)	31 Days	150 Years	(ALL)		0.3	Red	Ref Refrig	Refrigerate	0	
1000	Pro-Predictrx Metabolites	Blood	Chem lav sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		5.0	Lav	Ref Refrig	Refrigerate	0	
1001	Procainamide & NAPA	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)		0.3	Red (micro)	Ref Refrig		0	
1002	Procainamide & NAPA	Blood	Chem Sendout	Venous Draw	(ALL)	31 Days	150 Years	(ALL)		0.3	Red	Ref Refrig	Draw prior to next dose	0	
1003	Progesterone	Blood	Chem E170	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	SST	SP Chem		0	
1004	Prolactin	Blood	Chem Architect	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	SST	SP Chem		0	
1005	Prostate Specific Antigen	Blood	Chem E170	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	SST	SP Chem		0	
1006	Protein CSF	CSF	CSF	Syringe	(ALL)	0 Minutes	150 Years	(ALL)		0.5	Sterile T	Chem	Deliver Immediately	0	
1007	Protein Urine 24Hr	Urine	Urine 24hr	Urine	(ALL)	0 Minutes	150 Years	(ALL)		2.0	U24H	Urine Chem	Refrigerate	0	
1008	Protein Urine Random	Urine	Urine	Urine	(ALL)	0 Minutes	150 Years	(ALL)		2.0	UA	Urine Chem		0	
1009	Protein C Functional	Blood	Hem Blue Sendouts	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		2.0	Blue	Ref Frozen	Freeze	0	
1010	Protein C, Antigen	Blood	Hem Blue Sendouts	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Blue	Ref Frozen	Freeze	0	
1011	Protein Electrophoresis CSF	CSF	CSF	Syringe	(ALL)	0 Minutes	150 Years	(ALL)		2.5	Sterile T	Chem	Deliver Immediately	0	
1012	Protein Electrophoresis Serum	Blood	Blood	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		2.0	SST	Chem		0	
1013	Protein Electrophoresis Urine	Urine	Urine	Urine	(ALL)	0 Minutes	150 Years	(ALL)		8.0	UA	Chem		0	
1014	Protein Electrophoresis 24hr Ur	Urine	Urine 24hr	Urine	(ALL)	0 Minutes	150 Years	(ALL)		2.0	U24H	Urine Chem	Refrigerate	0	
1015	Protein Pericardial Fluid	Pericardial Fluid	Body Fluid	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)		0.3	Red	Chem		0	
1016	Protein Peritoneal Fluid	Peritoneal Fluid	Body Fluid	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)		0.3	Red	Chem		0	
1017	Protein Pleural Fluid	Pleural Fluid	Body Fluid	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)		0.3	Red	Chem		0	
1018	Protein S Antigen	Blood	Hem Blue Sendouts	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Blue	Ref Frozen	Freeze	0	
1019	Protein S Functional	Blood	Hem Blue Sendouts	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		3.0	Blue	Ref Frozen	Freeze	0	
1020	Protein S Panel	Blood	Blood	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		6.0	Blue	Ref Frozen	Freeze	0	
1021	Protein Synovial Fluid	Synovial fluid	Body Fluid	Syringe	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Red	Chem		0	
1022	Protein Total	Blood	Blood	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)		0.2	SST (micro)	Chem		0	
1023	Protein Total	Blood	Blood	Venous Draw	(ALL)	1 Months	150 Years	(ALL)		0.3	SST	Chem		0	
1024	Protein Urine Timed	Urine	Urine	Urine	(ALL)	0 Minutes	150 Years	(ALL)		2.0	UA	Urine Chem	Refrigerate	0	
1025	Proteinase-3 Ab	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	SST	Ref Refrig		0	
1026	Prothrombin Time	Blood	Blood Coag	Venous Draw	(ALL)	0 Minutes	12 Years	(ALL)		0.5	Blue (Peds)	Coag	Mix Immediately	0	
1027	Prothrombin Time	Blood	Blood Coag	Venous Draw	(ALL)	12 Years	150 Years	(ALL)		0.5	Blue	Coag	Mix Immediately	0	
1028	Purkinje Cells Ab W/RFX	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	SST	Ref Room	Room Temperature	0	
1029	Pyruvate Kinase	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		3.0	Yellow (ACD)	Ref Refrig	Refrigerate	0	
1030	Pyruvic Acid	Blood	Chem Sendout Frozen	Special Collection	(ALL)	0 Minutes	150 Years	(ALL)		5.0	Perchloric Acid	Ref Frozen	Freeze	0	
1031	Q Fever Ab (IGG,M) W/RFX	Blood	Chem Red Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		2.0	Red	Ref Refrig	Refrigerate	0	
1032	Quantiferon-TB Gold In-Tube	Blood	QUANT	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	QT Grey	Virol	FILL TO BLK MARK	0	
1033	Quantiferon-TB Gold In-Tube	Blood	QUANT	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	QT Red	Virol	FILL TO BLK MARK	0	
1034	Quantiferon-TB Gold In-Tube	Blood	QUANT	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	QT Purple	Virol	FILL TO BLK MARK	0	
1035	Quinidine	Blood	Chem Red Sendout	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)		0.3	Red (micro)	Ref Refrig		0	
1036	Quinidine	Blood	Chem Red Sendout	Venous Draw	(ALL)	31 Days	150 Years	(ALL)		0.3	Red	Ref Refrig	Draw prior to next dose	0	
1037	RPR Serum	Blood	VS	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)		0.2	SST	Serology		0	
1038	RPR Serum	Cord Blood	VS	Special Collection	(ALL)	0 Minutes	150 Years	(ALL)		1.0	SST	Serology		0	
1039	RPR Serum	Blood	VS	Venous Draw	(ALL)	1 Months	150 Years	(ALL)		0.2	SST	Serology		0	
1040	Reducing Substances	Urine	Urinalysis	Urine	(ALL)	0 Minutes	150 Years	(ALL)		0.5	UA	Urinalysis		0	
1041	Remove Any Test Not Needed	OTHER	Blood	Special Collection	(ALL)	0 Minutes	150 Years	(ALL)		1.0	NoContainer	Chem		0	
1042	Renal Function Panel	Blood	Blood	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	SST	Chem		0	
1043	Renin	Blood	Chem Sendout Frozen	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		3.0	Lav	Ref Frozen	Freeze	0	
1044	Renin Act/Aldosterone(P)	Blood	Chem Sendout Frozen	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		3.0	Lav	Ref Frozen	Freeze	0	
1045	Repo Exposure Panel	Blood	VS	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Red	Serology	Freeze	0	
1046	Reptilase Time	Blood	Blood Coag	Venous Draw	(ALL)	0 Minutes	12 Years	(ALL)		0.5	Blue	Coag	Mix Immediately	0	
1047	Reptilase Time	Blood	Blood Coag	Venous Draw	(ALL)	12 Years	150 Years	(ALL)		0.5	Blue	Coag	Mix Immediately	0	
1048	Research Phlebotomy	Blood	POC BLD	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	SST	Chem		0	
1049	Research Processing	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	SST	Chem	Freeze	0	
1050	Reticulin Ab (IgA) W/RFX	Blood	Chem Red Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Red	Ref Refrig	Refrigerate	0	
1051	Reticulin Ab (IgG)	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	SST	Ref Refrig	Refrigerate	0	
1052	Reticulocyte Count	Blood	Blood Hem	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)		0.2	Lav (micro)	Hem		0	
1053	Reticulocyte Count	Blood	Blood Hem	Venous Draw	(ALL)	1 Months	150 Years	(ALL)		1.0	Lav	Hem		0	
1054	Reticulocyte Count Manual	Blood	Blood Hem	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		0.5	Lav	Hem		0	
1055	Retinol Binding Protein	Blood	Chem Sendout Frozen	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	SST	Ref Frozen	Freeze	0	
1056	Rett Syndrome Mutation,NY	Blood	Chem lav sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		2.0	Lav	Ref Room	Room Temperature	0	
1057	Rheumatoid Factor Serum	Blood	Blood	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		0.5	SST	Chem		0	
1058	Rickettsial Ab PNL W/REFL	Blood	Chem Red Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		2.0	Red	Ref Refrig	Refrigerate	0	
1059	Rocky MT Spot (G,M)W/RFX	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		3.0	SST	Ref Refrig	Refrigerate	0	
1060	Rubella Antibodies IgM	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		5.0	SST	Ref Refrig	Refrigerate	0	
1061	Rubella Antibody Immune Status IgG by EIA	Blood	VS	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		0.3	SST	Serology	Room Temperature	0	
1062	S. Cerevisiae Ab (IgG/A)	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	SST	Ref Refrig		0	
1063	S.PNEUMONIAE IGG (14)	Blood	Chem Red Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Red	Ref Refrig	Refrigerate	0	
1064	S.PNEUMONIAE IGG (14)	Blood	Chem Red Sendout	Venous Draw	LabCorp	0 Minutes	150 Years	(ALL)		3.0	SST	Ref Refrig		0	
1065	Salicylate	Blood	Blood	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)		0.3	SST (micro)	Chem		0	

	Primary Synonym	Specimen Type	Accession Class	Collection Method	Service Resource	Age From	Age To	Coll Priority	Aliq	Min Vol	Container	Collection Class	Special Handling	Add Lbl	Opt
1066	Salicylate	Blood	Blood	Venous Draw	(ALL)	1 Months	150 Years	(ALL)		0.3	SST	Chem	Draw prior to next dose	0	
1067	Salvaged Potassium	Blood	Blood	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)		0.2	SST	Chem		0	
1068	Salvaged Potassium	Blood	Blood	Venous Draw	(ALL)	1 Months	150 Years	(ALL)		0.2	SST	Chem		0	
1069	Salvaged Hematocrit	Blood	Blood Hem	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)		0.2	Lav (micro)	Hem		0	
1070	Salvaged Hematocrit	Blood	Blood Hem	Venous Draw	(ALL)	1 Months	150 Years	(ALL)		0.5	Lav	Hem		0	
1071	Salvaged PTT	Blood	Blood Coag	Venous Draw	(ALL)	0 Minutes	12 Years	(ALL)		0.5	Blue (Peds)	Coag		0	
1072	Salvaged PTT	Blood	Blood Coag	Venous Draw	(ALL)	12 Minutes	150 Years	(ALL)		0.5	Blue	Coag		0	
1073	Schistosoma Ab	Blood	Chem Red Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Red	Ref Refrig	Refrigerate	0	
1074	Selenium	Blood	Chem Royal	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		2.0	Royal (Serum)	Ref Room	Room Temperature	0	
1075	Semen Analysis	Body Fluid	Body Fluid	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Sterile Plastic	Chem	Room Temperature	0	
1076	Serology Comment (MCO)	Blood	Chem Red Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		0.3	Red	Ref Room	Room Temperature	0	
1077	Serotonin Release Assay	Blood	Chem Sendout Frozen	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		2.0	Red	Ref Frozen	Freeze	0	
1078	Serotonin Release Assay	Blood	Chem Sendout Frozen	Venous Draw	LabCorp	0 Minutes	150 Years	(ALL)		8.0	SST	Ref Refrig		0	
1079	Serotonin, Whole Blood	Blood	Chem Sendout Frozen	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		10.0	Lav	Ref Frozen	Contact Laboratory	0	
1080	Sex Hormone Binding Globulin	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		6.0	SST	Ref Refrig	Refrigerate	0	
1081	Sickle Cell Prep	Blood	Blood Hem	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)		0.2	Lav (micro)	Hem		0	
1082	Sickle Cell Prep	Blood	Blood Hem	Venous Draw	(ALL)	1 Months	150 Years	(ALL)		1.0	Lav	Hem		0	
1083	Sickle Solubility Test	Blood	Blood Hem	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)		0.2	Lav (micro)	Hem		0	
1084	Sickle Solubility Test	Blood	Blood Hem	Venous Draw	(ALL)	1 Months	150 Years	(ALL)		1.0	Lav	Hem		0	
1085	Single Site BRAC Analysis	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		3.0	Contact Laboratory	Chem	Room Temperature	0	
1086	Sirolimus	Blood	Immuno Lav	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		2.0	Lav	Chem	Draw prior to next dose	0	
1087	Sodium	Blood	Blood	Venous Draw	(ALL)	0 Minutes	30 Months	(ALL)		0.2	SST (micro)	Chem		0	
1088	Sodium	Blood	Blood	Venous Draw	(ALL)	31 Days	150 Years	(ALL)		0.2	SST	Chem		0	
1089	Sodium Dialysate	DIALY	Special Collectio	Special Collection	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Red	Chem		0	
1090	Sodium Urine 24Hr	Urine	Urine 24hr	Urine	(ALL)	0 Minutes	150 Years	(ALL)		2.0	U24H	Urine Chem	Refrigerate	0	
1091	Sodium Blood	Blood Heparin	Blood Gases Arterial	Syringe	(ALL)	0 Minutes	150 Years	(ALL)		0.1	Hep Syringe	Chem	Deliver Immediately	0	
1092	Sodium Renal Perfusate	Perfusate	Body Fluid	Syringe	(ALL)	0 Minutes	150 Years	(ALL)		0.3	Hep Syringe	Chem		0	
1093	Sodium TPN Pharmacy	TPN	Special Collectio	Pharmacy	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Red	Chem		0	
1094	Sodium Urine Random	Urine	Urine	Urine	(ALL)	0 Minutes	150 Years	(ALL)		2.0	UA	Urine Chem		0	
1095	Soluble Transferrin Receptor	Blood	Chem Red Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Red	Ref Refrig		0	
1096	Specific Gravity	Urine	Urinalysis	Urine	(ALL)	0 Minutes	150 Years	(ALL)		0.5	UA	Urinalysis		0	
1097	Specific Gravity Fluid	Body Fluid	Body Fluid	Syringe	(ALL)	0 Minutes	150 Years	(ALL)		0.5	Lav	Urinalysis		0	
1098	Streptomycin, HPLC,Ser	CSF	Chem Sendout Frozen	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Sterile Plastic	Ref Frozen	Freeze	0	
1099	Streptomycin, HPLC,Ser	Blood	Chem Sendout Frozen	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		2.0	Red	Ref Frozen	Freeze	0	
1100	Sulfate, 24Hr Urine	Urine	Urine	Urine	(ALL)	0 Minutes	150 Years	(ALL)		10.0	U24H	Ref Refrig	Refrigerate	0	
1101	Syphilis Confirmation	Blood	VS	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	SST	Serology		0	
1102	T Cell Immune Function-ATP	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		3.0	Green	Ref Room	Contact Laboratory	0	
1103	T Cell Panel	Blood	Flow Lav	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Lav	Flow	Room Temperature	0	
1104	T and B Cell Panel	Blood	Flow Lav	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Lav	Flow	Room Temperature	0	
1105	T, B and NK Cell Panel	Blood	Flow Lav	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		0.5	Lav	Flow	Room Temperature	0	
1106	T. Vaginalis Culture	Genital Swab	Chem Sendout	Swab	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Contact Laboratory	Chem	Room Temperature	0	
1107	T3	Blood	Chem Architect	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		0.5	SST	SP Chem		0	
1108	T3 Free	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		3.0	SST	Ref Refrig	Refrigerate	0	
1109	T3 Uptake	Blood	Chem Architect	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		0.5	SST	SP Chem		0	
1110	T3,Reverse	Blood	Chem Red Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		2.0	Red	Ref Refrig	Refrigerate	0	
1111	T4 Free	Blood	Chem Architect	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		0.5	SST	SP Chem		0	
1112	T4 Free by Equilibrium Dialysis	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		5.0	SST	Ref Refrig	Refrigerate	0	
1113	T4 Free by Equilibrium Dialysis	Blood	Chem Sendout	Venous Draw	LabCorp	0 Minutes	150 Years	(ALL)		4.0	Red	Ref Frozen	Freeze	0	
1114	TBG	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		2.0	SST	Ref Refrig	Refrigerate	0	
1115	TCR Rearrange, QL, Cell	Bone Marrow	Chem lav sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Lav	Ref Refrig	Refrigerate	0	
1116	TCR Rearrange,QN,Cell	Blood	Chem lav sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		3.0	Lav	Ref Refrig	Refrigerate	0	
1117	TCR Rearrange,QN,Cell	Bone Marrow	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		3.0	Lav	Ref Refrig	Refrigerate	0	
1118	TPMT Genotype (NY)	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		8.0	Green	Ref Refrig	Refrigerate	0	
1119	TRACP,IHC W/O Interp	Paraffin Block	Chem Sendout	Special Collection	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Contact Laboratory	Ref Room	Room Temperature	0	
1120	Testosterone	Blood	Chem E170	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Red	SP Chem		0	
1121	Testosterone Free	Blood	Chem Red Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		3.0	Red	Ref Refrig	Refrigerate	0	
1122	Tetanus Toxoid Antibodies	Blood	Chem Sendout	Venous Draw	LabCorp	0 Minutes	150 Years	(ALL)		3.0	SST	Ref Refrig	Refrigerate	0	
1123	Theophylline	Blood	Blood	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		0.3	SST	Chem	Draw prior to next dose	0	
1124	Thiocyanate	Blood	Chem Red Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		0.3	Red	Ref Refrig	Refrigerate	0	
1125	Thioridazine	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		0.3	SST	Ref Room	Room Temperature	0	
1126	Thrombin Time	Blood	Blood Coag	Venous Draw	(ALL)	0 Minutes	12 Years	(ALL)		0.5	Blue (Peds)	Coag	Mix Immediately	0	
1127	Thrombin Time	Blood	Blood Coag	Venous Draw	(ALL)	12 Years	150 Years	(ALL)		0.5	Blue	Coag	Mix Immediately	0	
1128	Thyroglobulin Ab	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		2.0	SST	Ref Refrig	Refrigerate	0	
1129	Thyroglobulin Panel	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		2.0	SST	Ref Refrig	Refrigerate	0	
1130	Thyroid Antibodies	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		4.0	SST	Ref Refrig	Refrigerate	0	
1131	Thyroid Stimulating Hormone	Blood	Chem Architect	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	SST	SP Chem		0	
1132	Thyroid Stimulating Immunoglobulin	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	SST	Ref Refrig	Refrigerate	0	
1133	Thyrotropin Receptor Antibody	Blood	Chem Red Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		0.3	Red	Ref Refrig	Refrigerate	0	
1134	Thyroxine	Blood	Chem Architect	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		0.5	SST	SP Chem		0	
1135	Tiagabine(Gabitril)	Blood	Chem Red Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		2.0	Red	Ref Refrig	Refrigerate	0	
1136	Tick, Other Arthropods, ID	OTHER	Chem Sendout	Special Collection	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Contact Laboratory	Ref Room	Room Temperature	0	

	Primary Synonym	Specimen Type	Accession Class	Collection Method	Service Resource	Age From	Age To	Coll Priority	Aliq	Min Vol	Container	Collection Class	Special Handling	Add Lbl	Opt
1137	Tissue Transglutam Ab IgA	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	SST	Ref Refrig	Refrigerate	0	
1138	Tissue Transglutam Ab IgG	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	SST	Ref Refrig	Refrigerate	0	
1139	Tobramycin Peak	Serum	Peak	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)		0.3	SST (micro)	Chem		0	
1140	Tobramycin Peak	Serum	Peak	Venous Draw	(ALL)	31 Days	150 Years	(ALL)		0.3	SST	Chem		0	
1141	Tobramycin Trough	Serum	Trough	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)		0.3	SST (micro)	Chem	Draw prior to next dose	0	
1142	Tobramycin Trough	Serum	Trough	Venous Draw	(ALL)	31 Days	150 Years	(ALL)		0.3	SST	Chem	Draw prior to next dose	0	
1143	Topiramate	Blood	Chem Red Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Red	Ref Refrig	Refrigerate	0	
1144	Toxoplasma Antibodies IgG & IgM	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		2.0	SST	Ref Refrig	Refrigerate	0	
1145	Toxoplasmosis AB(IGG),CSF	CSF	CSF	Syringe	(ALL)	0 Minutes	150 Years	(ALL)		0.5	Sterile T	Ref Refrig	Deliver Immediately	0	
1146	Toxoplasmosis gondii Antibodies IgG	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	SST	Ref Refrig	Refrigerate	0	
1147	Transferrin	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		0.3	SST	Ref Refrig		0	
1148	Triglycerides	Blood	Blood	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)		0.2	SST (micro)	Chem		0	
1149	Triglycerides	Blood	Blood	Venous Draw	(ALL)	1 Months	150 Years	(ALL)		0.3	SST	Chem		0	
1150	Triglycerides Peritoneal Fluid	Peritoneal Fluid	Body Fluid	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)		0.3	Red	Chem		0	
1151	Triglycerides Pleural Fluid	Pleural Fluid	Body Fluid	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)		0.3	Red	Chem		0	
1152	Trofile Assay	Blood	Chem lav sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		3.0	PPT	Chem	Freeze	0	
1153	Troponin I	Blood	Blood Troponin	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)		0.3	SST (micro)	Chem		0	
1154	Troponin I	Blood	Blood Troponin	Venous Draw	(ALL)	31 Days	150 Years	(ALL)		0.3	SST	Chem		0	
1155	Trypsin	Blood	Chem Sendout Frozen	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	SST	Ref Frozen	Freeze	0	
1156	Tryptase	Blood	Chem Red Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Red	Ref Refrig	Refrigerate	0	
1157	Type II Collagen Ab	Blood	Chem Red Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		2.0	SST	Ref Refrig	Refrigerate	0	
1158	Unigold HIV-1	Blood	POC BLD	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	SST	Chem		0	
1159	Urea Nitrogen Urine 24Hr	Urine	Urine 24hr	Urine	(ALL)	0 Minutes	150 Years	(ALL)		2.0	U24H	Urine Chem	Refrigerate	0	
1160	Urea Nitrogen Urine Random	Urine	Urine	Urine	(ALL)	0 Minutes	150 Years	(ALL)		2.0	UA	Urine Chem		0	
1161	Urea Nitrogen Peritoneal Dialysate	DIALY	Body Fluid	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)		0.5	Red	Chem		0	
1162	Uric Acid	Blood	Blood	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)		0.3	SST (micro)	Chem		0	
1163	Uric Acid	Blood	Blood	Venous Draw	(ALL)	31 Days	150 Years	(ALL)		0.3	SST	Chem		0	
1164	Uric Acid Urine 24 Hr	Urine	Urine 24hr	Urine	(ALL)	0 Minutes	150 Years	(ALL)		2.0	U24H	Urine Chem	Refrigerate	0	
1165	Uric Acid Urine Random	Urine	Urine	Urine	(ALL)	0 Minutes	150 Years	(ALL)		2.0	UA	Urine Chem		0	
1166	Urinalysis	Urine	Urinalysis	Urine	(ALL)	0 Minutes	150 Years	(ALL)		3.0	UA	Urinalysis		0	
1167	Urinalysis with Microscopic	Urine	Urinalysis	Urine	(ALL)	0 Minutes	150 Years	(ALL)		2.0	UA	Urinalysis		0	
1168	Urine Microscopic	Urine	Urinalysis	Urine	(ALL)	0 Minutes	150 Years	(ALL)		5.0	UA	Urinalysis		0	
1169	Urine Toxicology 7 Drug	Urine	Urine	Urine	(ALL)	0 Minutes	150 Years	(ALL)		15.0	UA	Chem	Refrigerate	0	
1170	VAP (TM) Cholesterol Test	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		5.0	SST	Ref Refrig	Refrigerate	0	
1171	VDRL CSF	CSF	Chem Sendout Fluid	Body Fluid	(ALL)	0 Minutes	150 Years	(ALL)		0.5	Sterile Screw	Ref Refrig		0	
1172	VIP, Plasma	Blood	Chem Sendout Frozen	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		3.0	Lav	Ref Frozen		0	
1173	VMA Urine 24 hr	Urine	Urine 24hr	Urine	(ALL)	0 Minutes	150 Years	(ALL)		30.0	U24H HCL	Ref Refrig	Refrigerate	0	
1174	VWF Ag, Multimeric	Blood	Hem Blue Sendouts	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Blue	Ref Frozen	Freeze	0	
1175	VWF, Ristocetin Co-Factor	Blood	Hem Blue Sendouts	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		2.0	Blue	Ref Frozen	Freeze	0	
1176	Vaginal Wet Prep	Vaginal Exudates	Julian Date	Wash	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Sterile Plastic	Urinalysis		0	
1177	Valproic Acid	Blood	Blood	Venous Draw	(ALL)	0 Minutes	1 Months	(ALL)		0.3	SST (micro)	Chem		0	
1178	Valproic Acid	Blood	Blood	Venous Draw	(ALL)	31 Days	150 Years	(ALL)		0.3	SST	Chem	Draw prior to next dose	0	
1179	Valproic Acid, Free	Blood	Chem Red Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Red	Ref Refrig	Refrigerate	0	
1180	Vancomycin Peak	Serum	Peak	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		0.3	SST	Chem		0	
1181	Vancomycin Trough	Serum	Trough	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		0.3	SST	Chem	Draw prior to next dose	0	
1182	Varicella Antibody Immune Status IgG	Blood	VS	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	SST	Serology		0	
1183	Varicella Zoster IgM	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		2.0	SST	Ref Refrig	Refrigerate	0	
1184	Varicella Zoster IgM	Blood	Chem Sendout	Venous Draw	LabCorp	0 Minutes	150 Years	(ALL)		3.0	SST	Ref Room		0	
1185	Venous Blood Gas OR	Blood Heparin	Blood Gases Venous	Syringe	(ALL)	0 Minutes	150 Years	(ALL)		3.0	Hep Syringe	Chem		0	
1186	Viscosity, Serum	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		3.0	SST	Ref Refrig	Refrigerate	0	
1187	Vitamin A	Blood	Blood	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		0.3	SST	Ref Refrig	Protect from light	0	
1188	Vitamin B1 (Thiamine), Blood	Blood	Chem Sendout Frozen	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Lav	Ref Frozen	Protect from light	0	
1189	Vitamin B12	Blood	Chem E170	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	SST	SP Chem		0	
1190	Vitamin B6	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Lav	Ref Refrig	Protect from light	0	
1191	Vitamin C	Blood	Chem Sendout Frozen	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Red	Ref Frozen	Protect from light	0	
1192	Vitamin D 25 Hydroxy	Blood	Chem Red Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		2.0	Red	Ref Refrig	Refrigerate	0	
1193	Vitamin D 1 25 Dihydroxy	Blood	Chem Red Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		3.0	Red	Ref Refrig	Refrigerate	0	
1194	Vitamin E	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		0.3	Red	Ref Refrig	Protect from light	0	
1195	Vitamin K	Blood	Chem Sendout Frozen	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Lav	Ref Frozen	Protect from light	0	
1196	Von Willerbrand Factor Ag	Blood	Chem Sendout Frozen	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		2.0	Blue	Ref Frozen		0	
1197	WBC Man Diff	Blood	Blood Hem	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		0.5	Lav	Hem		0	
1198	West Nile Virus	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	SST	Ref Refrig		0	
1199	West Nile Virus Antibody, CSF	CSF	CSF	Syringe	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Sterile Screw	Ref Refrig		0	
1200	Whole Blood Glucose	Blood Heparin	Blood Gases Arterial	Syringe	(ALL)	0 Minutes	150 Years	(ALL)		0.1	Hep Syringe	Chem	Deliver Immediately	0	
1201	Whole Blood Lytes	Blood Heparin	Blood Gases Arterial	Arterial Draw	(ALL)	0 Minutes	150 Years	(ALL)		0.3	Hep Syringe	Chem	Deliver Immediately	0	
1202	XXXRapid Strep A	Throat Swab	Immunology	Swab	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Micro Swab	Serology		0	
1203	Xylose Tolerance 5 hr	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		5.0	Grey	Ref Refrig	Refrigerate	0	
1204	Xylose,Blood,1Hour Post	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		5.0	Grey	Ref Refrig	Refrigerate	0	
1205	Xylose,Blood,2Hour Post	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		5.0	Grey	Ref Refrig	Refrigerate	0	
1206	ZAP-70	Blood	Chem lav sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		5.0	Lav	Ref Room	Room Temperature	0	
1207	ZAP-70	Bone Marrow	Blood Heparin	Special Collection	(ALL)	0 Minutes	150 Years	(ALL)		3.0	Green	Ref Room		0	

	Primary Synonym	Specimen Type	Accession Class	Collection Method	Service Resource	Age From	Age To	Coll Priority	Aliq	Min Vol	Container	Collection Class	Special Handling	Add Lbl	Opt
1208	ZZCreat	Urine	Urine 24hr	Urine	(ALL)	0 Minutes	150 Years	(ALL)		1.0	U24H	Urine Chem	Refrigerate	0	
1209	Zinc Protoporphyrin (ZPP)	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Lav	Ref Refrig	Protect from light	0	
1210	Zinc Urine	Urine	Urine	Urine	(ALL)	0 Minutes	150 Years	(ALL)		10.0	U24H	Ref Room		0	
1211	Zinc, Plasma	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Royal (EDTA)	Ref Room	Room Temperature	0	
1212	Zonisamide (Zonegran)	Blood	Chem Red Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		2.0	Red	Ref Refrig	Refrigerate	0	
1213	pH Body Fluid	Body Fluid	Body Fluid	Syringe	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Hep Syringe	Chem		0	
1214	pH Stool	Stool	Stool	Special Collection	LabCorp	0 Minutes	150 Years	(ALL)		5.0	Stool Container	Ref Refrig	Refrigerate	0	
1215	pH Arterial Blood	Arterial	Blood	Syringe	(ALL)	0 Minutes	150 Years	(ALL)		3.0	Hep Syringe	Chem		0	
1216	pH Capillary Blood	Capillary - heelstick	Blood	Capillary	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Capillary	Chem		0	
1217	pH Renal Perfusate	Perfusate	Special Collectio	Syringe	(ALL)	0 Minutes	150 Years	(ALL)		0.5	Hep Syringe	Chem		0	
1218	pH Venous Blood	Blood	Blood	Syringe	(ALL)	0 Minutes	150 Years	(ALL)		3.0	Hep Syringe	Chem		0	
1219	wwABID Ref Lab	Blood	Blood Bank	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		6.0	Pink	BldBank		0	
1220	xxAnti GM1 IgG Titer	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	SST	Ref Room	Room Temperature	0	
1221	xxAnti GM1 IgM Titer	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	SST	Ref Room	Room Temperature	0	
1222	xxAnti-MAG Abs (Western Blot)	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	SST	Ref Room	Room Temperature	0	
1223	xxAnti-Ri Abs (Western Blot)	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	SST	Ref Room	Room Temperature	0	
1224	xxAnti-Yo Abs (Western Blot)	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	SST	Ref Room	Room Temperature	0	
1225	xxCyanide Confirmation, Blood	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		2.0	Grey	Ref Refrig	Refrigerate	0	
1226	xxExpedited Combo Retest	Blood	HIV Combo	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	SST	Chem		0	
1227	xxHBV PCR IU/mL	Blood	Chem Sendout Frozen	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		2.0	SST	Ref Frozen	Freeze	0	
1228	xxHBV PCR Copies/mL	Blood	Chem Sendout Frozen	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		2.0	SST	Ref Frozen	Freeze	0	
1229	xxHBV,DNA,QN,PCR W/REFL	Blood	Chem Sendout Frozen	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		2.0	SST	Ref Frozen	Freeze	0	
1230	xxHIV 1 Western Blot	Blood	VH	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	SST	HIV		0	
1231	xxHIV Combo Retest	Blood	HIV Combo	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	SST	Chem		0	
1232	xxHTLV I/II Immunoblot	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	SST	Ref Refrig		0	
1233	xxHU Ab Western Blot	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	SST	Ref Room	Room Temperature	0	
1234	xxHepatitis B DNA Quantitative	Blood	Chem Sendout Frozen	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		2.0	Lav	Ref Frozen	Freeze	0	
1235	xxHexagonal Phase Phospholipid	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		4.0	SST	Ref Refrig		0	
1236	xxKt/V	Blood	Post	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	SST	Chem		0	
1237	xxLym IGG, IGM, WB	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		3.0	SST	Ref Refrig		0	
1238	xxPTT-LA Incub Mix	Blood	Hem Blue Sendouts	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		3.0	Blue	Ref Refrig		0	
1239	xxPTT-LA Mix	Blood	Hem Blue Sendouts	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		3.0	Blue	Ref Refrig		0	
1240	xxRMSF, IgG, IFA	Blood	Chem Sendout	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		3.0	SST	Ref Refrig		0	
1241	xxdRVVT Confirm	Blood	Hem Blue Sendouts	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		1.0	Blue	Ref Refrig		0	
1242	xxdRVVT Mix	Blood	Hem Blue Sendouts	Venous Draw	(ALL)	0 Minutes	150 Years	(ALL)		3.0	Blue	Ref Refrig		0	